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Nome comercial: FRESA

EN Technical Name: DENTAL SURGICAL DRILLS / **Trade Name:** DRILL

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EN ATTENTION!!!

DO NOT USE IF THE PACK HAS BEEN TAMPERED WITH OR DAMAGED. NON-STERILE PRODUCT. STERILIZE BEFORE USE. REUSABLE. CLEAN AND STERILIZE BEFORE REUSE.

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- 01 Fresa

EN Commercial Presentation Forms of the Product: The Drills are supplied in individual packs. The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

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EN Indication of Use / Purpose: The drills are used in dental surgeries in general.

The drills are tools that have a cutting edge, which through rotating movements (controlled by the counter-angle or manually) perforate the bone structure of the patient.

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EN Mode of Product Use: For the surgical procedure, remove the drill from the instrument box, adapt it to the counter-angle or digital driver through the rod, and then check the lock of the drill on the counter-angle. With the drill in movement (varying from 200 to 600 RPM, depending on the purpose) start drilling.

After reaching the desired depth, remove the counter-angle-Drill assembly from the bone cavity.

Unlock the Drill and remove it from the counter-angle. They should be previously hygienized and sterilized according to the techniques below:

a) Cleaning: The cleaning of a surgical instrument (drill) that was in contact with tissues should be performed such that all parts of the item are exposed to the cleaning process. The cleaning process must involve an initial rinse of the item with cold water, then wash with a neutral to slightly basic detergent with low foam formation, rinse again with water (tap water or stored water) followed by a final rinse with deionized or purified water by reverse osmosis.

The initial cold rinse is performed to remove any coarse contamination present on the item. Cold water is used since temperatures above 60°C will clot proteins, making it difficult to remove them from the contaminated items. After rinsing with cold water, the items must be cleaned with a neutral to slightly basic detergent with low foam formation, with pH between 7.0 and 10.0.

If the contamination contains heavy or organic residues (protein or fat) or dry dirt, an enzymatic detergent can be used. The detergent used must be easy to rinse and must not leave residual compounds. A detergent with low foam formation reduces the formation of aerosol droplets, which may carry microorganisms. Detergent residues may cause stains and interfere in the action of chemical disinfectants. The washed items must be rinsed to remove the detergent, preferably with one final rinse in deionized or purified water by reverse osmosis.

The cleaning process must be conducted such that all the parts of the surgical instruments (drill) are exposed. The remaining contamination on an item after cleaning reduces the efficacy of any decontamination or subsequent sterilization process.

Clean the instruments (drills) as soon as possible after use. Do not allow blood and residues to become dry on the instruments (drills). If cleaning needs to be postponed, place the set of instruments (drill) in a container covered with enzymatic solution or appropriate detergent in order to delay drying.

b) Sterilization: It is a procedure that aims to completely eliminate microorganisms like: viruses, bacteria, microbes, fungi (in plant or spore form). After the cleaning process, the instruments (drills) are sterilized in an autoclave using temperature and exposure time according to the method below: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The surgical instruments should be arranged inside the container (perforated on the cover and lower part), or tray or in the box as long as they are arranged in a way that they are prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation, that is, allow all the steam to come out first in order to complete the Drying Cycle by itself. Thoroughly and periodically clean the autoclave to remove dirt and excess rust (iron oxide).

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EN Composition: ASTM F899 Stainless Steel is used as raw material for the Drills.

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EN Storage Conditions: Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excessive humidity.

The drills should be stored clean, decontaminated and completely dry. The pack in which the items are sterilized can provide an effective barrier to maintain the sterility of the item. After use, they should be kept in a clean, dry and cool place.

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EN Transport Conditions: The drills should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

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EN Handling Conditions: The handling should be performed with the due cares to avoid falls and damages, it must be performed on shelves and counters, according to the activity that is being executed.

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EN Warnings: Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function. Check the functioning of the product before initiating surgery. It should not be used if it has anatomical changes.

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EN Precautions: When handling the instruments (drills), use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packed, instrument boxes (drills) do not maintain their sterility. The drills should be replaced as soon as the professional notices its reduced efficacy. Exclusive use by the dental professional.

Below is a list of potential complications:

- Injury or pain in the soft tissues, in case of incorrect use or breakage of the instruments.
- Infection, if the instrument is not conveniently clean and sterilized.
- Damages to the nerves or nearby organs as a result of slide or bad positioning of a damaged instrument.
- Damages caused by instruments used to screw, due to excess forces that occur during use.
- Increase in the time of operation due to dismounting of the instruments during surgery.

Due to the necessary mechanical functionalities, majority of the instruments (drills) are made in non-implantable materials. In case an instrument breaks, no fragment should remain inside the patient because it can cause post-operative complications, such as allergies, infections or complications of the biological kind, associated to the release of metallic components, possibly requiring future intervention.

In order to avoid environmental contamination or improper use of Implacil non-articulated cutters, when they are to be used, they should be disposed of in an appropriate place, that is, in a place for potentially contaminated materials.

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EN Warranty Term: IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product against any material or manufacturing defect that may appear on it within 365 days, counting from the date of purchase by the first buyer. The product will have indefinite validity if transported, stored and packed suitably according to this User Manual. Note: the warranty does not cover wear from product use.

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