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EN **ATTENTION!** Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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EN Carefully read all the instructions contained in this “User Manual” before using this product. Ensure that all the instructions are informed in this manual's content.

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EN GENERAL INFORMATION

The Implacil Dental Instruments are manufactured with precision under strict processing guidelines and regulations. They are designed to offer durability and enable reuse.

They are manufactured with a material that allows long period of use when manipulated appropriately. They are supplied clean and are not sterile and must be sterilized before use. After use, they must be properly cleaned, decontaminated, sterilized and stored.

The information below refers to cleaning, decontamination, precaution and manipulation, user responsibility, sterilization and storage of the surgical instruments in order to help ensure a longer life.

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This Manual was originally written in Portuguese.

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Nome comercial: Instrumentos Ortodónticos Implacil

EN Technical name: Instruments for dental use

Trade Name: Implacil Dental Instruments

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EN Applicable to the following commercial models:

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EN Stainless Steel ASTM 899 and Stainless Steel 316L are used as raw material for the Implacil Dental Instruments.

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EN Material in conformity with the standards:

- NBR 13911 - Surgical instruments - Metal material - Specifications for dowels, bars and wires of stainless steel.
- ASTM F 899 - Standard Specification for Wrought Stainless Steels for Surgical Instruments.

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EN Indication: The Implacil Dental Instruments used in dental surgery in general.

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EN Special conditions for transport, storage and containerization of the product: Keep it in the original package protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35°C and humidity between 45% and 80% in order to ensure the stability of the packaging.

The instruments should be transported in a clean and dry place and at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN Instructions for product use: The Surgical Instruments should be used to assist in dental surgery. They should be sanitized and sterilized beforehand according to the following techniques:

A) Cleaning: The cleaning of a surgical instrument that has been in contact with tissues should be executed in such a way that all the parts of the item are exposed to the cleaning process. This process should involve an initial rinse of the item with cold water, washing with neutral to slightly alkaline detergent, with low foam formation, a rinse with water (from the faucet or conditioned) followed by a final rinse with deionized or reverse osmosis purified water. The initial cold rinse is performed to remove any gross contamination present in the item. Cold water is used, since temperatures above 140°F will coagulate proteins, making it difficult to remove them from the contaminated items. After the cold water rinse, the items should be cleaned with a neutral to slightly alkaline detergent with low foam formation, and pH between 7.0 and 10.0. If the contamination contains heavy organic residues (protein or fat) or dry dirt, an enzymatic detergent may be used. The detergent used should also be easy to rinse and not leave residual compounds. A detergent with low foam formation reduces the formation of aerosol droplets, which may transport microorganisms. Detergent residues can cause stains and interfere in the action of chemical disinfectants. The washed items should be rinsed to remove the detergent, preferably with a final rinse in deionized or reverse osmosis purified water. The cleaning process should be executed in such a way that all the parts of the surgical instrument are exposed. Contamination remaining in an item after cleaning reduces the efficacy of any subsequent decontamination or sterilization process.

Clean the instruments as soon as possible after use. Do not allow blood and residues to dry on the instruments. If the cleaning has to be postponed, place groups of instruments in a container covered with enzymatic solution or appropriate detergent in order to delay drying.

B) Sterilization: Sterilization is a procedure designed for total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments are sent for sterilization in an autoclave, using temperature and exposure time according to the following method: sterilization temperature: 132°C, sterilization cycle: 20 minutes and drying cycle: 20 - 60 minutes. The surgical instruments should be arranged inside the container (with perforated lid and bottom), or tray or in the box; provided they are accommodated in such a way as to allow their preparation for the sterilization process. DO NOT OPEN the autoclave to avoid rapid condensation; in other words, let all the steam out first, so that the Drying Cycle terminates on its own. Clean the autoclave rigorously and periodically, removing the dirt and excess rust formed (iron oxide).

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EN Warnings and or precautions to be adopted: When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility. The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient. Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes. Extreme care must be taken when handling the instruments to prevent injuries. The "instrument set" are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

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EN Undesirable secondary or side effects and contraindications: Incorrect cleaning and maintenance may render the instruments inappropriate for their intended use, provoke corrosion, disassembly, distortion and/or breakage or also cause injuries to the patient or operation team. See below a list of potential complications:

- Lesion or pain in the soft tissues in case of incorrect use or breakage of the instruments.
- Infection, if the instruments are not properly cleaned and sterilized.
- Damage to the adjacent nerves or organs, as a result of slipping or incorrect positioning of a damaged instrument.
- Damage caused by the instruments used for screwing due to excessive forces that occur during use.
- Increase of surgical time due to the need to dismantle the instruments during surgery.
- Due to necessary mechanical functionalities, most instruments are made from non-implantable materials. In the eventuality of an instrument breaking, no fragment should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metal components, possibly requiring subsequent intervention.

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EN Precautions in case of disposal: To avoid environmental contamination or improper use of the Surgical Instruments, these must be disposed of in an appropriate place, that is, in a specific place for potentially contaminated materials.

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EN Product warranty: IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO LTDA. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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