IFU-CE-004 Pilar Protético

Prosthetic Abutment / Pilar Protético / Moncone Protesico

ATENÇÃO !!!

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Fabricante recomenda o uso único.

ATTENTION!!!

Do not use if packaging is opened or damaged.

Non sterile product.

Reutilization is not recommended.

Single use recommended by the manufacturer.

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Product: “UN II Morse Taper Neck”:

Description: Implacil Morse Taper Neck Abutment is straight, cylindrical, and is made from commercially pure titanium grade 2, according to ASTM-F67 Standard. The abutments are of single use with collars (a) of 3.5 mm or 5.5 mm. It shows Morse Taper fitting, with 11o angulation in its lower base (b) to fit the implant used, working as a non rotation feature. Its 3.3 mm and 4.5 mm diameters are compatible with all commercial Morse Taper implant diameters. It is used in screwed or cemented prosthesis, especially for single unit restorations.

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Products: “Angled Neck UN II Morse Taper”

Description: Implacil Angled Neck Abutment is a 15o or 25oangle tapered abutment, made from commercially pure titanium grade 4, according to ASTM-F67 Standard. It comes with a fixing screw which is made from titanium grade 5, according to ASTM-

F136 Standard. The abutments are of single use with total height (a) of 11.5 mm and presented with collars (a) of 0,8mm; 1,50mm; 2,50mm and 3,50mm. Its lower base (b) shows Morse Taper to fit the implant used, working as a non rotation feature. Its diameters are related to the setting platforms of commercial implants: 3.5 mm; and 4.5 mm. They are used in cemented prosthesis, single- or multi-unit, when angle correction is needed.

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Products "Mini Tapered UN II Morse Taper":

Description: Implacil Mini Tapered Abutment is made from commercially pure titanium grade 4, according to ASTM-F67 Standard, presented with collars (a) of 0,8mm; 1,50mm; 2,50mm; 3,50mm; 4,50mm and 5,50mm and comes with a screw to fix the abutment to the implant. The screw is made from titanium grade 5, according to ASTM-F136 Standard. The abutments are of single use. Its upper base (b) has a 3.30 mm hex. Their lower base (c) has Morse Taper to fit the implant used, working as a non rotation feature. Their diameters are: 3,5mm and 4,5mm. They are used in single or multi-unit screwed prosthesis, especially in aesthetic zone, when inter-occlusion space is limited (minimum height of 4.5 mm).

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Product: “Aesthetic Morse Taper Adapter”:

Description: Implacil's Aesthetic Morse Taper Adapter Abutment is straight, cylindrical, and is made from commercially pure titanium grade 2, according to ASTM-F67 Standard. The abutments are of single use with collars (a) of 4 mm or 6 mm. It shows Morse Taper fitting, with 11° angulation in its lower base (b) to fit the implant used, working as a non rotation feature. Its 3.5 mm diameter is compatible with all commercial Morse Taper implant diameters. It is used in screwed or cemented prosthesis, especially for single unit restorations where the implant is in the height of the cortical.

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Product: "O'ring UN II" Morse Taper:

Description: Implacil O'Ring Abutment is straight, cylindrical, made from commercially pure titanium grade 2, according to ASTM-F67 Standard. The abutments are of single use with collars (a) of 0,8 mm; 1,5 mm; 2,5 mm; 4,5 mm and 5,5 mm. Its upper base (c) shows a 2.5 mm diameter hex; and on the top it shows a 2.0 mm diameter sphere (c). Its lower base (d) shows an morse taper fitting, to fit the implant used. It is used in overdentures.

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Product: "Mini Tapered Angled Neck UN II Morse Taper":

Description: Implacil Angled Neck Abutment is a 17° or 30° angle tapered abutment, made from commercially pure titanium grade V, according to ASTM-F136 Standard. The abutments are of single use with collars (a) of 0,8mm; 1,50mm; 2,50mm and 3,50mm. Its lower base (b) shows Morse Taper to fit the implant used, working as a non rotation feature. Its diameters is 4,8 mm. They are used in cemented prosthesis, single- or multi-unit, when angle correction is needed.

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Commercial Models "Tapered Aesthetic Straight UN II HI" and "Tapered Aesthetic Straight UN II HE"

A) Product description: Implacil Tapered Aesthetic Straight Abutment consists of a straight, tapered abutment made from commercially pure grade-2 titanium accordance with standard ASTM-F67, and a screw for (a) fixing the abutment to the implant, made of grade-5 titanium in accordance with standard ASTM-F136. The abutments are single-use pieces presented with collars of (b) 1.0mm, 2.0mm and 3.0mm. Its upper base (c) shows a 3.30mm hex. Its lower base (d) shows an internal or external hex that works as a non-rotational feature and is adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm, 4.0mm and 5.0mm. This abutment is used in the final preparation of screwed prosthesis elements, both unitary or multiple, especially in aesthetic zones.

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Accompanying parts and accessories: This product is individually packaged. Each package contains: a titanium prosthetic abutment and a titanium fixation screw.

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Raw-material: The raw materials used are commercially pure titanium ASTM F-67 for the Tapered Aesthetic Straight Abutment and titanium ASTM F136 for the Fixation Screw.

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Commercial Models “Tapered Aesthetic Angled UN II HI” and “Tapered Aesthetic Angled UN II HE”

A) Product description: Implacil Tapered Aesthetic Angled Abutment consists of a tapered abutment with an angle of 17º or 30º, made from commercially pure grade 2 titanium in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, manufactured in grade 5 titanium in accordance with standard ASTM-F136. The abutments are single use pieces presented with collars of (a) 1.0mm, 2.0mm and 3.0mm. Its lower base has an internal or external hex (b) that works as a non-rotational feature and is adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm, 4.0mm and 5.0mm. This abutment is used in the final preparation of screwed prosthesis elements, both unitary or multiple, especially in aesthetic zones where angle correction is required. Several different horizontal positions are possible.

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Raw-material: The raw materials used are commercially pure titanium ASTM F-67 for the Tapered Aesthetic Angled Abutment and titanium ASTM F136 for the Fixation Screw.

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Commercial Models “Mini Tapered UN II HI” and “Mini Tapered UN II HE”

A) Description: Implacil Mini Tapered Abutment consists of a straight tapered abutment made from commercially pure grade 2 titanium in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, manufactured in grade 5 titanium in accordance with standard ASTM-F136. The abutments are single pieces presented with collars (a) of 1.0mm; 2.0mm and 3.0mm. Its upper base (b) has a 3.30mm hex. Its lower base (c) has an internal or external hex that works as a non-rotational feature and is adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the final preparation of screwed prosthesis elements, both unitary or multiple, especially in aesthetic zones, in case of limited interocclusal space (minimum height of 4.5mm).

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Raw Material: The raw materials used are commercially pure titanium ASTM F-67 for the Mini Tapered Abutment and titanium ASTM F136 for the Fixation screw.

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Commercial Models “Mini Tapered Rotational UN II HI” and “Mini Tapered Rotational UN II HE”

A) Description: Implacil Mini Tapered Rotational Abutment consists of a straight tapered abutment made from commercially pure grade 2 titanium, in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, manufactured in grade 5 titanium in accordance with standard ASTM-F136. The abutments are single pieces presented with collars (a) of 1.0mm; 2.0mm and 3.0mm. Its upper base (b) has a 3.30mm hex. Its lower base (c) has internal or external fitting adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the final preparation of screwed prosthesis elements, both unitary or multiple, especially in aesthetic zones, in case of limited interocclusal space (minimum height of 4.5mm).

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Raw Material: The raw materials used are commercially pure titanium ASTM F-67 for the Mini Tapered Abutment and Titanium ASTM F136 for the Fixation screw.

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Commercial Model “Mini Tapered UN II Angled Neck HI” and “Mini Tapered UN II Angled Neck HE”

A) **Description:** Implacil Mini Tapered Angled Neck Abutment consists of a 17º or 30º angle tapered abutment made from commercially pure grade 2 titanium in accordance with standard ASTM-F136. The abutments are single pieces with collars of 0.80mm, 1.50mm; 2.50mm; 3.50mm. It has an internal hex in its lower base that works as a non-rotational feature and is adaptable to the type of implant used. Its diameter is 4.8 mm. This abutment is used in the final preparation of screwed prosthesis elements, both unitary or multiple, where angle correction is needed.

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Accompanying parts and accessories:This product is individually packaged. Each package contains: a titanium prosthetic abutment and a titanium fixation screw.

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Commercial Model “Straight Rotational Neck UN II HI” and “Straight Rotational Neck UN II HE”

A) Description: Implacil Straight Rotational Neck consists of a straight cylindrical abutment made from commercially pure grade 2 titanium in accordance with standard ASTM-F67. The abutments are single pieces and have an overall height (a) of 11.5mm. Its lower base has internal or external fitting, adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the preparation of temporary prosthesis elements, both unitary or multiple, fixed with specific cement for dental procedures.

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Accompanying parts and accessories: This product is individually packaged. Each package contains a titanium prosthetic abutment.

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Raw Material: The raw material used for the Straight Rotational Neck is commercially pure titanium ASTM F-67.

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Commercial Models “Straight Anti-Rotational Collar UN II HI” and “Straight Anti-Rotational Collar UN II HE”

A) Description: Implacil Straight Anti-Rotational Collar consists of a straight cylindrical collar made from commercially pure grade 2 titanium, in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, made from pure grade 5 in accordance with standard ASTM-F136. The abutments are single pieces and have an overall height (a) of 11.5mm. Its lower base has an internal or external hex (b) that works as a non-rotational feature and is adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5,0mm. This abutment is used in the final preparation of cemented prosthesis elements, both unitary or multiple.

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Accompanying parts and accessories: This product is individually packaged. Each package contains a titanium prosthetic abutment and a titanium fixation screw.

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Raw Material: The raw materials used are commercially pure titanium ASTM F-67 for the Straight Anti-Rotational Collar and Titanium ASTM F136 for the Fixation screw.

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Commercial Models “Angled Neck UN II HI” and “Angled Neck UN II HE”

A) Description: Implacil Angled Neck Abutment consists of a 15º or 25º angled cylindrical abutment made from commercially pure grade 2 titanium, in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, manufactured in grade 5 titanium, in accordance with standard ASTM-F136. The abutments are single pieces and have an overall height (a) of 11.5mm. Its lower base has an internal or external base (b) that works as a non-rotational feature and is adaptable to the type of

implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the final preparation of cemented prosthesis elements, both unitary or multiple, where angle correction is needed.

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Commercial Models "O`ring UN II HI" and "O`ring UN II HE"

A) Description: Implacil O`ring abutment consists of a straight cylindrical abutment made from commercially pure grade 2 titanium in accordance with standard ASTM-F67. The abutments are single pieces, with collars (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm and 5.0mm. Its upper base (b) has a 2.5mm diameter hex with a (c) 2.0mm diameter sphere on top. Its lower base (d) has internal or external fitting, adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the final preparation of total overdenture prostheses.

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Accompanying parts and accessories: This product is individually packaged. Each package contains a titanium prosthetic abutment and a titanium coping.

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Commercial Models "Hexagonal UN II" and "Hexagonal HE"

A) Description: Implacil Hexagonal Abutment consists of a straight cylindrical abutment made from commercially pure grade 2 titanium, in accordance with standard ASTM-F67, and a screw for fixing the abutment to the implant, manufactured in grade 5 titanium, in accordance with standard ASTM-F136. The abutments are single pieces with collars (a) of 1.0mm; 2.0mm and 3.0mm. Its upper base (b) has a 3.30mm hex. Its lower base (c) has an internal or external hex that works as a non-rotational feature and is adaptable to the type of implant used. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is used in the final preparation of unitary prosthesis elements where a screw hole is not needed.

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Raw Material: The raw materials used are commercially pure titanium ASTM F-67 for the Hexagonal Abutment and Titanium ASTM F136 for the Fixation screw.

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Commercial Models “Ucla UN II HI” and “Ucla UN II HE”

A) Description: Implacil UCLA UNII HI Abutment consists of a straight cylindrical abutment made from Titanium ASTM F67. The abutments are single pieces with an overall height of 11.5mm. Its base has external fitting (a), adaptable to Implacil System implants with internal (HI) or external (HE) hex connection. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm and 5.0mm. This abutment is normally used in the preparation of cemented or screwed temporary prosthesis elements.

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Accompanying parts and accessories: This product is individually packaged. Each package contains a UCLA UN II HI made from Ti ASTM F67 and a screw made from Ti ASTM F136.

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Raw Material: The raw materials used are commercially pure titanium ASTM F-67 for the Ucla UN II Abutment and Titanium ASTM F136 for the Fixation screw.

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Commercial Models “Ucla UN II AR HI” and “Ucla UN II AR HE”

A) Description: Implacil UCLA UN II AR Abutment consists of a straight cylindrical abutment made from Titanium F67. The abutments are single pieces with an overall height of 11.5mm. Its base has a hexagonal external fitting (a), that prevents its rotation and is adaptable to all Implacil System implants with internal (HI) or external (HE) hexagonal connection. Its diameters are relative to commercial implants setting platforms: 3.5mm; 4.0mm; 5.0mm. This abutment is used in the preparation of cemented or screwed temporary prosthesis elements.

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Accompanying parts and accessories: This product is individually packaged. Each packaging consists of a Titanium F67 UCLA UN II AR and a Titanium F136 screw .

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Commercial Model "Mini Conical Conical"

A) Description: The Implacil Mini Conical Coffe consists of a straight cylindrical abutment made of titanium grade 4 commercially pure according to ASTM-F67 and a screw to fix the abutment in the implant, made of titanium alloy ASTM-F136. The pillars are single use, have total height of 12.5mm and diameter of 4.8mm. In its lower base has an internal fitting, with the function of adapting to the Prosthetics Pillars of the Mini Conical model of the Implacil System. It also has a retention shoulder (b) to support the prosthesis. It is used in the preparation of temporary prosthetic screw element.

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Parts and accessories: The product is individually packaged. Each package is composed of: a Mini Conical Hood and a Mini Conical Hood.

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Specifications and technical characteristics:

Installation torque: 15N.cm

Dimensions: Base Ø (mm) 4.8;

Retention: Yes;

Anti-Rotational: No.

Raw Material: As a raw material for the Mini Conical Coater, commercially pure Titanium ASTM F-67 is used and for the Fixing and Working Screw, ASTM F136 Titanium Alloy.

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Implacil Aesthetic Cone Coating consists of a straight cylindrical abutment, manufactured in commercial grade pure titanium grade 4, according to ASTM-F67, and by a screw to fix the abutment in the implant, made of titanium alloy ASTM-F136. The pillars are single use, have a total height of 14mm and a diameter of 4.8mm. In its lower base has an internal fitting, with the function

of adapting to the Prosthetic Pillars of the Esthetic Cone model of the Implacil System. It also has a retention shoulder (b) to support the prosthesis. It is used in the preparation of temporary prosthetic screw element.

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Parts and accessories: The product is individually packaged. Each package is composed of: a Aesthetic Cone Coating and a Aesthetic Cone Coating Hood.

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Specifications and technical characteristics:

Installation torque: 15N.cm

Dimensions: Base Ø (mm) 4.8;

Retention: Yes;

Anti-Rotational: No.

Raw Material: As a raw material for the Aesthetic Conical Coater is used commercially pure Titanium ASTM F-67 and for the Fixing and Working Screw, ASTM F136 Titanium Alloy.

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Physical principle and product technology fundamentals, applied to their operation and action indication: The Prosthetic Abutments are mechanical components that work as a link between dental implant and prosthesis, offering support to the latter, and can correct height and angle between the implant and the tooth. They are fixed to the implant with a screw. The prosthesis elements can be screwed or cemented to the abutments, and it is up to the professional to decide the best method to be used in each situation.

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Optional, consumables and support materials

Optionals: Not applicable.

Consumables: Not applicable.

Manufacturing items for exclusive use of the company's products: All pieces described in these Usage Instructions are to be used exclusively on the Prosthetic Abutment.

Attention: The use of any piece, accessory or material not specified or described in these Usage Instructions is of the entire responsibility of the user.

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Special conditions for product transportation, storage and packaging: Store at room temperature, in a dry place, away from light and heat. Avoid exposition to excessive heat and humidity. Do not subject the packaging to excessive weight to avoid compromising the sealing that ensures the product's sterility. Implacil De Bortoli Prosthetic Abutments must be transported in clean, dry places at room temperature. Transportation must be carried out properly to make sure the original packaging do not fall or get damaged and to avoid mechanical shocks that might damage the product.

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Directions for product usage Implacil Prosthetic Abutments are packaged in aluminum sachets and a secondary packaging. Check to see if the packaging is in perfect conditions. The product must be rejected if packaging is damaged.

- Open the exterior packaging. Remove the sachet from the packaging and check if it is perfectly sealed.
- Remove the abutment from the sachet. The product is non-sterile. The sterilization process should be carried out by the professional according to the existing laws (NBR 14332) or any legislation or existing law that complements or supersedes it.
- Screw the abutment to the implant.
- Check the maximum torque to be applied in model Description (item C in these Usage Instructions).
- The abutments are previously sanitized in a classified room. If you feel the need to sanitize them again, follow the steps below:
- Rinse the item with cold water.
- Wash it with a neutral to slightly basic detergent.
- Rinse with de-ionized or purified water by reverse osmosis.
- Sterilization is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (whether in vegetative or sporulated form). After the drying process, the Prosthetic Pillars, placed in the surgical envelope, are sent to the autoclave, using temperature and exposure time, according to the following method: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 - 60 minutes. DO NOT open autoclave to prevent rapid condensation; ie let all the steam come out first, making the Drying Cycle complete on its own. Thoroughly and periodically clean the autoclave, removing dirt and excess rust formed (iron oxide).

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Warnings and/or precautions to be adopted Implacil Prosthetic Abutments can only be used by specialized professionals in the area. It is necessary to elaborate a prosthetic execution plan, observing the right diameter, angle and length for the adaptation of the prosthetic component to the implant in order to avoid future complications. Implacil Prosthetic Abutments were designed for single use, the risk of reuse is biological contamination and their reprocessing is prohibited. Do not use it in any way if the packaging is violated. The product is not sterile, so a sterilization procedure must be performed by the professional before any intervention on the patient. The professional must observe the pressure exerted in the moment of application so as to not damage the item, and inform the patient about the necessity of periodic monitoring and avoiding physical and mechanical efforts in order not to submit the product to undue efforts, as well as the necessity of proper hygiene.

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Indication, purpose or use for the product

The abutments are indicated for the final preparation of single or multiple prosthetic elements, where functional and/or aesthetic rehabilitation is needed, and are fixed by a screw or dental procedure-specific cement. Any corrections needed between implant and tooth axis can be performed in relation to its vertical dimension or angle.

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Undesirable secondary or side effect and contraindications

The product application final result may be unsuccessful if an inadequate technique is used or if the patient is not subjected to the proper examinations and prosthetic planning. In these cases the product might be lost or fractured. Incorrect application of the product may cause side effects in the place where it was applied, such as pain, swelling, sensitivity, tissue reaction and infection. Implacil Prosthetic Abutments are not indicated in cases of patients with chronic periodontal inflammation, inadequate oral hygiene or parafunctional habits, active intra-oral infection or in cases of immediate load where the primary stability of the implant is not adequate.

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Product safety and efficacy Implacil Prosthetic Abutments are completely safe as long as the safety rules are observed in all procedures performed. The product was developed in non-toxic material for odontologic use. The product's characteristics will not change as long as it's stored and kept according to its usage indications.

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Installation or connection to other products Implacil Prosthetic Abutments are connected to previously installed and osseointegrated implant, according to the models compatible to each type of connection. The use of any piece, accessory or material not specified or described in these instructions is the entire responsibility of the user.

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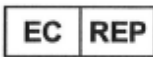
Concerns in case of product disposal In order to avoid environmental contamination or improper use of the Prosthetic Abutments, the ones not used must be discarded in hospital waste or any specific location for potentially contaminated materials.

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Term of Warranty IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO LTDA. assures the owner of this product warranty against any material or manufacture defects, if presented in a period of 365 days, from the date of acquisition by the first acquirer. If properly transported, stored or packaged according with this Usage Instructions, the product has indeterminate expiry date.

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Instruction of Use for all countries except Brazil

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