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*Instructions for Use / Instrucciones de Uso / Istruzioni per l'uso*

### **PARABÉNS !**

### **CONGRATULATIONS! / ¡FELICIDADES! / CONGRATULAZIONI!**

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**EN** For greater safety:

- a) Carefully read all the instructions contained herein before using this product.
- b) Ensure that all the instructions are informed in this manual's content.

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**EN** This Manual was originally written in Portuguese.

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**EN** 1.1 Commercial Models “Tapered Aesthetic Straight UN II HI” and “Tapered Aesthetic Straight UN II HE”

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**EN** 1.2 Commercial Models “Tapered Aesthetic Angled UN II HI” and “Tapered Aesthetic Angled UN II HE”

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**EN 1.14 Commercial Models “Tapered Aesthetic Coping”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.15 Commercial Models “Straight AR Abutment with collar UNII HE” and “Straight AR Abutment with collar UNII HI”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.16 Commercial Models “Mini Tapered Fit UNII HE” and “Mini Tapered Fit UNII HI”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.17 Commercial Models “Mini Tapered Coping AR”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.18 Commercial Models “Taper Aesthetic Coping AR”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.23 Commercial Models “T-Base Micro Tapered Abutment”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 1.24 Commercial Models “Classic Abutment”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN** 9. Safety and efficacy of the product

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**EN** 10. Installation or connection to other products

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**EN** 11. Concerns in case of disposal of the product

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**EN** **Technical name:** Prosthetic Abutment / **Trade Name:** Prosthetic Abutment

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**EN A) Product description**

The Implacil Tapered Aesthetic Straight Abutment consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use, presented with collar heights of 1.0mm, 2.0mm, and 3.0mm. Its upper base has a hexagon of 3.30mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the screwed prosthetic multiple-teeth element, especially in the aesthetic zone.

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**EN C) Technical specifications and features**

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Tapered Aesthetic Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN 1.2 - Commercial Models** “Tapered Aesthetic Angled UN II HI” and “Tapered Aesthetic Angled UN II HE”

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**EN A) Description**

The Implacil Tapered Aesthetic Angled Abutment consists of a tapered abutment with angulation of 17° or 30°, manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use, presented with collar heights of 1.0mm, 2.0mm, and 3.0mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final manufacture of screwed, single or multiple prosthetic elements, especially in aesthetic zone, when there is need for angulation correction, enabling different horizontal rotational positions.

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The product is packaged individually  
Each pack consists of:

- one titanium prosthetic abutment
- one titanium alloy fixation screw

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Tapered Aesthetic Angled Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

The Implacil Mini Tapered Abutment consists of a tapered straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use, presented with collar heights of 1.0mm, 2.0mm, and 3.0mm. Its upper base has a hexagon of 3.30mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, especially in the aesthetic zone, in cases with limited interocclusal space (minimum height of 4.5 mm).

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Mini Tapered Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

The Implacil Mini Tapered Abutment consists of a tapered straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use, presented with collar heights of 1.0mm, 2.0mm, and 3.0mm. Its upper base has a hexagon of 3.30mm. It has an inner or outer fitting on its lower base (b) that is used to adapt it to the type of implant. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, especially in the aesthetic zone, in cases with limited interocclusal space (minimum height of 4.5 mm).

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#### EN A) Description

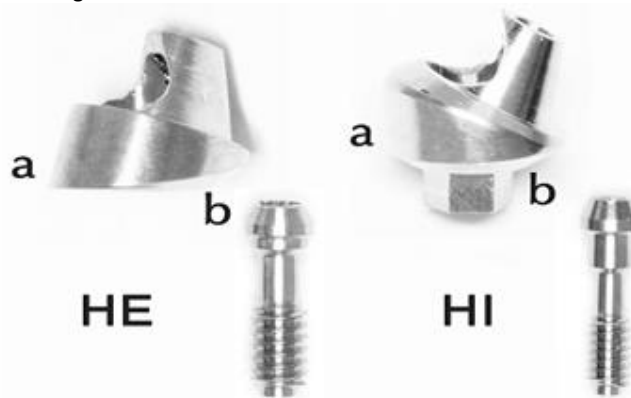
The Implacil Mini Tapered Angled Abutment consists of a straight tapered abutment with angulation of 17° or 30°, manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use and have collars of 1.00mm, 2.00mm and 3.00mm. It has an Internal Hexagon on its lower base that is used to adapt it to the implant and make it anti-rotational. It has a diameter of 4.8 mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, where angulation correction is necessary.

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#### EN B) Accompanying parts and accessories

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Mini Tapered Abutment UN II Angled HE and HI and Titanium ASTM F136 Alloy is used for the Fixation screw.

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**EN 1.6 – Commercial Model** “Straight Rotational Abutment UN II HI” and “Straight Rotational Abutment UN II HE”

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#### EN A) Description

The Implacil Straight Rotational Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use and have a total height (a) of 11.5 mm. It has an inner or outer fitting on its lower base that is used to adapt it to the type of implant to be used. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in making the temporary, single-tooth or multiple-teeth prosthetic element, fixed with cement specific for the dental procedure.

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**EN Raw Material:** The raw material used for the Straight Rotational Abutment is commercially pure Titanium ASTM F-67.

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**EN 1.7 - Commercial Models** “Straight Anti-Rotational Abutment UN II HI” and “Straight Anti-Rotational Abutment UN II HE”

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#### EN A) Description

The Implacil Straight Anti-Rotational Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use and have a total height of 11.5 mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the cemented prosthetic element, single-tooth or multiple-teeth.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Straight Anti-Rotational Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

The Implacil Angled Abutment consists of a cylindrical abutment with angulation of 15° or 25°, manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use and have a total height of 11.5 mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.3mm, 4.0mm and 5.0mm. It is used in the final production of the single-tooth or multiple-teeth cemented prosthetic element, where angulation correction is necessary.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Angled Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

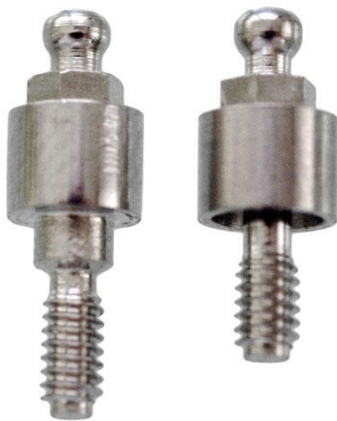
The Implacil O`ring Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The Copping is manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use, available with collar heights of 1.0mm, 2.0mm, 3.0mm, 4.0mm and 5.0mm. Its upper base has a hexagon with 2.5 mm in diameter, and over it is a ball of 2.0 mm in diameter. It has an inner or outer fitting on its lower base that is used to adapt it to the type of implant. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the complete *overdenture* prosthesis.

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Each pack consists of:

- one titanium prosthetic abutment
- one titanium coping

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**EN Raw Material:** The raw material used for the O´ring Abutment and for the Copping is commercially pure Titanium ASTM F-67.

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**EN A) Description**

The Implacil Hexagonal Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use, presented with collar heights of 1.0mm, 2.0mm, and 3.0mm. Its upper base has a hexagon of 3.30mm. It has an inner or outer hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of aesthetic single-tooth prosthetic element in which no hole for the screw is desired.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Hexagonal Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

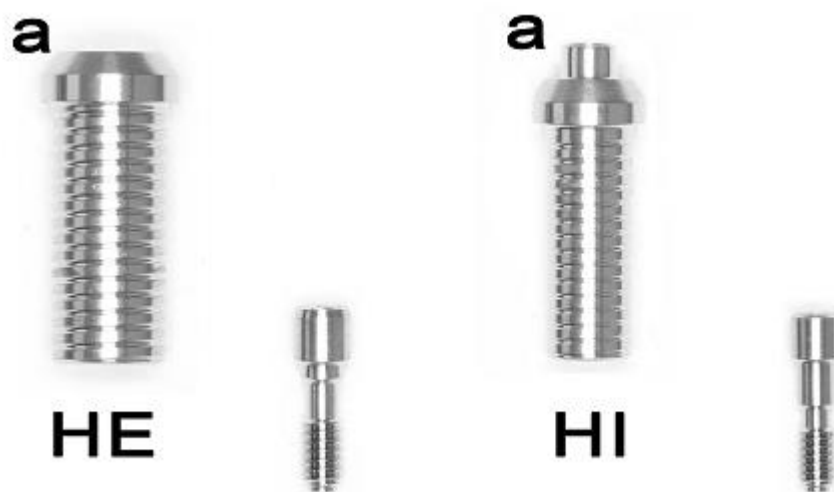
The Implacil UCLA UNII HI and HE Abutments consist of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use and have a total height of 11.5 mm. It is produced with an outer fitting on the base that is used to adapt it to the implants of the Implacil System with inner hexagonal fitting (HI) or outer (HE) hexagonal fitting. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is commonly used in the production of the temporary cemented or screwed prosthetic element.

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Each pack consists of:

- one UCLA in Ti ASTM F67
- a screw in titanium ASTM F136 alloy

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Ucla UN II Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

The Implacil UCLA UNII AR Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use and have a total height of 11.5 mm. It is produced with an outer or inner fitting on the base in the shape of a hexagon that is used avoid its rotation as well as adapt it to the implants of the Implacil System with inner hexagon fitting (HI) or outer hexagon fitting (HE). Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm, 5.0mm. It is used in the production of the temporary cemented or screwed prosthetic element.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Ucla UN II AR Abutment and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Description**

The Implacil **Mini Tapered Abutment** consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 12.5mm and diameter of 4.8mm. It has an inner fitting on its lower base that is used to adapt it to the Prosthetic Abutments of the Mini Tapered model of the Implacil System. It also has a retention shoulder to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

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**Dimensões**

**EN C) Technical specifications and features** Placement torque: 15N.cm

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Mini Tapered Coping and Titanium ASTM F136 alloy is used for the fixation and working screw.

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**EN A) Description**

The Implacil **Taper Aesthetic Abutment** consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 14mm and diameter of 4.8mm. It has an inner fitting on its lower base that is used to adapt it to the Taper Aesthetic model of Prosthetic Abutments of the Implacil System. It also has a retention shoulder to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

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**EN B) Accompanying parts and accessories**

The product is packaged individually

Each pack consists of:

- one titanium Tapered Aesthetic Coping
- one titanium alloy fixation screw
- one titanium alloy working screw

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**EN C) Technical specifications and features Placement torque:** 15N.cm

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Tapered Aesthetic Coping and Titanium ASTM F136 alloy is used for the fixation and working screw.

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**EN 1.15 - Commercial Models** “Straight Abutment AR with collar UNII HE” and “Straight Abutment AR with collar UNII HI”

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**EN A) Description**

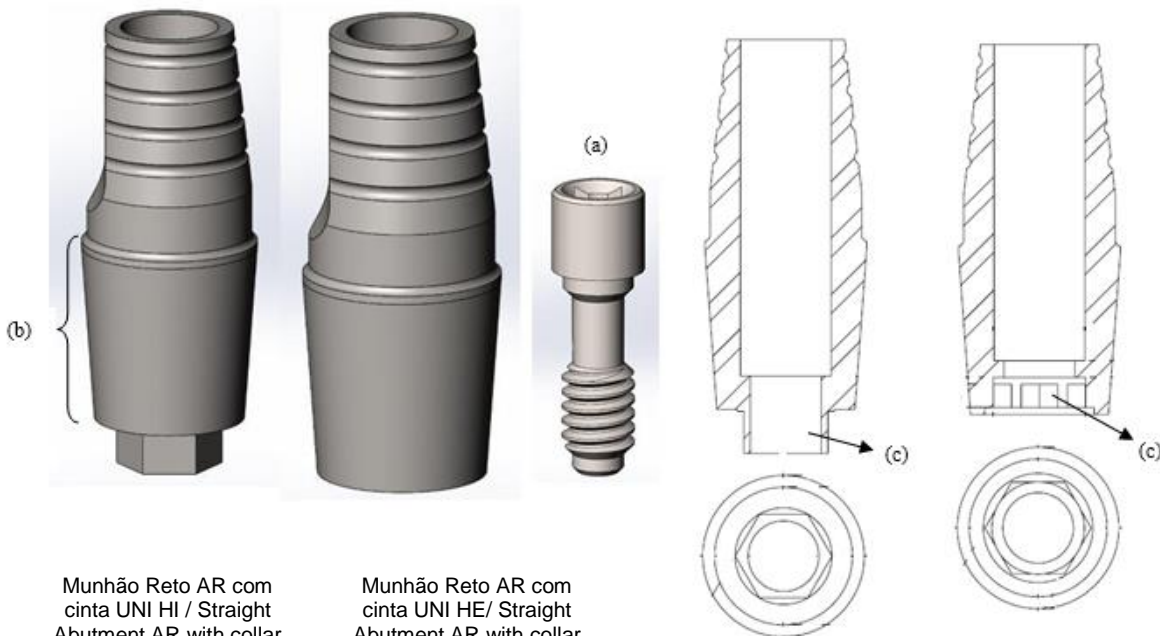
The Implacil Straight Abutment AR with collar consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are for single use and have a collar height of 1.0, 2.0, 3.0, 4.0 and 5.0. It has an internal or external hexagon on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm, 4.0mm and 5.0mm. It is used in the final production of the cemented prosthetic element, single-tooth or multiple-teeth.

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**EN B) Accompanying parts and accessories**

The product is packaged individually

Each pack consists of:

- one titanium Straight Abutment AR with collar UNII HE or one titanium Straight Abutment AR with collar UNII HI
- a screw in titanium ASTM F136 alloy

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the "Straight Abutment AR with collar UNII HE" and "Straight Abutment AR with collar UNII HI" and Titanium ASTM F136 is used for the Fixation screw.

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**EN 1.16 - Commercial Models** “Mini Tapered Fit UNII HE” and “Mini Tapered Fit UNII HI”

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**EN A) Description**

The Implacil Mini Tapered Fit consists of a straight tapered abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use, available with collar (a) heights of 1.0mm, 2.0mm, 3.0mm, 4.0mm and 5.0mm. It has an inner or outer hexagon on its lower base (b) that is used to adapt it to the type of implant. Its diameters are related to the seating platforms of the commercially available implants: 3.5mm and 4.0mm. It is used in the final production of the screwed prosthetic element, multiple-teeth, especially in the aesthetic zone, in cases with limited interocclusal space.

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**EN C) Technical specifications and features Placement torque: 20N.cm**

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**EN Raw Material:** Commercially pure Titanium ASTM F-67 is used as raw material for the “Mini Tapered Fit” and “Mini Tapered Fit UNII HI” abutment.

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**EN A) Description**

The Implacil **Mini Tapered Abutment** consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 12.5mm and diameter of 4.8mm. There is an internal fitting on its lower base in hexagonal shape (a) to avoid its rotation, and to adapt to the Prosthetic Abutments of the Mini Tapered model of the Implacil System. It also has a retention shoulder to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Mini Tapered Coping AR and Titanium ASTM F136 alloy is used for the fixation and working screw.

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**EN A) Description**

The Implacil **Taper Aesthetic Coping** consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 14mm and diameter of 4.8mm. There is an internal fitting on its lower base in hexagonal shape (a) to avoid its rotation, and to adapt to the Prosthetic Abutments of the Taper Aesthetic model of the Implacil System. It also has a retention shoulder to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

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The product is packaged individually. Each pack consists of:

- one titanium Taper Aesthetic Coping
- one titanium alloy fixation screw
- one titanium alloy working screw

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**EN C) Technical specifications and features Placement torque: 15N.cm**

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Taper Aesthetic Coping AR and Titanium ASTM F136 alloy is used for the fixation and working screw.

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**EN 1.19 Commercial Models** “Mini Tapered Coping SP” and “Mini Tapered Coping SP AR”

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**EN A) Product description**

The Implacil “Mini Tapered Coping SP” and “Mini Tapered Coping SP AR” consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 12.5mm and diameter of 4.8mm. The Mini Tapered Coping SP (a) has an internal fitting on its lower base, the Mini Tapered Coping SP AR (b) has an internal fitting on its lower base, with hexagonal shape to avoid their rotation, both used to adapt to the Prosthetic Abutments of the Mini Tapered model of the Implacil System. It also has a retention shoulder (c) to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

These Coping models do not have working screw.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Mini Tapered Coping SP and Mini Tapered Coping SP AR Titanium ASTM F136 is used for the Fixation screw.

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**EN** 1.20 Commercial Models “Taper Aesthetic Coping SP” and “Taper Aesthetic Coping SP AR”

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#### **EN** A) Product description

The Implacil “Taper Aesthetic Coping SP” and “Taper Aesthetic SP AR” consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw of the abutment on the implant, manufactured in titanium ASTM-F136 alloy. The abutments are of single use and have a total height of 14mm and diameter of 4.8mm. The Taper Aesthetic Coping SP (a) has an internal fitting on its lower base, the Taper Aesthetic Coping SP AR (b) has an internal fitting on its lower base, with hexagonal shape to avoid their rotation, both used to adapt to the Prosthetic Abutments of the Taper Aesthetic model of the Implacil System. It also has a retention shoulder (c) to support the prosthesis. It is used in the production of the temporary screwed prosthetic element.

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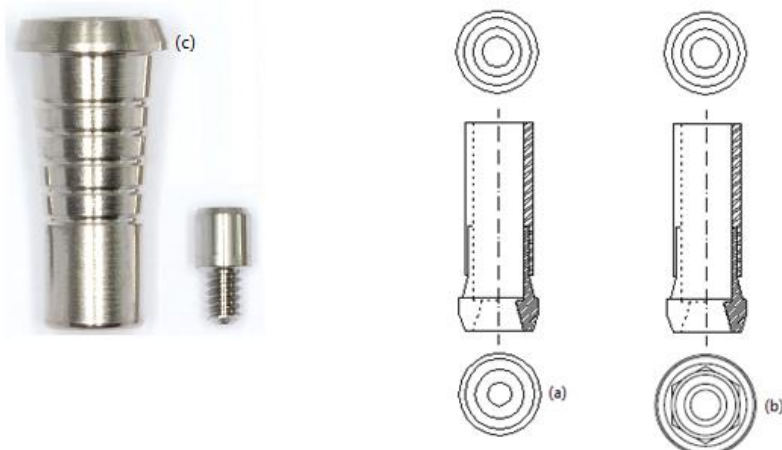
#### **EN** B) Accompanying parts and accessories

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- one titanium Taper Aesthetic Coping SP or one titanium Taper Aesthetic Coping SP AR.
- one titanium alloy fixation screw.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the Taper Aesthetic Coping SP and Taper Aesthetic Coping SP AR Titanium ASTM F136 is used for the Fixation screw.

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**1.21 Modelos Comerciais** “Pilar Base T UNII HE” e “Pilar Base T UNII HI”

**EN 1.21 Commercial Models** “T-Base Abutment UNII HE” and “T-Base Abutment UNII HI”

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**EN A) Product description**

The Implacil T-Base UNII HE and HI Abutments consist of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and are used to reference the scanning process for the production of the prosthesis on implant.

The lower base has an internal fitting (T-Base Abutment UNII HE) or external fitting (T-Base Abutment UNII HI) used to adapt to the type of implant to be used. Their diameters are related to the seating platforms of the commercially available implants: 3.5mm and 4.0mm. It is used in the manufacture of the single-tooth prosthetic element, attached with a fixation screw manufactured in titanium alloy grade 5 according to ASTM-F136.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the T-Base Abutments HE and HI and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Product description**

The Implacil T-Base Mini Tapered UNII HE and HI Abutments consist of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and are used to reference the scanning process for the production of the prosthesis on implant.

It has an inner fitting on its lower base that is used to adapt it to the mini tapered abutment. The diameter is 4.8 mm. It is used in the manufacture of the multiple-teeth prosthetic element, attached with a fixation screw manufactured in titanium alloy grade 5 according to ASTM-F136.

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**EN Raw Material:** Commercially pure titanium ASTM F-67 is used as raw material for the T-Base Mini Tapered Abutments and Titanium ASTM F136 alloy is used for the Fixation screw.

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**EN A) Product description**

The Implacil T-Base Micro Tapered Abutments consist of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and are used to reference the scanning process for the production of the prosthesis on implant.

It has an inner fitting on its lower base that is used to adapt it to the micro tapered abutment. The diameter is 3.5mm. It is used in the manufacture of the multiple-teeth prosthetic element, attached with a fixation screw manufactured in titanium alloy grade 5 according to ASTM-F136.

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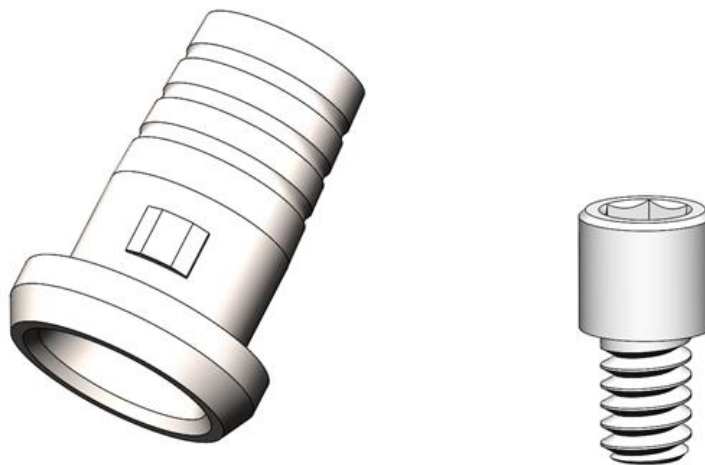
- one T-Base Micro Tapered Abutment
- one fixation screw

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**EN A) Product description**

The Implacil Classic Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are for single use and have a total height of 14.5mm. It has an outer fitting on its lower base that is used to adapt it to the type of implant to be used. Its diameters are related to the seating platforms of the commercially available implants: 3.25; 3.75; 4.25. It is used in making the temporary, single-tooth or multiple-teeth prosthetic element, fixed with cement specific for the dental procedure.

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**EN Raw Material:** The raw material used for the Classic Abutment is commercially pure Titanium ASTM F-67.

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**EN 2. Physical principle and technological fundamentals of the product, applied for its operation and its action:**

The Prosthetic Abutments are mechanical components that act as a connecting link between the implant and the dental prosthesis, providing support and able to perform height and angulation corrections between the implant and the tooth. They are threaded to the implants. The prosthetic elements can be attached to the abutments in cemented or screwed form, with the professional responsible for deciding on the best method to be used.

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**EN 4. Special transport, storage and packaging conditions of the product:**

Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excess humidity. Do not put excess load on the packaging to avoid damaging the seals that ensure the preservation of the product.

The Implacil De Bortoli Prosthetic Abutments should be transported in a clean dry place at room temperature. The transportation should be carried out in a suitable manner to avoid falls and damages to its original pack and mechanical impacts that can damage the product.

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Fabricante recomenda o uso único./

WARNING!!!

**EN** Do not use if the packaging is open or damaged.

Non-Sterile Product.

Reuse is not recommended.

Manufacturer recommends single use./

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**EN 5. Instructions for product use**

- The Implacil Prosthetic Abutments have been packed inside an aluminum sachet and a secondary packaging.
- Check if the pack is in perfect conditions. Reject the product if the pack is damaged.
- Open the outer packaging. Remove the sachet from the pack and check if it is perfectly sealed.
- Remove the abutment from inside the sachet.
- The product is non-sterile, and this procedure should be carried out by the professional according to the effective standards (NBR 14332), or any effective legislation or standard that complements or substitutes it.
- Screw the abutment to the implant.
- Check the maximum torque to be applied in the model description (item C of this Instruction for Use).

- The abutment is previously sanitized in a classified room. Follow the stages below if you wish to sanitize again:
- - Initial rinse of the item with cold water.
- - Wash with neutral to slightly basic detergent
- - Final rinse with deionized or purified water by reverse osmosis.
- The product is not sterile, and this procedure should be carried out by the professional according to the following stages:
- - Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The items should be arranged inside the container (perforated on the cover and lower part), or tray or in the box, or surgical envelop, provided they are arranged in a way that they can be prepared for the sterilization process. Do NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to allow the Drying Cycle to be completed on its own. Clean the autoclave thoroughly and periodically.

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**EN** The product is presented in individual pack.

Pack content:

1 Prosthetic Abutment HI / HE

3 Tracking Labels with product identification for obligatory attachment: on the clinical record, on the document to be submitted to the patient, and on the invoice documentation, according to the model below:

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**EN** This instruction for use will be available in electronic format on the Implacil De Bortoli website (<http://implacil.com.br/instrucoes-de-uso/>) according to the effective legislation on the availability of instruction for use of healthcare products in non-printed format – Normative Instruction IN No. 4 dated June 15, 2012. The printed instructions for use will be available to the customer sent free of charge (including sending) when requested from Customer Service. 0800-779-0011.

To locate it, check the registration number of the product at ANVISA and review on the Label of the product purchased.

Adobe Acrobat Reader is necessary in order to read the instructions for use on the website.

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**EN** Type of pack:

The material and type of pack used to package the Implacil De Bortoli Prosthetic Abutments are widely known and of universal use, for both drugs and healthcare products and they are available in the ANVISA Packaging Table.

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**EN Characteristics and dimensions of the pack:**

Primary packaging: Aluminized sachet with transparent plastic film on the front part - 80 mm x 75 mm

ANVISA Packaging Table as: Aluminum and Polyethylene Sachet – code 0665

Secondary packaging: Cardboard Box - 85 mm x 80 mm

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**EN 6. Warnings and/or precautions to be adopted**

The Implacil De Bortoli Prosthetic Abutments should only be used by professionals specialized in the field. It is necessary to elaborate a prosthetic execution plan, observing the diameter, angle and length for adaptation of the prosthetic component to the implant in order to avoid future complications.

The Implacil De Bortoli Prosthetic Abutments were designed for single use, prohibiting reprocessing. Do not use it at all if the pack is damaged. The product is not sterile, thus this procedure must be performed by the professional before any intervention on the patient.

The professional must observe the force applied during application of the product in order to not damage it and inform the patient of the need for periodic monitoring to avoid physical and mechanical stresses in order to not subject the product to undue stresses, as well as the need for correct hygiene.

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**EN 7. Indication, purpose or intended use of the product**

The abutments are indicated for the final production of the single-tooth or multi-teeth prosthetic element, where functional and/or esthetic rehabilitation is required, attached with the use of screw or cement specific for dental procedure, able to make the necessary corrections of the implants on the axis of the teeth, both in the vertical and angulation dimension.

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**EN 8. Undesirable secondary effects or side effects and contraindications**

The final result of the application of the product may not be successful in cases where the technique used is not suitable and the patient is not subjected to the tests indicated and to the due prosthetic planning, with possible occurrence of loss or fracture of the product. The incorrect application of the product may cause side effects on the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection.

The use of Implacil Prosthetic Abutments is contraindicated in cases where the patient has chronic periodontal inflammation, unsuitable oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Load where the primary stability of the implant is not suitable.

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**EN 9. Safety and efficacy of the product**

The Implacil Prosthetic Abutments are completely safe provided the safety rules are observed in all procedures performed. The product was developed with material of non-toxic medical-dental use. When stored and preserved according to the indications of use, the characteristics of the product do not suffer changes or modifications.

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**EN 10. Installation or connection to other products**

The Implacil Prosthetic Abutments are connected to previously installed and osteointegrated implants, according to the models compatible for each type of connection.

The user is solely responsible for the use of any part, accessory or material not specified or foreseen in these directions for use.

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**EN 11. Precautions in the Case of Disposal**

To avoid environmental contamination or the undue use of the Prosthetic Abutments, when these are no longer useful, the disposal must be in hospital waste or site for potentially contaminated materials.

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#### EN 13. Warranty Term

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. The product may have indefinite validity if transported, stored and packed suitably according to these Instructions for Use.

Dental implants are estimated to have a service life of approximately 15 years, therefore, as established by Act no. 8078 dated September 11, 1990, provided there is a complaint during the limitation period of ninety days after the defect is observed, even if the defect is only manifested after the end of the contractual warranty period, Implacil De Bortoli agrees to replace the device with an identical or similar device.

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#### EN 14. Responsibility Term

The company IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A., represented by its technical person responsible Nilton De Bortoli Junior and by its legal representative Mário Sérgio De Bortoli, undersigned, assume the technical and legal responsibility for the product and that all the information provided referring to the product PROSTHETIC ABUTMENT contained herein is true.

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**Mário Sérgio De Bortoli**

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale