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IFU-006 INSTRUCTIONS FOR USE/ IFU-006 INSTRUCCIONES DE USO/ IFU-006 ISTRUZIONI PER L'USO

PRODUTO / PRODUCT/ PRODUCTO/PRODOTTO:

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EN Trade Name: Implacil surgical instruments

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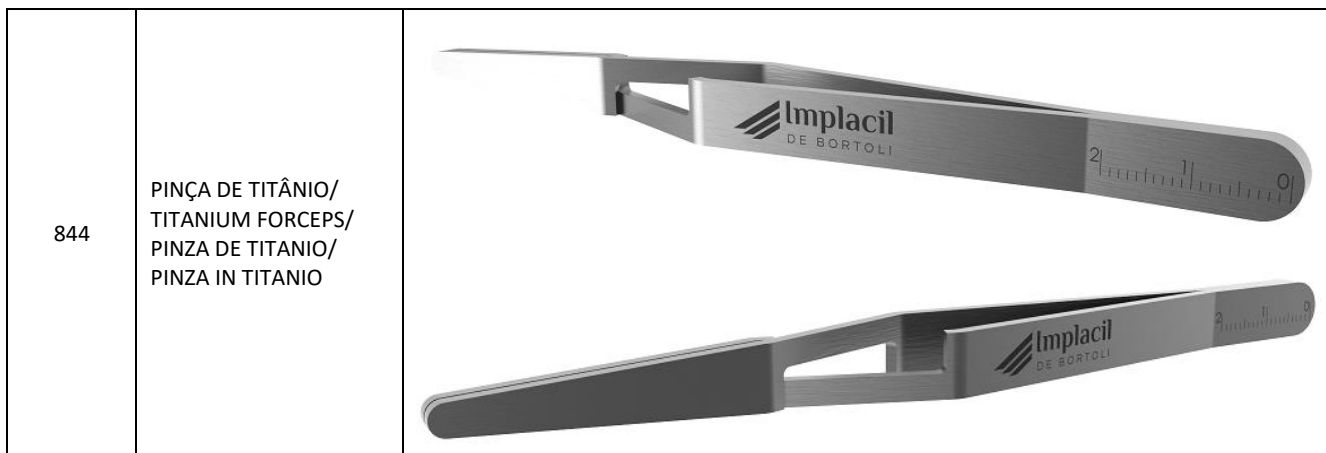
EN 1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT

The Implacil Surgical Instruments are presented in individual packaging.

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EN - Packaging for TAPERED TUNNEL CHECK:

The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

- Packaging for FORCEPS:

The primary packaging consists of a Surgical Grade Envelop.

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List of items contained inside the package

EN - 1 Surgical Instrument

- This instruction for use will be available in electronic format on the Implacil De Bortoli website (www.debortoli.com.br/manuais.asp) according to the effective legislation on the availability of instruction for use of healthcare products in non-printed format – IN Normative Instruction No. 4 dated June 15, 2012. The printed instructions for use will be available to the customer sent free of charge when requested through Customer Service. 0800-779-0011.

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EN 2- INDICATION OF USE AND PURPOSE

The Implacil instruments are pieces produced in standardized titanium, provided in different sizes and lengths, which are used to aid dental surgeries in general.

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EN 3 - MODE OF PRODUCT USE

Tunnel Check: Instrument used to measure the depth of the hole where the implant is to be placed;

Morse Taper Implant Forceps: used to aid dental surgeries where the morse taper implant will be used.

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EN Titanium Forceps: used to aid dental surgeries in general, the fact that it is completely made of titanium provides the lightness of the material and enables it to come in contact with various types of materials without contaminating them.

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EN The Implacil non-articulated non-cutting instruments should be used to aid dental surgeries. They should be previously hygienized and sterilized according to the techniques below:

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EN a) Cleaning:

The cleaning of a surgical instrument that was in contact with tissues should be performed such that all parts of the item are exposed to the cleaning process. The cleaning process must involve an initial rinse of the item with cold water, then wash with a neutral to slightly basic detergent with low foam formation, rinse again with water (tap water or stored water) followed by a final rinse with deionized water or water purified by reverse osmosis.

The initial cold rinse is performed to remove any coarse contamination present on the item. Cold water is used since temperatures above 140°F will clot proteins, making it difficult to remove them from the contaminated items. After rinsing with cold water, the items must be cleaned with a neutral to slightly basic detergent with low foam formation, with pH between 7.0 and 10.0.

If the contamination contains heavy or organic residues (protein or fat) or dry dirt, an enzymatic detergent can be used. The detergent used must be easy to rinse and must not leave residual compounds. A detergent with low foam formation reduces the formation of aerosol droplets, which may carry microorganisms. Detergent residues may cause stains and interfere in the action of chemical disinfectants. The washed items must be rinsed to remove the detergent, preferably with one final rinse in deionized or purified water by reverse osmosis.

The cleaning process must be performed such that all the parts of the surgical instrument are exposed. The remaining contamination on an item after cleaning reduces the efficacy of any decontamination or subsequent sterilization process.

Clean the instruments as soon as possible after use. Do not allow blood and residues to become dry on the instruments. If cleaning needs to be postponed, place the set of instruments in a container covered with enzymatic solution or appropriate detergent in order to slow down drying.

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EN b) Sterilization

It is a procedure that aims to completely eliminate microorganisms like: viruses, bacteria, microbes, fungi (in plant or spore form). After the cleaning process, the instruments are sterilized in an autoclave using temperature and exposure time according to the method below: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The surgical instruments should be arranged inside the container (perforated on the cover and lower part), or tray or in the box as long as they are arranged in a way that they can be prepared for the sterilization process. Do NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to allow the Drying Cycle to be completed on its own. Thoroughly and periodically clean the autoclave, removing the dirt and excess rust formed (iron oxide).

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EN 4- COMPOSITION: Titanium F67 is used as raw material for Surgical Instruments.

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EN 6- STORAGE CONDITIONS

Keep it in the original packaging protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

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EN 7 - TRANSPORT CONDITIONS

The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

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EN 8 - MANIPULATION CONDITIONS

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN 9 - WARNINGS

When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility.

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EN 10 - PRECAUTIONS

The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient.

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes.

Extreme care must be taken when handling the instruments to prevent injuries. The "instrument sets" are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

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EN 11 – CONTRAINDICATIONS: Not applicable.

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EN 14 - Warranty Term

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

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Mário Sérgio De Bortoli

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale

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EN Authorized Representative in the European Community:

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