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#### **GENERAL INFORMATION**

The Implacil instruments are manufactured with precision under strict processing guidance and regulations.

They are designed to offer durability and reuse possibility.

They are manufactured with a material that allows for a long usage period when handled appropriately.

They are supplied clean and not sterilized and should be sterilized before use.

After use they should be properly cleaned, decontaminated, sterilized and stored.

The following information refers to the cleaning, decontamination, care and manipulation, user responsibility, sterilization and storage of surgical instruments in order to guarantee a long life.

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\* for all the types listed above, there is a conventional model and an irrigated model.

Note: the difference between conventional drills and irrigated drills is that the irrigated drills have a hole in them.

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Material used in the manufacturing: Stainless Steel ASTM 899

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The drills are used in human use in general orthodontic surgeries.

The conical and cylindrical milling cutters make it possible to prepare the bone bed for the placement of implants through the perforation of the bone, which can be type I, II, III and IV. Noting that the diameter of the cutter must be compatible with the model of the implant to be used.

The Mini spear cutter is indicated to make an initial marking that allows guiding the use of the next milling cutters.

The Mini Trefina cutter is suitable for bone collection or even removal of implants.

Only Implacil De Bortoli products are indicated.

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Special conditions for transportation, storage and conditioning of the product:

Store at room temperature, in a well-ventilated place, away from light and heat. Avoid exposure to heat and excessive humidity.

The drills should be transported in a clean, dry location and at room temperature. The transportation should be done in a proper manner, to avoid falls and damage to the original package.

The drills should be clean, decontaminated and completely dry when stored. The package in which the items are sterilized may offer an effective barrier to maintain the item's sterility. After usage, they should be stored in a clean, dry, well-ventilated location.

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**5. Product Use instructions**

The Implacil drills should be used to aid in orthodontic surgeries.

Directions: For the surgical procedure, remove the drill from the instrument box, adapt it on the counter-angle through the spindle and verify the drill's lock in the counter-angle.

With the drill in movement (200 to 600 RPM) start the drilling.

After reaching the desired depth, remove the drill counter-angle set from the bone cavity.

Unlock the Drill and remove it from the counter-angle.

Previously they should be cleaned and sterilized according to the techniques below:

**a) Cleaning:**

The cleaning of a surgical instrument (drill) that was in contact with tissues should be guided in a manner that all the parts of the item are exposed to the cleaning process. The cleaning process should involve an initial rinse of the item with cold water, a wash with neutral to slightly basic detergent, with low foam formation, a rinse with water (tap or conditioned) followed by a final rinse with deionized water or water purified through reverse osmosis.

The initial cold rinse is carried out in order to remove any coarse contamination present on the item. Cold water is used, since temperatures above 140°F will coagulate proteins, making them hard to remove from the contaminated items. After the rinse with cold water, the items should be cleaned with a neutral to slightly basic detergent with low bubble formation, and pH between 7.0 and 10.0. If the contamination has heavy organic residues (protein or fat) or dry grime, an enzymatic detergent can be used. The detergent used should also be easy to rinse and not leave residue deposits. A low bubble formation detergent reduces the formation of aerosol droplets, which can transport microorganisms. Detergent residues can cause stain and interfere in the action of chemical disinfectants. The washed items should be rinsed to remove the detergent, preferably with a final wash with deionized water or water purified through reverse osmosis. The cleaning process should be carried out in a manner that all parts of the surgical instrument (drill) are exposed. The remaining contamination of an item after cleaning reduces the efficiency of any subsequent decontamination or sterilization process.

Clean the instruments (drills), as soon as possible, after use. Do not allow blood and residues to dry on the instruments (drills). If the cleaning needs to be put off, place groups of instruments (drill) in a container covered with enzymatic solution or proper detergent in order to delay drying.

**b) Sterilization (By wet heat)**

It is a procedure that seeks the total elimination of the microorganisms, such as: virus, bacteria, microbes, fungi (whether in the vegetative or spored form). After the drying process, the instruments (drills) are forwarded to sterilization in the autoclave, using temperature and exposure time according to the following method: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The surgical instruments should be placed inside the container (perforated on the lid and on the bottom part), or tray or in the box; as long as they are accommodated in a manner that they can be prepared for the sterilization process. Do NOT open the autoclave to avoid fast condensation; or in other words, let all the vapor out first allowing the Drying Cycle to complete itself. Strictly and periodically clean the autoclave, removing the grime and excess rust formed (iron oxide).

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**6. Warnings and/or precautions to be adopted**

You should have maximum care in the product's cleaning, sterilization, accommodation (stocking), transportation and handling process to avoid mechanic collisions which alter its anatomy and harm its functionality.

Before beginning the surgery, verify the product's functioning. It should not be used if it presents anatomic alterations. Upon handling the instruments (drills), use extreme caution to avoid injuries. Unless otherwise indicated, instrument sets are not sterilized and should be sterilized before use. Instruments should be autoclaved inside the instrumental box as in item 4b. instrumental boxes (drills) when not packaged do not maintain their sterility.

The drills should be substituted as soon as the professional realizes their decreased efficiency, but not more than 20 times of use.

Exclusive use for the orthodontic professional.

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**7. Unwanted secondary or side effects and contraindications**

Incorrect maintenance, cleaning and maintenance can make the instruments inappropriate for their intended use, cause corrosion, disassembly, distortion and/or breaking or even causing injuries to the patient or operation team.

Below is a list of potential complications:

- Injury or pain in the soft tissue, in the case of incorrect use or instrument breakage.
- Infection, if the instruments are not conveniently cleaned and sterilized.

- Damage to neighboring nerves or organs, as the result of slipping or poor positioning of a damaged instrument.
- Damage caused by instruments used to screw, due to excessive forces that occur during use.
- Increase of the operation time due to having to disassemble the instruments during the surgery.
- Due to necessary mechanical functionalities, the majority of the instruments (drills) are made from non implantable materials. In the case of the instrument splitting, no fragment should remain in the patient, because it can cause post-op complications, such as allergies, infections of biological nature complications, associated to the release of metallic components, possibly needing later intervention.
- The use of blunt drill, improper rotation or high drilling speed increases the risk of necrosis of bone tissue.

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#### **8. Precautions in Case of Disposal**

To avoid environmental contamination or improper use of the non-articulated cutting Implacil instrumentals, when they are no longer usable, they should be disposed of in a proper location, or in other words, in a location for potentially contaminated materials, according to applicable local / regional regulations.

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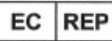
## 10. Warranty

IMPLACIL MATERIAL ODONTOLÓGICO LTDA. ensures, to the owner of this product, guarantee against any material or manufacturing defect that may be presented in the term of 365 days, counting from the acquisition date by the first acquirer. If transported, stored and packaged properly according to this User Manual, the product will have 5 (five) years of validity.

Note: the warranty does not cover wear from product use.

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