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IFU-013 INSTRUCTIONS FOR USE/IFU-013 INSTRUCCIONES DE USO/ IFU-013 ISTRUZIONI PER L' USO

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EN Technical name: Dental Surgical Drills

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EN * for all the types listed above, there is the conventional model and the irrigated model.

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EN Lengths available for each diameter 5, 7, 9, 11, 13 and 15 mm.

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EN Note: the difference between the conventional and irrigated drills is that the irrigated drills have a hole in them, as shown below:

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
EN Enlarged image of the reamers called "long":

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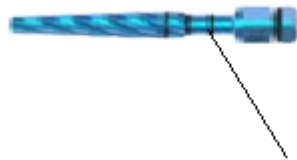EN The reamers called "long" are exactly identical to the already registered conventional ones, but the shanks are 5 mm longer.

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- 01 Fresa

EN 1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT

The Implacil Drills are supplied in individual packs.

The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

List of items contained inside the package

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EN 2- INDICATION OF USE AND PURPOSE

The drills are used in dental surgeries in general.

The drills are tools that have a cutting edge, which through rotating movements (controlled by the counter-angle or manually), perforate the bone structure of the patient.

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EN 3- MODE OF PRODUCT USE

For the surgical procedure, remove the drill from the instrument box, adapt it to the counter-angle through the shank, and check the lock of the drill on the counter-angle. With the drill in movement (200 to 600 RPM) start to drill. After reaching the desired depth, remove the counter-angle-Drill assembly from the bone cavity. Unlock the Drill and remove it from the counter-angle.

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EN They should be previously hygienized and sterilized according to the techniques below:

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EN a) Cleaning:

The cleaning of a surgical instrument (drill) that was in contact with tissues should be performed such that all parts of the item are exposed to the cleaning process.

The cleaning process must involve an initial rinse of the item with cold water, then wash with a neutral to slightly basic detergent with low foam formation, rinse again with water (tap water or stored water) followed by a final rinse with deionized water or water purified by reverse osmosis.

The initial cold rinse is performed to remove any coarse contamination present on the item.

Cold water is used since temperatures above 140°F will clot proteins, making it difficult to remove them from the contaminated items.

After rinsing with cold water, the items must be cleaned with a neutral to slightly basic detergent with low foam formation, with pH between 7.0 and 10.0.

If the contamination contains heavy or organic residues (protein or fat) or dry dirt, an enzymatic detergent can be used.

The detergent used must be easy to rinse and must not leave residual compounds. One

A detergent with low foam formation reduces the formation of aerosol droplets, which may carry microorganisms. Detergent residues may cause stains and interfere in the action of chemical disinfectants.

The washed items must be rinsed to remove the detergent, preferably with one final rinse in deionized or purified water by reverse osmosis.

The cleaning process must be conducted such that all the parts of the surgical instruments (drill) are exposed. The remaining contamination on an item after cleaning reduces the efficacy of any decontamination or subsequent sterilization process.

Clean the instruments (drills) as soon as possible after use. Do not allow blood and residues to become dry on the instruments (drills). If cleaning needs to be postponed, place the set of instruments (drill) in a container covered with enzymatic solution or appropriate detergent in order to delay drying.

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EN b) Sterilization:

It is a procedure that aims to completely eliminate microorganisms like: viruses, bacteria, microbes, fungi (in plant or spore form). After the cleaning process, the instruments (drills) are sterilized in an autoclave using temperature and exposure time according to the method below:

Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes.

The surgical instruments should be arranged inside the container (perforated on the cover and lower part), or tray or in the box as long as they are arranged in a way that they can be prepared for the sterilization process.

Do NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to allow the Drying Cycle to be completed on its own.

Thoroughly and periodically clean the autoclave, removing the dirt and excess rust formed (iron oxide).

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EN 6- STORAGE CONDITIONS

Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excess humidity.

The drills should be stored clean, decontaminated and completely dry. The packaging in which the items are sterilized can provide an effective barrier to maintain the sterility of the item. After use, they should be kept in a clean, dry and cool place.

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EN 7- TRANSPORT CONDITIONS

The drills should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

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EN 8- MANIPULATION CONDITIONS

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function.

Check the functioning of the product before initiating surgery. It should not be used if it has anatomical changes.

When handling the instruments (drills), use extreme caution in order to avoid injuries.

Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use.

Instruments should receive autoclave inside the instrument box according to 4.2.3b. Unpackaged instrument boxes (drills) do not maintain the sterility.

The drills should be replaced as soon as the professional notices its reduced efficacy.

Exclusive use by the dental professional.

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EN 10- PRECAUTIONS

Incorrect maintenance, cleaning and maintenance can make the instruments inappropriate for their intended use, cause corrosion, dismounting, distortion and/or break or even cause injuries on the patient or operating team.

Below is a list of potential complications:

Injury or pain in the soft tissues, in case of incorrect use or breakage of the instruments.

Infection, if the instrument is not conveniently clean and sterilized.

Damages to the nerves or nearby organs as a result of slide or bad positioning of a damaged instrument.

Damages caused by instruments used to screw, due to excess forces that occur during use.

Increase in the time of operation due to dismounting of the instruments during surgery.

Due to the necessary mechanical functionalities, majority of the instruments (drills) are made in non-implantable materials. In case an instrument breaks, no fragment should remain inside the patient because it can cause post-operative complications, such as allergies, infections or complications of the biological kind, associated to the release of metallic components, possibly requiring future intervention.

To avoid environmental contamination or undue use of the Implacil non-articulated cutting surgical instruments, when these are unusable, the instruments should be disposed in the appropriate place, i.e., in a place for potentially contaminated materials.

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