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- 01 Instrumento Cirúrgico

EN Commercial Presentation Forms of the Product: Implacil Surgical Instruments are supplied in individual packs. The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

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EN Indication of Use / Purpose: Implacil De Bortoli's rotary surgical instruments are pieces made of standard stainless steel, offered in various sizes and lengths, used to assist in general dental surgery.

All the instruments that integrate this register have connection to equipment, so they are called rotators.

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EN Mode of Product Use:

Implant Inserter For Motor HE: Apply the necessary torque for the placement of HE implants.

Implant Inserter For Motor HI: Apply the necessary torque for the placement of HI implants.

AR Morse Taper Insertion Driver For Motor: Apply the torque required for the placement of Morse AR Cone implants.

Driver Motor for Graft Screw: Apply the necessary torque for the placement of bone graft screws.

Cutter Extension Driver: Increase the length of the cutter.

Driver Insertion HE Motor: Apply the necessary torque for the placement of HE implants.

Driver Insertion HI Motor: Apply the necessary torque for the placement of HI implants.

Driver Insertion HI/CM Motor: Apply the necessary torque for the placement of HI/CM implants.

Driver Insertion HI/CM ST Motor: Apply the necessary torque for the placement of HI/CM implants of lengths of 5 and 6mm.

Driver Insertion HE ST Motor: Apply the necessary torque for the placement of HE implants of lengths of 5 and 6mm.

Driver Insertion HI ST Motor: Apply the necessary torque for the placement of HI implants of lengths of 5 and 6mm.

AR Due Cone Morse Taper Insertion Driver For Motor: Apply the necessary torque for the placement of morse cone AR implants.

Driver ORTH Motor: Apply the necessary torque for the placement of ORTH implants.

Driver ORTH Motor Long: Apply the necessary torque for the placement of ORTH implants.

Driver ORTH Motor Short: Apply the necessary torque for the placement of ORTH implants.

Driver Insertion Z Motor: Apply the necessary torque for placement of the Z abutment.

Installation driver labeled "long" are 5mm longer than the length of their rod, and their use is identical to conventional size keys, whether or not it is a professional choice.

The rotary instruments of Implacil De Bortoli should be used to assist in dental surgeries. Previously they should be sanitized and sterilized according to the following techniques:

a) Cleaning:

The cleaning of a surgical instrument that has been in contact with tissues must be conducted so that all parts of the item are exposed to the cleaning process. The cleaning process should involve an initial rinse of the item with cold water, a wash with a neutral detergent to slightly basic, with low foaming, a rinse with water (tap or conditioned) followed by a final rinse with deionized water or purified by reverse osmosis.

The initial cold rinse is conducted to remove any coarse contamination present in the item. Cold water is used, as temperatures above 140 ° F will coagulate proteins, making it difficult to remove them from contaminated items. After rinsing with cold water, the items should be cleaned with a mild to slightly basic detergent with low foaming, and pH between 7.0 and 10.0.

If the contamination contains heavy organic wastes (protein or fat) or dry dirt, an enzymatic detergent may be used. The detergent used should also be easy to rinse and leave no residual compounds. A low foaming detergent reduces the formation of aerosol droplets, which can carry micro-organisms. Detergent residues can cause stains and interfere with the action of chemical disinfectants. The washed items should be rinsed for detergent removal, preferably with a final rinse in deionized water or purified by reverse osmosis.

The cleaning process should be conducted in such a way that all parts of the surgical instrument are exposed. The remaining contamination on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on instruments. If cleaning needs to be delayed, place groups of instruments in a container covered with enzyme solution or appropriate detergent to delay drying.

b) Sterilization:

It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (whether in vegetative or sporulated form). After the drying process, the instruments are routed to the autoclave using temperature and time of exposure, according to the following method: Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 - 60 The surgical instruments must be arranged inside the container (perforated in the lid and at the bottom), or tray or box; since accommodated so that they can be prepared for the sterilization process. DO NOT open autoclave to prevent rapid condensation; ie let all the steam come out first, making the Drying Cycle complete on its own. Thoroughly and periodically clean the autoclave, removing dirt and excess rust formed (iron oxide).

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5- PRAZO DE VALIDADE

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EN EXPIRY DATE: 05 years from date of manufacture

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EN Storage Conditions: Keep in its original packaging in a place protected from direct sun and rain.

The product in question is not thermosensitive and does not show any variation in environments with high or low levels of humidity, until it can be sterilized by autoclaving. It is nevertheless recommended that the product is maintained between 15 ° and 35 ° C and at a humidity between 45% and 80%, to ensure stability of the package.

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EN Transport Conditions: The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

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EN Handling Conditions: The handling should be performed with the due cares to avoid falls and damages, it must be performed on shelves and counters, according to the activity that is being executed.

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EN Warnings: Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function. Check the functioning of the product before initiating surgery. It should not be used if it has anatomical changes.

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ES Precauciones: Improper maintenance, cleaning and sterilization may render the instruments unsuitable for its intended use, cause corrosion, disassembly, distortion and / or breakage, or cause injury or infection to the patient or operating personnel.

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Não se aplica.

EN Contraindications: Not applicable.

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EN Warranty Term: IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product against any material or manufacturing defect that may appear on it within 365 days, counting from the date of purchase by the first buyer. The product will have indefinite validity if transported, stored and packed suitably according to this User Manual. Note: the warranty does not cover wear from product use.

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IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A.