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**IFU-020 INSTRUCTIONS FOR USE/IFU-020 INSTRUCCIONES DE USO/ IFU-020 ISTRUZIONI PER L'USO**

### **PRODUTO:**

### **PRODUCT/ PRODUCTO/PRODOTTO:**

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**EN** Laboratory Materials

**ES** Materiales de Laboratorio

**ITA** Materiali di Laboratorio

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**EN** Commercial models:

**ES** Modelos comerciales:

**ITA** Modelli commerciali:

Pilar Calcinável (Uclas Plásticas):

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**EN** Calcinable Abutment (Plastic Uclas):

Ucla UN II HI (without hexagon, without shoulder); Ucla UN II HI R (without hexagon, with shoulder); Ucla UN II HI AR (com hexágono, without shoulder); Ucla UN II HI RAR (with hexagon, with shoulder); Ucla UN II HE (without hexagon, with shoulder); Ucla UN II HE AR (with hexagon, with shoulder); Ucla UN II HI (without hexagon, without shoulder); Ucla UN II HI R (without hexagon, with shoulder); Ucla UN II HE (without hexagon, with shoulder); Ucla UN II HE AR (with hexagon, with shoulder); Plastic Ucla UNII HE ST (without hexagon, with shoulder)

Plastic Ucla UNII HE AR ST (with hexagon, with shoulder); Plastic Ucla UNII HI R ST (without hexagon, with shoulder); Plastic Ucla UNII HI RAR ST (with hexagon, with shoulder); Plastic Ucla UNII HI ST (without hexagon, without shoulder); Plastic Ucla UNII HI AR ST (with hexagon, without shoulder); Plastic Classic Ucla;

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**EN** Plastic Copings:

Tapered Aesthetic Plastic Coping with Hex; Tapered Aesthetic Plastic Coping without Hex; Mini Tapered Plastic Coping L11; Morse Taper Abutment AR C Plastic Coping; Morse Taper Abutment AR P Plastic Coping; Morse Taper Abutment Coping for Angled; Plastic Hexagonal Abutment Coping; Plastic Rotational Coping; Abutment CM P; Rotational Abutment CM C Plastic Coping, Micro Tapered Plastic Coping.

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Analog HE 3.5 / 4.0 / 5.0

Analog HI 3.5 / 4.0 / 5.0

Tapered Aesthetic Anti-Rotational Analog / Tapered Aesthetic Rotational Analog

Mini Tapered Rotational Analog

Morse Taper Abutment C/P Analog 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6

Morse Taper Angled Abutment Analog 3.5X4 / 3.5X6 / 4.5X4 / 4.5X6

Morse Taper Implant Analog / CM AR

Classic Analog 3.25 / 3.75 / 4.25

Hexagonal Abutment Analog

Micro Tapered Analog

Ideale Morse Taper Abutment Analog 3.3X4

Ideale Morse Taper Abutment Analog 3.3X6

Ideale Morse Taper Abutment Analog 4.5x4

## Ideale Morse Taper Abutment Analog 4.5x6

31509 ANALOG D/G CM AR  
31510 ANALOG D/G HE 3.5 AR  
31511 ANALOG D/G HE 4.0 AR  
31512 ANALOG D/G HE 5.0 AR  
31513 ANALOG D/G HI 3.5 AR  
31514 ANALOG D/G HI 4.0 AR  
31515 ANALOG D/G HI 5.0 AR  
31518 ANALOG D/G IDEALE 3.3X4 AR  
31519 ANALOG D/G IDEALE 3.3X6 AR  
31520 ANALOG D/G IDEALE 4.5X4 AR  
31521 ANALOG D/G IDEALE 4.5X6 AR  
31517 ANALOG D/G TAPERED AESTHETIC AR  
31516 ANALOG D/G TAPERED AESTHETIC R  
30880 ANALOG D/G MINI TAPERED R  
31522 ANALOG D/G MICRO TAPERED R

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**EN** Prosthetic Selection Kit

COLLAR 0.8MM SELECTION KIT CM; COLLAR 1.5MM SELECTION KIT CM; COLLAR 2.5MM SELECTION KIT CM; COLLAR 3.5MM SELECTION KIT CM; COLLAR 4.5MM SELECTION KIT CM; COLLAR 5.5MM SELECTION KIT CM; STRAIGHT ABUTMENT CM 3.3X4.0MM SELECTION KIT CM; STRAIGHT ABUTMENT CM 3.3X6.0MM SELECTION KIT CM; STRAIGHT ABUTMENT CM 4.5X4.0MM SELECTION KIT CM; STRAIGHT ABUTMENT CM 4.5X6.0MM SELECTION KIT CM; ANG ABUTMENT 15° SELECTION KIT CM; ANG ABUTMENT 25° SELECTION KIT CM; MINI TAPERED ANG 17° 3.5MM SELECTION KIT CM; MINI TAPERED ANG 30° 3.5MM SELECTION KIT CM; MINI TAPERED STRAIGHT 3.5MM SELECTION KIT CM; TAPERED AESTHETIC STRAIGHT 3.5MM SELECTION KIT CM; COLLAR DRIVER SELECTION KIT CM; ABUTMENT DRIVER HEX.1.17MM SELECTION KIT CM; PROSTHETIC SELECTION KIT CM ; Irrigation Connector I; Prosthetic Selection Case CM.

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**EN Manufactured and Distributed by:** IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A.

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**EN 1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT**

The products contemplated in this instruction for use are presented in individual packaging.

The packaging consists of a blister sealed with Tyvek®.

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**EN** List of items contained inside the packaging

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**EN** 3 Tracking Labels\* with product identification for obligatory attachment: on the clinical record, on the document to be submitted to the patient, and on the invoice documentation, according to the model below:

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**EN** \*Items supplied with the tracking label:

Calcinable Abutment (Plastic Uclas)  
 Calcinable O'ring for Positioning  
 Plastic Copings  
 Plastic Uclas

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**EN** This instruction for use will be available in electronic format on the Implacil De Bortoli website ([www.implacildebortoli.com.br](http://www.implacildebortoli.com.br)) according to the effective legislation on the availability of instruction for use of healthcare products in non-printed format – Normative Instruction IN No. 4 dated June 15, 2012. The printed instructions for use will be available to the customer sent free of charge (including sending) when requested from Customer Service. 0800-779-0011.

To locate it, check the registration number of the product at ANVISA and review on the Label of the product purchased.

Adobe Acrobat Reader is necessary in order to read the instructions for use on the website.

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### **EN 2- INDICATION AND HOW TO USE THE PRODUCT**

The laboratory materials are indicated to aid in the making of single-tooth or multiple-teeth prosthetic element, according to description and application of each product.

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HE PLÁSTICA/  
PLASTIC HE/ HE  
PLÁSTICO / HE  
PLASTICA

HI PLÁSTICA/ Plastic  
HI / HI PLÁSTICO /  
HI PLASTICA

Modelos:

Ucla Plástica HE Anti-Rotacional 3,5 / 4,0 / 5,0

Ucla Plástica HE Rotacional 3,5 / 4,0 / 5,0

Ucla Plástica HE Anti-Rotacional ST 4,0

Ucla Plástica HE Rotacional 4,0

Ucla Plástica HI Rotacional 3,5 / 4,0 / 5,0

Ucla Plástica HI RAR (c/ hexágono c/ ombro) 3,5 / 4,0 / 5,0

Ucla Plástica HI AR (c/ hexágono s/ ombro) 3,5 / 4,0 / 5,0

Ucla Plástica HI R (s/ hexágono s/ ombro) 3,5 / 4,0 / 5,0

#### **EN - Calcifiable Abutment (Plastic Uclas):**

Calcifiable Component, used for casting, also known as plastic coping that fits directly over the laboratory model to be waxed in the ideal position and cast, becoming an abutment or customized metal structure and then applied over the implant;

- Best indication for making customized abutments;
- Versatile component that may be cemented or screwed, applied for overdenture, protocol and single-tooth or multiple-teeth elements;
- Anti-Rotational or Rotational;
- Supplied with definitive screw;
- Installation Torque: 30Ncm;

- Driver for installation – Square Driver – no. 4 – 1.3 mm.

Single-use product.

This product should not be sterilized.

Models:

Plastic Ucla HE Anti-Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HE Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HE Anti-Rotational ST 4.0

Plastic Ucla HE Rotational 4.0

Plastic Ucla HI Rotational 3.5 / 4.0 / 5.0

Plastic Ucla HI RAR (with hexagon, with shoulder) 3.5 / 4.0 / 5.0

Plastic Ucla HI AR (with hexagon, without shoulder) 3.5 / 4.0 / 5.0

Plastic Ucla HI R (without hexagon, without shoulder) 3.5 / 4.0 / 5.0

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Modelli:

Ucla in Plastica HE Anti-Rotazionale 3,5 / 4,0 / 5,0

Ucla in Plastica HE Rotazionale 3,5 / 4,0 / 5,0

Ucla in Plastica HE Anti-Rotazionale ST 4,0

Ucla in Plastica HE Rotazionale 4,0

Ucla in Plastica HI Rotazionale 3,5 / 4,0 / 5,0

Ucla in Plastica HI RAR (c/ esagono c/ spalla) 3,5 / 4,0 / 5,0

Ucla in Plastica HI AR (c/ esagono s/ spalla) 3,5 / 4,0 / 5,0

Ucla in Plastica HI R (s/ esagono s/ spalla) 3,5 / 4,0 / 5,0

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**EN - Calcinable O`ring for Positioning:**

Prosthetic component manufactured in acrylic, used for fixation of overdenture.

- Component with calcinable plastic individual ball head used for casting
- Cast in a laboratory over the bars with parallelometer, distributed in the ideal position correcting divergent implants;
- It is not used as a single-tooth element.

Single-use product.

This product should not be sterilized.

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**EN - Plastic Copings:** Component manufactured in polyacetel, used as a connecting link between the prosthetic abutment and the dental implant.

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Models:

Plastic Cemented Rotational Coping 3.5 / 4.5

Plastic Cemented Anti-Rotational Coping 3.5 / 4.5

Plastic Screwed Rotational Coping 3.5 / 4.5

Plastic Screwed Anti-Rotational Coping 3.5 / 4.5

Plastic Micro Tapered Coping 3.5

The rotational copings are indicated for multiple-teeth elements.

The anti-rotational copings are indicated for single-tooth elements.

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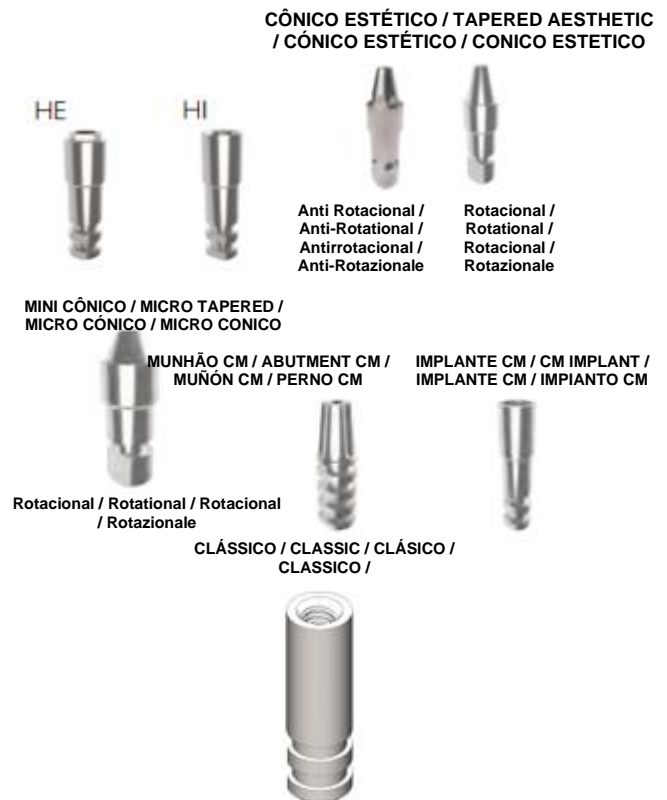
**EN - Analogs:** The analogs are manufactured in stainless steel and are used as a replica of the prosthetic abutments or implant, used in the prosthetic laboratory with the intention of simulating the position of the abutment / implant in the mouth of the patient through mold allowing the performance of prosthetic work.

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Analog HE 3.5 / 4.0 / 5.0

Analog HI 3.5 / 4.0 / 5.0

Tapered Aesthetic Anti-Rotational Analog

Tapered Aesthetic Rotational Analog

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Ideale Morse Taper Abutment Analog 4.5x6

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**EN** - The digital laboratory analogs / plaster (hybrid) Implacil De Bortoli, should be used as replicas of dental implants, prosthetic components or prosthetic intermediates, to be inserted in a 3D model printed, milled or applied to plaster, in order to duplicate the location, orientation and restorative platform of the implant, or prosthetic component applied to patients.

Through digital files, they offer high precision for the execution of laboratory processes in CAD (Computer Aided Design) and CAM (Computer Aided Machine). Due to its hybrid format, it can also be used in common laboratory plaster fixation processes.

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**EN - SCANCORP:** Scancorp is used in combination with the T Base abutments;  
 For scanning in the CAD CAM system.

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Barra Clip - 35°

Barra Clip - 70°

**EN - Clip Bar** Used for retention of removable prostheses, on two or more cylindrical bars cast using plastic clips fixed in the prosthesis.

- Clip type plastic component that adapts to the bar for overdentures, used as a retaining system in which the implants are joined together by a metal bar;
- It can also be used in combination with the Calcinable O'ring (Mixed System);
- Indicated for the mandible and maxilla.

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## **EN FOR USE IN LABORATORY PLANNING**

- The most complete selection kit for Morse Taper components: helps in the selection of the transmucosal implant, type of component, angulation, diameter and height of the element to be used.
- The only system with the option of 3 (three) combinations of the same collars, which helps in the planning for multiple-teeth elements;
- Easy to handle;
- High precision;
- Made in aluminum.

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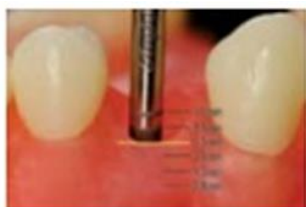
## PASSO A PASSO/ STEP BY STEP/ESP/ESP/PASSI:

O Túnel Check aplicado para medição, conforme ilustração acima, mostra que temos a profundidade de 3.5 mm até o limite gengival / The Tunnel Check applied for measurement, as illustrated above, shows that we have the depth of 3.5 mm to the gingival limit / Túnel Check aplicado para medición, según la ilustración anterior, muestra que tenemos la profundidad de 3.5 mm hasta el límite de la encía / Il Tunnel Check applicato per la misurazione, come illustrato qui sopra,

Descontado o valor de 2 mm selecionado então a cima 15 mm/ Discounting the value of 2 mm selecting the collar 1.5 mm/ Descontado el valor de 2 mm seleccionado entonces arriba de 15 mm / Detratto il valore di 2 mm si seleziona il

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**EN** Planning is no doubt one of the most important phases of the treatment and a success determining factor of the prosthesis over implants.

With the Selection kit we are able to analyze:

- Diameter;
- Abutment Height;
- Type of cemented or screwed system;
- Straight or angled.

The following criteria must be observed when planning a treatment that involves prosthesis over implants, namely:

- Ideal position of the crown in the arch;
- Ideal position of the implant (taking as reference the already established position of the crown).
- Selection of the most suitable component to obtain the best crown/implant connection result.

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#### EN Irrigation Serum Connector

Serves to connect the irrigation hoses if necessary.  
Manufactured in Stainless Steel.

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**EN 3- EXPIRY DATE:** 5 years from the date of manufacture.

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#### EN 4- STORAGE CONDITIONS

Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excess humidity.

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#### EN 5- TRANSPORT CONDITIONS

The laboratory materials should be transported in a clean and dry place at room temperature.  
The transport should be performed appropriately to avoid falls and damages in their original pack.  
Do not use the product if the packaging is damaged.

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#### **EN 6- MANIPULATION CONDITIONS**

Maximum care must be taken in the accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function.

The product should not be used if it has anatomical changes.

When handling the materials, use extreme caution in order to avoid injuries.

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**EN 7- WARNINGS:** Exclusive use by the dental professional.

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#### **EN 8- PRECAUTIONS**

Incorrect maintenance, cleaning and maintenance can make the instruments inappropriate for their intended use, cause corrosion, dismounting, distortion and/or break or even cause injuries on the patient or operating team.

Below is a list of potential complications:

- Injury or pain in the soft tissues, in case of incorrect use or breakage of the instruments.
  - Infection, if the instrument is not conveniently clean and sterilized.
  - Damages to the nerves or nearby organs as a result of slide or bad positioning of a damaged instrument.
  - Damages caused by instruments used to screw, due to excess forces that occur during use.
  - Increase in the time of operation due to dismounting of the instruments during surgery.
  - Due to the necessary mechanical functionalities, majority of the instruments (drills) are made in non-implantable materials. In case an instrument breaks, no fragment should remain inside the patient because it can cause post-operative complications, such as allergies, infections or complications of the biological kind, associated to the release of metallic components, possibly requiring future intervention.
- To avoid environmental contamination or undue use of the Implacil non-articulated cutting surgical instruments, when these are unusable, the instruments should be disposed in the appropriate place, i.e., in a place for potentially contaminated materials.

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**EN 9 – CONTRAINDICATIONS:** Not applicable.

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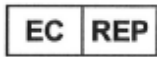
**EN 12 - Warranty Term**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

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**Mário Sérgio De Bortoli**

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale

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Tel/Fax: +351265238237

E-mail: [cinterqual.portugal@sapo.pt](mailto:cinterqual.portugal@sapo.pt)

**EN Authorized Representative in the European Community:**

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