escolhidos.

EN 2. Physical principal and technological fundaments for the product, applied for its functionality and action: Implacil Healing Abutments are mechanical components used to model the gingival tissue during the period of waiting before the implant is fitted. They are fixed to the implant using a screw. They are also indicated for protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments.

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EN 3. Optional extras, consumer materials and support materials: Optional extras: Non-applicable. Consumer Materials: Non-applicable. Support material: Instructions for use. Manufactured items for exclusive use with the Company's products: All of the components described in this Instruction for Use are for exclusive use with the Healing Abutment product. Attention: The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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EN 4. Special conditions for product transport, storage and packing: Storage: Store away from direct sunshine and rain and in its original packaging. Transport: During transport, protect the packaging from direct rain. Packing: Keep away from direct sunshine and rain.

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EN 5. Instructions for use of the product:

- Implacil Healing Abutments come in a blister pack sealed with Tyvek. The blister is placed inside a cardboard carton.
- Check that the packaging is in perfect condition. Do not use the product if the packaging is damaged.
- Open the outer packaging. Remove the blister from the box and check that it is perfectly sealed.
- Remove the Healing Abutment from the blister pack.
- Screw it onto the implant.

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EN 6. Warnings and/or precautions to be taken: The utilization of Implacil Healing Abutments must only be carried out by specialized dental surgeons. A conscientious routine of oral hygiene at home and regular professional treatment are required in order to conserve the restorations in good condition. Implacil Healing Abutments are designed for single use and their re-utilization is strictly prohibited. Under no circumstances must they be used if the packing seal is broken. The dental surgeon must pay attention to the amount of force used when applying the product so as not to damage it and must also inform the patient of the need for periodic monitoring that avoids physical and mechanical effort in order to avoid subjecting the product to undue strain.

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EN 7. Indication, purpose or use to which the product will be put: Implacil Healing Abutments are indicated for the cicatrization of the soft tissue (gums) protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments, while waiting for the prosthetic component to be made.

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EN 8. Undesired secondary or side effects and contraindications: The end result of the application of the product may be unsuccessful if inappropriate techniques used and this may lead to loss or fracturing of the product. Incorrect application of the product may lead to side effects in the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection. The utilization of Implacil Healing Abutments is contraindicated in cases where the patient presents chronic periodontal inflammation and inadequate oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Loading where the primary stability of the implant is insufficient.

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EN 9. Product safety and efficiency: Implacil Healing Abutments are totally safe, provided that these instructions for use are followed in all of the procedures carried out. This product is manufactured using non-toxic medical-odontological material produced by the world's leading bio-materials company (Carpenter Engineered Materials). When stored and conserved in accordance with the recommendations for use, the product characteristics will suffer no alterations or modifications, always respecting the shelf life of the same.

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EN 10. Installation or connection to other products: Implacil Healing Abutments are connected to previously installed and osseointegrated implants, according to the compatible models for each type of connection. The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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
EN 11. Care to be Taken in Disposal of Product: To avoid environmental contamination or inappropriate use of the Healing Abutments, after utilization they must be disposed of in a suitable location, in other words, in a place for potentially contaminated materials.

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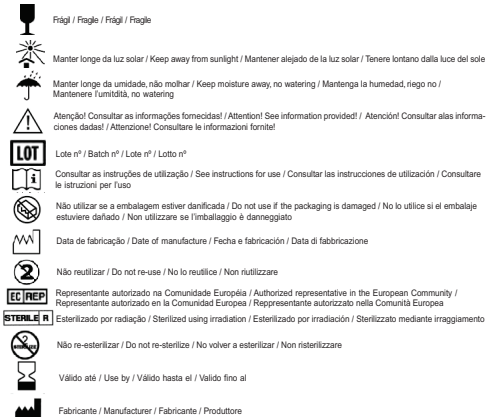EN 13. Term of Guarantee:

IMPLACIL DE BORTOLI MATERIAL ODONTOLÓGICO LTDA. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. Provided it is suitably transported, stored and packaged, in accordance with these Instructions for Use, the product will have an indefinite shelf-life.

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