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IFU-006 INSTRUCTIONS FOR USE/ IFU-006 INSTRUCCIONES DE USO/ IFU-006 ISTRUZIONI PER L'USO

### PRODUTO / PRODUCT/ PRODUCTO/PRODOTTO:

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**EN Trade Name:** Implacil surgical instruments

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#### **EN 1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT**

The Implacil Surgical Instruments are presented in individual packaging.

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EN - Packaging for TAPERED TUNNEL CHECK and TUNNEL CHECK:

The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

- Packaging for FORCEPS:

The primary packaging consists of a Surgical Grade Envelop.

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List of items contained inside the package

EN - 1 Surgical Instrument

- This instruction for use will be available in electronic format on the Implacil De Bortoli website ([www.debortoli.com.br/manuais.asp](http://www.debortoli.com.br/manuais.asp)) according to the effective legislation on the availability of instruction for use of healthcare products in non-printed format – IN

Normative Instruction No. 4 dated June 15, 2012. The printed instructions for use will be available to the customer sent free of charge when requested through Customer Service. 0800-779-0011.

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**EN** **2- INDICATION OF USE AND PURPOSE**

The Implacil instruments are pieces produced in standardized titanium, provided in different sizes and lengths, which are used to aid dental surgeries in general.

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**EN** **3 - MODE OF PRODUCT USE**

**Tunnel Check:** Instrument used to measure the depth of the hole where the implant is to be placed;

Morse Taper Implant Forceps: used to aid dental surgeries where the morse taper implant will be used.

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**EN** **Titanium Forceps:** used to aid dental surgeries in general, the fact that it is completely made of titanium provides the lightness of the material and enables it to come in contact with various types of materials without contaminating them.

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**EN** The Implacil non-articulated non-cutting instruments should be used to aid dental surgeries. They should be previously hygienized and sterilized according to the techniques below:

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**EN** The cleaning of a surgical instrument that has been in contact with tissue must be conducted in such a way that all parts of the item are exposed to the cleaning process.

For cleaning Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation\*.

3. Soak the products in the prepared detergent solution and soak for at least 5 minutes. While soaking, brush items with a soft-bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) in running water.

5. Visually inspect each product for visible dirt. If there is any visible dirt, repeat the cleaning process (step 3 and 4) until no more visible dirt is left.

\*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the instruments.

Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzyme solution or appropriate detergent to delay drying.

Surgical instruments must be replaced as soon as the professional perceives their reduced effectiveness.

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**EN** It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form).

After the drying process, the instruments are sent to sterilization in an autoclave, using temperature and exposure time, according to the following method.

This product must be properly sanitized and sterilized before use. Please use for steam sterilization only the steam sterilization according to the following parameters:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

Surgical instruments must be placed inside the container (perforated in the lid and at the bottom), or tray or box; provided that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

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**EN 4 - COMPOSITION:** Titanium F67 is used as raw material for Surgical Instruments.

The Conical Check Tunnel 3.5; 4.0 and 5.0 are machined from Titanium alloy ASTM F-136 Gr.5.

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**EN 6- STORAGE CONDITIONS**

Keep it in the original packaging protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

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**EN 7 - TRANSPORT CONDITIONS**

The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

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**EN 8 - MANIPULATION CONDITIONS**

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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#### **EN 9 - WARNINGS**

When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility.

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#### **EN 10 - PRECAUTIONS**

The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient.

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes.

Extreme care must be taken when handling the instruments to prevent injuries. The "instrument sets" are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

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**EN 11 – CONTRAINDICATIONS:** Not applicable.

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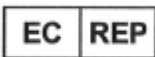
**EN 14 - Warranty Term**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

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**Mário Sérgio De Bortoli**

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale

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Tel/Fax: +351265238237

E-mail: [cinterqual.portugal@sapo.pt](mailto:cinterqual.portugal@sapo.pt)

**EN Authorized Representative in the European Community:**

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