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**EN** For greater safety:

- a) Carefully read all the instructions contained herein before using this product.
- b) Ensure that all the instructions are informed in this manual's content.

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**EN** This Manual was originally written in Portuguese.

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**EN** 1.1 Commercial Models “UN II Morse Taper Abutment”

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN** 1.2 Commercial Models “Angled UN II Morse Taper Abutment”

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 7. Indication, purpose or intended use of the product**

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**EN 8. Undesirable secondary effects or side effects and contraindications**

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**EN 9. Safety and efficacy of the product**

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**EN 10. Installation or connection to other products**

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**EN 11. Concerns in case of disposal of the product**

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**EN A) Description**

The Implacil UN II Morse Taper Abutment consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and have a collar height of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm or 5.5mm. It has a Morse Taper fit with angulation of 11° on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters of 3.5mm and 4.5mm are compatible with all the Morse Taper implant diameters sold; see the insertion driver with the implant in the compatibility table in item C. It is used in the final production of the cemented prosthesis element, especially of single-tooth restorations.

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**EN C) Technical specifications and features**

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**EN Raw Material:**

The raw material used for the UN II Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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EN 1.2 Commercial Models “Angled UN II Morse Taper Abutment”

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#### EN A) Description

The Implacil Angled UN II Morse Taper Abutment consists of an abutment with angulation of 15° or 25° and a fixation screw, manufactured in Titanium Alloy ASTM F136 grade V. The abutments are of single use and have a total height of 11.5 mm and collars of 0.8mm, 1.5mm; 2.5mm; and 3.5mm. It has a Morse Taper on its lower base that is used to adapt it to the implant and make it anti-rotational. Its diameters are: 3.5mm and 4.5mm. It is used in the final production of the single-tooth or multiple-teeth cemented prosthetic element, where angulation correction is necessary.

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#### EN B) Accompanying parts and accessories

The product is packed individually

Each pack consists of:

- one titanium prosthetic abutment
- one titanium fixation screw

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**EN Raw Material:**

Titanium alloy ASTM F136 grade V is used as raw material for both the Angled UN II Tapered Esthetic Abutment and the fixation screw.

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**EN 1.3 - Commercial Models “Mini Tapered UNII Morse Taper”**

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#### **EN A) Product description**

The Implacil Mini Tapered Abutment consists of a straight tapered abutment manufactured in commercially pure titanium grade 4, according to ASTM-F67, available in collar heights of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm and 5.5mm. The abutments are of single use. It has a morse taper on its lower base that is used to adapt it to the morse taper implant and make it anti-rotational. The diameter is 4.8 mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, especially in the esthetic zone, in cases with limited interocclusal space.

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**EN Raw Material:**

The raw material used for the Straight Tapered Esthetic Abutment is commercially pure Titanium ASTM F-67.

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**EN A) Description**

The Implacil Mini Tapered Angled UN II CM Abutment consists of an abutment with angulation of 17° or 30° and a fixation screw, manufactured in Titanium Alloy ASTM F136 grade V. The abutments are of single use and have collars of 0.8mm, 1.5mm; 2.5mm; and 3.5mm. It has a Morse taper on its lower base that is used to adapt it to the implant and make it anti-rotational. It has a diameter of 4.8 mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, where angulation correction is necessary.

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**EN B) Accompanying parts and accessories**

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**EN Raw Material:**

Titanium alloy ASTM F136 grade V is used as raw material for both the Mini Tapered Angled UN II Morse Taper Abutment and the fixation screw.

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**EN A) Description**

The Implacil O’ring Abutment consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use, available with collar heights of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm and 5.5mm. Its upper base has a hexagon with 2.5 mm in diameter, and over it is a ball of 2.0 mm in diameter. Its lower base has a morse taper fit that is used to adapt it to the implant. It is used in the final production of the complete overdenture prosthesis.

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**EN Raw Material:**

The raw material used for the O'ring UN II Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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#### **EN** Description

The Implacil Esthetic Tapered UN II Morse Taper Abutment consists of a straight tapered abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use, available with collar heights of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm and 5.5mm. Its upper base has a hexagon of 3.3mm. It has a morse taper on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters are related to the seating platforms of the commercially available implants: 4.8mm. It is used in the final production of the multiple teeth screwed prosthetic element, especially in the aesthetic zone.

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**EN Accompanying parts and accessories**

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**EN Raw Material:**

The raw material used for the Esthetic Taper UN II Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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#### **EN** A) Description

The Implacil Esthetic Morse Taper Adapter consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use, has zero collar and taper height of 4.0 mm. Its diameter of 3.5 is compatible with the diameter of all the Morse Taper implants sold. It is used in the final production of the screwed or cemented prosthetic element, especially for single-tooth restorations where the implant is at the height of the cortical.

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**EN Raw Material:**

The raw material used for the Esthetic Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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### 1.8 Modelo Comercial “Pilar Smart Cone Morse”

**EN** 1.8 Commercial Models “Smart Morse Taper Abutment”

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#### **EN** Description

The Implacil Smart Morse Taper Abutment consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and have a collar height of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm or 5.5mm. It has a Morse Taper fit with angulation of 11° on its lower base that is used to adapt it to the type of implant and make it anti-rotational. The diameters of 3.5mm and 4.5mm are compatible with the diameters of all the Morse Taper implants sold. It is used in the final production of prosthesis element, and it has an internal thread (a) on its upper end, which allows the abutment to be cemented or screwed.

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**EN Technical specifications and features**

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**EN Raw Material:**

The raw material used for the Smart Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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**1.9 Modelo Comercial “Pilar Smart Biocontrol Cone Morse”**

**EN 1.9 Commercial Model “Smart Biocontrol Morse Taper Abutment”**

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**EN Description**

The Implacil Smart Biocontrol Morse Taper Abutment consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and have a collar height of 0.8mm, 1.5mm,

2.5mm, 3.5mm, 4.5mm or 5.5mm, the Smart Biocontrol Morse Taper Abutment has a portion of its mechanically treated collar (blasting) (a), which aims to avoid the bone resorption and consequently increase the success of the procedure. It has a Morse Taper fit with angulation of 11° on its lower base that is used to adapt it to the type of implant and make it anti-rotational. The diameters of 3.5mm and 4.5mm are compatible with the diameters of all the Morse Taper implants sold. It is used in the final production of prosthesis element, and it has an internal thread (b) on its upper end, which allows the abutment to be cemented or screwed.

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**EN Raw Material:**

The raw material used for the Smart Biocontrol Morse Taper Abutment is commercially pure Titanium ASTM F-67.

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#### EN A) Description

The Implacil Micro Tapered Morse Taper Abutment consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and have a collar height of 0.8mm, 1.5mm, 2.5mm, 3.5mm or 4.5mm. It has a Morse Taper fit with diameter of 3.5 mm and is compatible with all the diameters of the Morse Taper implants sold. It is used in the final production of prosthesis element, and it has an internal thread on its upper end, which allows the abutment to be screwed.

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**EN Raw Material:**

The raw material used for the Micro Tapered Morse Taper is commercially pure Titanium ASTM F-67.

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#### **EN** A) Description

The Implacil Micro Tapered Titanium Coping consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67, and a fixation screw (a) of the abutment on the implant, manufactured in titanium ASTM-F136.

The abutments are of single use and have a total height of 11mm and diameter of 3.5mm.

It has an inner fit on its lower base that is used to adapt it to the Micro Tapered Morse Taper Abutment of the Implacil System. It is used in the production of the temporary screwed prosthetic element.

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**EN Raw Material:**

Commercially pure titanium ASTM F-67 is used as raw material for the Titanium Micro Tapered Coping.

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### **EN 2. Physical principle and technological fundamentals of the product, applied for its operation and its action:**

The Prosthetic Abutments are mechanical components that act as a connecting link between the implant and the dental prosthesis, providing support and able to perform height and angulation corrections between the implant and the tooth. They are threaded to the implants. The prosthetic elements can be attached to the abutments in cemented or screwed form, with the professional responsible for deciding on the best method to be used.

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**EN Special transport, storage and packaging conditions of the product:**

Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excess humidity. Do not put excess load on the pack to avoid damaging the seals that ensure the sterility of the product.

The Implacil De Bortoli Prosthetic Abutments should be transported in a clean dry place at room temperature. The transportation should be carried out in a suitable manner to avoid falls and damages to its original pack and mechanical impacts that can damage the product.

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#### **EN 5. Instructions for product use**

- The Implacil Prosthetic Abutments have been packed inside an aluminum sachet and a secondary packaging.
- Check if the pack is in perfect conditions. Reject the product if the pack is damaged.
- Open the outer packaging. Remove the sachet from the pack and check if it is perfectly sealed.
- Remove the abutment from inside the sachet.
- The product is non-sterile, and this procedure should be carried out by the professional according to the effective standards (NBR 14332), or any effective legislation or standard that complements or substitutes it.
- Screw the abutment to the implant.
- Check the maximum torque to be applied in the model description (item C of this Instruction for Use).
- The abutment is previously sanitized in a classified room. Follow the stages below if you wish to sanitize again:
- - Initial rinse of the item with cold water.
- - Wash with neutral to slightly basic detergent
- - Final rinse with deionized or purified water by reverse osmosis.
- The product is not sterile, and this procedure should be carried out by the professional according to the following stages:
- - Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The items should be arranged inside the container (perforated on the cover and lower part), or tray or in the box, or surgical envelop, provided they are arranged in a way that they can be prepared for the sterilization process. Do NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to allow the Drying Cycle to be completed on its own. Clean the autoclave thoroughly and periodically.

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**EN** The product is presented in individual pack.

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**EN** 3 Tracking Labels with product identification for obligatory attachment: on the clinical record, on the document to be submitted to the patient, and on the invoice documentation, according to the model below:

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**EN** This instruction for use will be available in electronic format on the Implacil De Bortoli website ([www.implacildebortoli.com.br](http://www.implacildebortoli.com.br)) according to the effective legislation on the availability of instruction for use of healthcare products in non-printed format – Normative Instruction IN No. 4 dated June 15, 2012. The printed instructions for use will be available to the customer sent free of charge (including sending) when requested from Customer Service. 0800-779-0011.

To locate it, check the registration number of the product at ANVISA and review on the Label of the product purchased.

Adobe Acrobat Reader is necessary in order to read the instructions for use on the website.

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**EN** Type of pack:

The material and type of pack used to package the Implacil De Bortoli Prosthetic Abutments are widely known and of universal use, for both drugs and healthcare products and they are available in the ANVISA Packaging Table.

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**EN** Characteristics and dimensions of the pack:

Primary packaging: Aluminized sachet with transparent plastic film on the front part - 80 mm x 75 mm

ANVISA Packaging Table as: Aluminum and Polyethylene Sachet – code 0665

Secondary packaging: Cardboard Box - 85 mm x 80 mm

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#### **EN 6. Warnings and/or precautions to be adopted**

The Implacil De Bortoli Prosthetic Abutments should only be used by professionals specialized in the field. It is necessary to elaborate a prosthetic execution plan, observing the diameter, angle and length for adaptation of the prosthetic component to the implant in order to avoid future complications.

The Implacil De Bortoli Prosthetic Abutments were designed for single use, prohibiting reprocessing. Do not use it at all if the pack is damaged. The product is not sterile, thus this procedure must be performed by the professional before any intervention on the patient.

The professional must observe the force applied during application of the product in order to not damage it and inform the patient of the need for periodic monitoring to avoid physical and mechanical stresses in order to not subject the product to undue stresses, as well as the need for correct hygiene.

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#### **EN 7. Indication, purpose or intended use of the product**

The abutments are indicated for the final production of the single-tooth or multi-teeth prosthetic element, where functional and/or esthetic rehabilitation is required, attached with the use of screw or cement specific for dental procedure, able to make the necessary corrections of the implants on the axis of the teeth, both in the vertical and angulation dimension.

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#### **EN 8. Undesirable secondary effects or side effects and contraindications**

The final result of the application of the product may not be successful in cases where the technique used is not suitable and the patient is not subjected to the tests indicated and to the due prosthetic planning, with possible occurrence of loss or fracture of the product. The incorrect application of the product may cause side effects on the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection.

The use of Implacil Prosthetic Abutments is contraindicated in cases where the patient has chronic periodontal inflammation, unsuitable oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Load where the primary stability of the implant is not suitable.

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#### **EN 9. Safety and efficacy of the product**

The Implacil Prosthetic Abutments are completely safe provided the safety rules are observed in all procedures performed. The product was developed with material of non-toxic medical-dental use. When stored and preserved according to the indications of use, the characteristics of the product do not suffer changes or modifications.

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#### **EN 10. Installation or connection to other products**

The Implacil Prosthetic Abutments are connected to previously installed and osteointegrated implants, according to the models compatible for each type of connection.

The user is solely responsible for the use of any part, accessory or material not specified or foreseen in these directions for use.

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#### **EN 11. Precautions in the Case of Disposal**

To avoid environmental contamination or the undue use of the Prosthetic Abutments, when these are no longer useful, the disposal must be in hospital waste or site for potentially contaminated materials.

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#### **EN 13. Warranty Term**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product warranty against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. The product may have indefinite validity if transported, stored and packed suitably according to these Instructions for Use.

Dental implants are estimated to have a service life of approximately 15 years, therefore, as established by Act no. 8078 dated September 11, 1990, provided there is a complaint during the limitation period of ninety days after the defect is observed, even if the defect is only manifested after the end of the contractual warranty period, Implacil De Bortoli agrees to replace the device with an identical or similar device.

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#### **EN Responsibility Term**

The company IMPLACIL DE BORTOLI MATERIAL ODONTOLÓGICO S.A., represented by its technical person responsible Nilton De Bortoli Junior and by its legal representative Mário Sérgio De Bortoli, undersigned, assume the technical and legal responsibility for the product and that all the information provided referring to the product PROSTHETIC ABUTMENT CM contained in these Instructions for Use is true.

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**Mário Sérgio De Bortoli**

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale

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IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A.