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GENERAL INFORMATION

The Implacil instruments are manufactured with precision under strict processing guidance and regulations.

They are designed to offer durability and reuse possibility.

They are manufactured with a material that allows for a long usage period when handled appropriately.

They are supplied clean and not sterilized and should be sterilized before use.

After use they should be properly cleaned, decontaminated, sterilized and stored.

The following information refers to the cleaning, decontamination, care and manipulation, user responsibility, sterilization and storage of surgical instruments in order to guarantee a long life.

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This instructions for use will be available on the Implacil De Bortoli website (www.implacildebortoli.com.br) in accordance with the current legislation on the provision of instructions for use in non-printed health products - Regulation (EU) No. 207 / 2012 - regarding electronic instructions for the use of medical devices.

The printed instructions for use can be requested through the Customer Service - 0800.779.0011 or by e-mail implacil@implacil.com.br, being sent at no additional cost.

The deadline for submission is 7 days from the request.

To find it, check the "IFU" number and the version of the instruction that is indicated on the Label of the purchased product.

To read the instructions for use on the website, the Adobe Acrobat Reader program is required.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as provided by Implacil De Bortoli.

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Nome comercial: Fresa

Technical name: DENTAL SURGICAL DRILLS / Commercial name: Drill

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Applicable to the following commercial models:

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Material used in the manufacturing: Stainless Steel ASTM 899

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Drills are used for human use in dental surgery in general.

The conical and cylindrical milling drills make it possible to prepare the bone bed for the placement of implants through the perforation of the bone, which can be type I, II, III and IV. Noting that the diameter of the drill must be compatible with the model of the implant to be used.

The Mini spear drill is indicated to make an initial marking that allows guiding the use of the next milling drills.

The Mini Trefina drill is suitable for bone collection.

Only Implacil De Bortoli products are indicated.

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Special conditions for transportation, storage and conditioning of the product:

Store at the temperature indicated on the packaging, in a cool place, protected from light and heat.

Avoid exposure to heat and excessive humidity.

The drills must be transported in a clean and dry place and kept at the temperature indicated on the packaging. The transportation should be done in a proper manner, to avoid falls and damage to the original package.

The drills should be clean, decontaminated and completely dry when stored. The package in which the items are sterilized may offer an effective barrier to maintain the item's sterility. After usage, they should be stored in a clean, dry, well-ventilated location.

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5. Product Use instructions

Implacil drills should be used to assist in dental implant insertion surgery.

How to use: For the surgical procedure, remove the drill from the instrument box, adapt it to the contra-angle by means of the shank, check the locking of the drill in the contra-angle.

Use saline solution for abundant external irrigation during drilling.

Maximum pressure for using the drill should be 2kg.

Start drilling with the drill in motion (200 to 550 RPM).

Stay with the drill for a maximum of 5 seconds during the osteotomy.

After reaching the desired depth, remove the contra-angle-drill assembly from the bone cavity.

Unlock the drill and remove it from the contra-angle.

The drills must be sanitized and sterilized before use / reuse according to the following techniques:

the cleaning

The cleaning of a surgical instrument (drills) that has been in contact with the tissues must be conducted so that all parts of the item are exposed to the cleaning process.

To clean Implacil drills use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil abutments.

1. Initial rinse with cold water. The initial cold rinse is carried out to remove any gross contamination present in the item. Cold water is used, since above 60°C / 140 ° F they make proteins coagulate, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation *.

3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.

4. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (step 3) until there is no visible dirt left.

*** Recommendation for the cleaning step**

Stage	Product	Specifications
Cleaning	Enzyme Detergent	Composition: Protease, Lipase and Amylase enzymes Dilution: 1mL for each liter of drinking water Action time: 5 min (Manual); Water temperature: 40°C a 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument (drill) are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean the instruments (drills) as soon as possible after use. Do not allow blood and waste to dry on the instruments (drills). If cleaning needs to be delayed, place groups of instruments (drill) in a container covered with enzyme solution or appropriate detergent in order to delay drying.

The drills should be replaced as soon as the professional realizes their reduced effectiveness, but not more than 20 times of use.

b) STERILIZATION

• The product is not sterile, and this procedure must be performed by the professional following the steps:

Sterilize the products the day before or the day of the procedure.

ATTENTION: It is not recommended to autoclave these products in their original packaging.

The drills must be individually wrapped in a surgical envelope.

Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	Fractional Vacuum
Temperature	134°C
Sterilization time	15 minutes
Drying Time	20 minutes

1 Use fractional vacuum autoclave.

2 The actual drying time required depends directly on the parameters of the user's sole responsibility (load configuration and density, sterilizer conditions) and therefore must be determined by the user. However, drying times of less than 20 min should not be applied.

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6. Warnings and/or precautions to be adopted

You should have maximum care in the product's cleaning, sterilization, accommodation (stocking), transportation and handling process to avoid mechanic collisions which alter its anatomy and harm its functionality.

Before beginning the surgery, verify the product's functioning. It should not be used if it presents anatomic alterations. Upon handling the instruments (drills), use extreme caution to avoid injuries. Unless otherwise indicated, instrument sets are not sterilized and should be sterilized before use. Instruments should be autoclaved inside the instrumental box as in item 4b. instrumental boxes (drills) when not packaged do not maintain their sterility.

The drills should be substituted as soon as the professional realizes their decreased efficiency, but not more than 20 times of use. The drills were designed to fit any contra-angle or handpiece of any brand, as the design drawings following ISO 1797 - Dentistry. Shanks for rotary and oscillating instruments

Exclusive use for the dental professional**6. Advertencias y/o precauciones que serán adoptadas**

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7. Undesirable side or side effects and contraindications**7.1 Undesirable side or side effects**

Below is a list of potential complications:

- The use of a blunt drill, with inadequate rotation or with a high drilling speed increases the risk of bone tissue necrosis by overheating

- Injury or, soft tissue pain, in case of incorrect use or breakage of the instruments.
- Infection, if the instruments are not properly cleaned and sterilized.
- Damage to nearby nerves or organs as a result of slipping or poor positioning of a damaged instrument.
- Damage caused by the instruments used for screwing, due to excessive forces that occur during use.
- Increased operating time for having to disassemble the instruments during surgery.

7.2 Contraindications

- Incorrect maintenance, cleaning and maintenance may render the instruments unsuitable for their intended use, cause corrosion, disassembly, distortion and / or breakage or cause injury to the patient or the operating team.
- Due to the necessary mechanical functionality, most instruments (drills) are made of non-implantable materials. In the event that an instrument breaks, no fragments should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metallic components, possibly requiring further intervention.

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8. Precautions in Case of Disposal

To avoid environmental contamination or improper use of the non-articulated cutting Implacil instrumentals, when they are no longer usable, they should be disposed of in a proper location, or in other words, in a location for potentially contaminated materials, according to applicable local / regional regulations.

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10. Warranty

IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. ensures, to the owner of this product, guarantee against any material or manufacturing defect that may be presented in the term of 365 days, counting from the acquisition date by the first acquirer.

If transported, stored and packaged properly according to this User Manual, the product will have 5 (five) years of validity.

Note: the warranty does not cover wear from product use.

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Instruction of Use for all countries except Brazil

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IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A.