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EN NOTE! NON-STERILE PRODUCT.

The Autoclave Cases are supplied NON-STERILE and must be sterilized before use.

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EN 1 – Identification: Technical name: Box / Case for Dental Instruments

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EN 1.1 Product description:

The Sterilization Autoclavable Case consists of boxes produced in Polysulfone or Radel®, which offers excellent heat resistance and high mechanical resistance, indicated for conditioning medical products (Implacil instruments and implants) for sterilization. The use of cases optimizes the storage and organization of the instruments during surgery, contributing to its success and safety.

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EN Applicable to the following commercial models:

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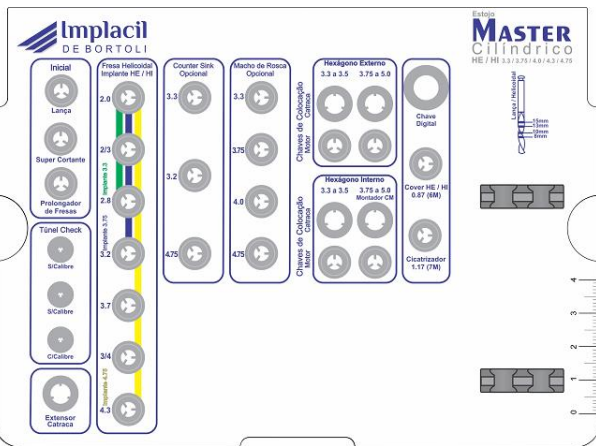
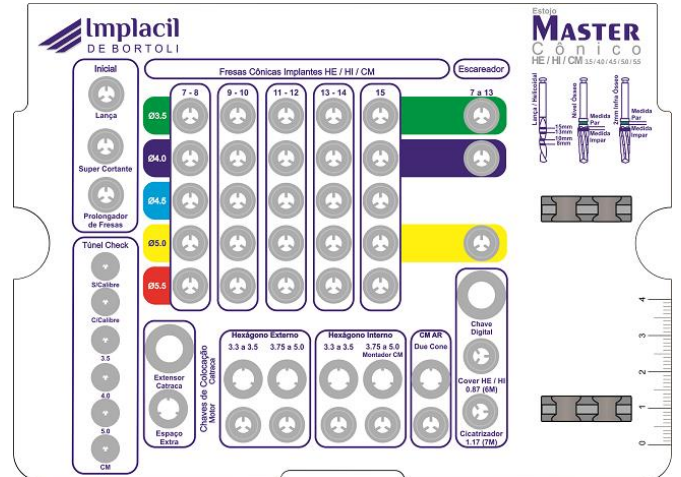
EN NOTE: The pieces that appear in some images are merely for illustration purposes, not supplied with the cases.

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EN 2. Indication:

The Sterilization Autoclavable Cases consist of boxes produced in polymer, indicated for conditioning medical products (Implacil instruments and implants) for sterilization. The use of cases optimizes the storage and organization of the instruments during surgery, contributing to its success and safety.

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EN 3. Raw Material: The following are used as raw material for the cases:

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Estojo Implaguide; / Estojo Bone Fix; / Estojo ROG

EN Radel® for models:

Surgical Case; / Tapered Surgical Case – Morse Taper; / Cylindrical Surgical Case – Tapered – Morse Taper;
Cylindrical Surgical Case; / Prosthetic Case; / Prosthesis Driver Case;
Sinus Lift Case; / Sinus Lift Case – 5 Pieces; / Sinus Lift Case – 13 Pieces;
Drill Case; / Morse Taper Drill Case; / Tapered Drill Case; / Expander Case; / Osteotomy Case;
Reamer Case; / Autogenous Graft Case; / Compact Case; / Implaguide Case; / Bone Fix Case;
ROG Case;

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EN Polysulfone for models:

Master Cylindrical Case HE/HI 3.3 3.75 4.0 4.3 4.75
Master Tapered Case HE/HI/CM 3.5 4.0 4.5 5.0 5.5
Excellence Cylindrical Tapered Case HE/HI/CM 3.3 3.75 3.5 4.0
Excellence Cylindrical Tapered Case HE/HI/CM 3.3 3.75 3.5 4.0 5.0
Excellence Guided Surgical Raptor Case CM 3.5 4.0
Prosthetic Case HE/HI/CM
Master Excellence Cylindrical Tapered Case HE/HI/CM 3.3 3.75 4.75 3.5 4.0 4.5 5.0 5.5
Veloce Case

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Estuche Master Cilíndrico HE/HI 3.3 3.75 4.0 4.3 4.75
Estuche Master Cónico HE/HI/CM 3.5 4.0 4.5 5.0 5.5
Estuche Excellence Cilíndrico Cónico HE/HI/CM 3.3 3.75 3.5 4.0
Estuche Excellence Cilíndrico Cónico HE/HI/CM 3.3 3.75 3.5 4.0 5.0
Estuche Excellence Cirugía Guiada Raptor CM 3.5 4.0
Estuche Protésico HE/HI/CM
Estuche Master Excellence Cilíndrico Cónico HE/HI/CM 3.3 3.75 4.75 3.5 4.0 4.5 5.0 5.5
Estuche Veloce

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EN Both raw materials have the following characteristics: translucent and ductile, high resistance and rigidity, high heat resistance (HDT), biocompatible and compatible with the sterilization method used.

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EN 4. Special transport, storage and packaging conditions of the product:

Storage: Keep it in the original packaging protected from rain and direct sunlight.

Transport: Protect the packaging from direct rainfall when transporting.

Packing: Keep in a place protected from rain and direct sunlight, at room temperature.

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EN 5. Instructions for product use:

- The Sterilization Autoclavable Cases must be sanitized before the sterilization process and its consequent inclusion in a sterilized surgical field.

Follow the instructions below:

CLEANING

Cleaning must be conducted in such a way that all parts of the item are exposed to the cleaning process.

To clean Implacil cases use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil cases.

1. Initial rinse with cold water. The initial cold rinse is carried out to remove any gross contamination present in the item. Cold water is used, as temperatures above 60°C / 140 ° F will clot proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation *.

3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.

4. Rinse: after removing the cases from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (steps 3 and 4) until there is no visible dirt left.

* Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline, disinfectants, hydrogen peroxide or alcohol for cleaning or rinsing the cutters.

Never use steel wool or sponges and abrasive products, so that the cutters are not damaged.

Cleaning: Enzyme Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40 ° C to 55 ° C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the cases are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean the cases as soon as possible after use. Do not allow blood and waste to dry on cases. If cleaning needs to be delayed, place the case in a container covered with enzyme solution or appropriate detergent to delay drying.

The cases should be replaced as soon as the professional realizes their reduced effectiveness or degradation, according to the manufacturer of the raw material, the cases withstand more than 1,000 cycles of steam sterilization at 134 °C without any significant loss of properties.

STERILIZATION

This product must be properly sanitized and sterilized before use. Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

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EN 6. Warnings and/or precautions to be adopted:

The customer is fully responsible for the validation of the sterilization process. We recommend only the steam autoclave process for sterilization of the Sterilization Autoclavable Cases.

Standard applicable to sterilization in steam autoclave: *BS EN ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices.*

We do not recommend sterilization by ethylene oxide, gamma radiation or dry heat. We recommend the immediate use after the sterilization process.

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Normativa applicabile per la sterilizzazione in autoclave a vapore: *BS EN ISO 17665-1: 2006 – Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices.*

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EN 7. Undesirable secondary or side effects and contraindications:

The incorrect use or manipulation may cause damage and possible malfunction of the product.

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EN 8. Precautions in the Case of Disposal:

To avoid environmental contamination or improper use of the Sterilization Autoclavable Cases, when these are no longer usable, they should be disposed in the appropriate location, i.e., in a place for potentially contaminated materials.

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EN 9. Expiry date: 5 years from the date of manufacture.

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EN 11. Warranty Term:

IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product warranty against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. The product may have indefinite validity if transported, stored and packed suitably according to these Instructions for Use.

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CNPJ: 05.741.680/0001-18

Email: implacil@terra.com.br

EN Manufactured and Distributed by:

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EN SAC [Customer Service] (55 11) 3271-0424

ANVISA [National Health Surveillance Agency] Registration no. Exempt from registration

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IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A.