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IFU-018 INSTRUCTIONS FOR USE / IFU-018 INSTRUCCIONES DE USO / IFU-018 ISTRUZIONI D'USO

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#### **EN 1- INDICATION OF USE AND PURPOSE**

Surgical Instruments are used in general orthodontic surgery.

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#### **EN 2- Special conditions of transport, storage and packaging of the product:**

The surgical instruments must be transported in a clean, dry place, at room temperature.

Transportation must be done properly, to avoid dropping and damage to its original packaging. Surgical instruments should be stored clean, decontaminated and completely dry.

The packaging in which items are sterilized may provide an effective barrier to maintain the sterility of the item.

After use, they should be stored in a clean, dry and cool place.

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### **EN 3- Instructions for use of the product:**

Surgical Instruments are to be used to assist in orthodontic surgery. Previously they should be sanitized and sterilized according to the following techniques:

**a) Cleaning:** Cleaning of a surgical instrument that has been in contact with tissues should be conducted so that all parts of the item are exposed to the cleaning process.

For cleaning the Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used, as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation\*.

3. Dip the products in the prepared detergent solution and soak for at least 5 minutes. While it is soaking, brush the items with a soft bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (step 3 and 4) until no more dirt is visible.

\*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

To clean or rinse the instruments, never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide, or alcohol.

Never use steel wool or steel sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process should be conducted so that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean the instruments, as soon as possible, after use. Do not allow that blood and debris dry on the instruments. If the cleaning must be delayed, place groups of instruments in a covered container with an appropriate enzymatic solution or detergent in order to delay drying.

The surgical instruments must be replaced as soon as the professional realize its diminished effectiveness.

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**EN b) Sterilization:** It is a procedure aimed at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments will be sent to autoclave for sterilization using temperature and exposure time, according to the following method:

This product should be properly sanitized and sterilized before use. For the sterilization, please use only steam sterilization according to the following parameters:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

DO NOT open the autoclave to prevent rapid condensation; i.e. let all steam escape first making the Drying Cycle complete by itself. Clean the autoclave rigorously and periodically, removing dirt and the excess of rust formed (iron oxide).

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#### **EN 4- Warnings and/or precautions to be taken:**

Every utmost care must be taken during the process of cleaning, sterilization, storage, transport, and handling of the product to avoid mechanical shocks that alter its anatomy and impair its functionality. Before the beginning of the surgery, check the functioning of the product. It should not be used if it presents anatomical alterations.

When handling the instruments, use extreme caution in order to avoid injury. Unless otherwise stated, instrument sets are not sterile and should be sterilized before use. Instruments must be autoclaved inside the instrument case.

Instrument boxes when not packed, do not maintain sterility. The Surgical Instruments must be replaced as soon as the professional perceives its effectiveness has decreased. Exclusive use of the dental professional.

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#### **EN 5- Undesirable side effects and contraindications:**

Incorrect maintenance and/or lack of cleaning may render the instruments unsuitable for their intended use, cause corrosion, disassembly, distortion and/or breakage, or cause injury to the patient or operating personnel. Below is a list of potential complications:

- Injury or, pain in soft tissue, in case of incorrect use or breakage of the instruments.

- Infection, if instruments are not properly cleaned and sterilized.
- Damage to nerves or nearby organs as a result of slippage or mispositioning of a malfunctioning instrument.
- Damage to the instruments used to screw due to excessive forces occurring during the use.
- Increased operation time by having to disassemble the instruments during surgery.
- Because of the mechanical functionality required, most of the instruments are made of non-implantable materials. If an instrument breaks, no fragment should remain in the patient, because it can cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metallic components, possibly requiring subsequent intervention.

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**EN 6- Precautions in case of disposal:**

To avoid environmental contamination or misuse of the Surgical Instruments, when discarding them, they must be disposed of in an appropriate place, that is, a place for potentially contaminated materials.

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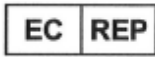
#### **EN 8- Warranty Term:**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees to the owner of this product, warranty against any defect of material or manufacture that presents itself in it within 365 days, from the date of purchase by the first acquirer.

If transported, stored and packaged properly in accordance with this User's Manual, the product will be valid indefinitely.

Note: Warranty does not cover wear and tear of product use

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