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EN ATTENTION! Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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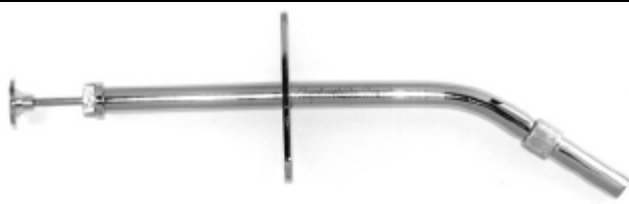
EN Material in conformity with the standards:

- NBR 13911 - Surgical instruments - Metal material - Specifications for dowels, bars and wires of stainless steel.
- ASTM F 899 - Standard Specification for Wrought Stainless Steels for Surgical Instruments.

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EN Indication: The Surgical Instruments are used in dental surgery in general.

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EN Special conditions for transport, storage and containerization of the product: The Surgical Instruments should be transported in a clean, dry place and at room temperature. Transportation should be executed adequately to avoid falls and damage in the original packaging. Surgical Instruments should be stored clean, decontaminated and completely dry. The packaging in which the items are sterilized may offer an effective barrier to maintain the sterility of the item. After use they should be kept in a clean, dry and well-ventilated place.

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EN Instructions for product use: The Surgical Instruments should be used to assist in dental surgery. They should be sanitized and sterilized beforehand according to the following techniques:

A) Cleaning:

The cleaning of a surgical instrument that has been in contact with tissue must be conducted in such a way that all parts of the item are exposed to the cleaning process.

For cleaning Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation*.

3. Soak the products in the prepared detergent solution and soak for at least 5 minutes. While soaking, brush items with a soft-bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) in running water.

5. Visually inspect each product for visible dirt. If there is any visible dirt, repeat the cleaning process (step 3 and 4) until no more visible dirt is left.

*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the instruments.

Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzyme solution or appropriate detergent to delay drying.

Surgical instruments must be replaced as soon as the professional perceives their reduced effectiveness.

B) Sterilization:

Sterilization is a procedure designed for total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments are sent for sterilization in an autoclave, using temperature and exposure time according to the following method:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

DO NOT OPEN the autoclave to avoid rapid condensation; in other words, let all the steam out first, so that the Drying Cycle terminates on its own. Clean the autoclave rigorously and periodically, removing the dirt and excess rust formed (iron oxide).

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EN Warnings and or precautions to be adopted: Particular care should be taken in the cleaning, sterilization, accommodation (stockpiling), transportation and handling of the product to avoid mechanical shocks that change its anatomy and affect its functionality. Before the start of the surgery verify the product operation. The product should not be used if it exhibits anatomical alterations. When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box pursuant to item 4b. Instrument boxes removed from their packaging do not remain sterile. The Surgical Instruments should be replaced as soon as the professional perceives their diminished efficacy. For exclusive use by dental professionals.

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EN Undesirable secondary or side effects and contraindications: Incorrect cleaning and maintenance may render the instruments inappropriate for their intended use, provoke corrosion, disassembly, distortion and/or breakage or also cause injuries to the patient or operation team. See below a list of potential complications:

- Lesion or pain in the soft tissues in case of incorrect use or breakage of the instruments.
- Infection, if the instruments are not properly cleaned and sterilized.
- Damage to the adjacent nerves or organs, as a result of slipping or incorrect positioning of a damaged instrument.
- Damage caused by the instruments used for screwing due to excessive forces that occur during use.
- Increase of surgical time due to the need to dismantle the instruments during surgery.
- Due to necessary mechanical functionalities, most instruments are made from non-implantable materials. In the eventuality of an instrument breaking, no fragment should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metal components, possibly requiring subsequent intervention.

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EN Precautions in case of disposal: To avoid environmental contamination or improper use of the Surgical Instruments, these must be disposed of in an appropriate place, that is, in a specific place for potentially contaminated materials.

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EN Product warranty: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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