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EN **ATTENTION!** Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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EN Carefully read all the instructions contained in this “User Manual” before using this product. Ensure that all the instructions are informed in this manual’s content.

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EN GENERAL INFORMATION

The Implacil De Bortoli instruments are manufactured with precision under strict processing guidelines and regulations. They are designed to offer durability and enable reuse.

They are manufactured with a material that allows long period of use when manipulated appropriately. They are supplied clean and are not sterile and must be sterilized before use. After use, they must be properly cleaned, decontaminated, sterilized and stored.

The information below refers to cleaning, decontamination, precaution and manipulation, user responsibility, sterilization and storage of the surgical instruments in order to help ensure a longer life.

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This Manual was originally written in Portuguese.

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EN Technical name: Instruments for dental use

Trade Name: Non-Articulated Non-Cutting Surgical Instruments

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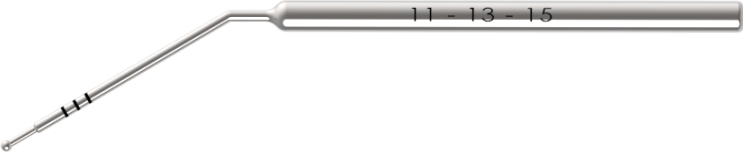
EN Applicable to the following commercial models:

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EN Material in conformity with the standards:

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EN Indication: The Implacil De Bortoli non-articulated non-cutting surgical instruments are pieces produced in standard stainless steel, provided in different sizes and lengths, which are used to aid dental surgeries in general.

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EN Special conditions for transport, storage and containerization of the product: Keep it in the original package protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35°C and humidity between 45% and 80% in order to ensure the stability of the packaging.

The instruments should be transported in a clean and dry place and at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN Instructions for product use: The Implacil non-articulated non-cutting instruments should be used to aid dental surgeries. They should be previously hygienized and sterilized according to the techniques below:

A) Cleaning:

The cleaning of a surgical instrument that has been in contact with tissue must be conducted in such a way that all parts of the item are exposed to the cleaning process.

For cleaning Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation*.

3. Soak the products in the prepared detergent solution and soak for at least 5 minutes. While soaking, brush items with a soft-bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) in running water.

5. Visually inspect each product for visible dirt. If there is any visible dirt, repeat the cleaning process (step 3 and 4) until no more visible dirt is left.

*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the instruments.

Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzyme solution or appropriate detergent to delay drying.

Surgical instruments must be replaced as soon as the professional perceives their reduced effectiveness.

B) Sterilization: Sterilization is a procedure designed for total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments are sent for sterilization in an autoclave, using temperature and exposure time according to the following method:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

DO NOT OPEN the autoclave to avoid rapid condensation; in other words, let all the steam out first, so that the Drying Cycle terminates on its own. Clean the autoclave rigorously and periodically, removing the dirt and excess rust formed (iron oxide).

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EN Warnings and or precautions to be adopted: When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility.

The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient.

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes.

Extreme care must be taken when handling the instruments to prevent injuries. The "instrument set" are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

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EN Undesirable secondary or side effects and contraindications: Incorrect cleaning and maintenance may render the instruments inappropriate for their intended use, provoke corrosion, disassembly, distortion and/or breakage or also cause injuries to the patient or operation team. See below a list of potential complications:

- Lesion or pain in the soft tissues in case of incorrect use or breakage of the instruments.
- Infection, if the instruments are not properly cleaned and sterilized.
- Damage to the adjacent nerves or organs, as a result of slipping or incorrect positioning of a damaged instrument.
- Damage caused by the instruments used for screwing due to excessive forces that occur during use.
- Increase of surgical time due to the need to dismantle the instruments during surgery.
- Due to necessary mechanical functionalities, most instruments are made from non-implantable materials. In the eventuality of an instrument breaking, no fragment should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metal components, possibly requiring subsequent intervention.

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EN Precautions in case of disposal: To avoid environmental contamination or improper use of the Surgical Instruments, these must be disposed of in an appropriate place, that is, in a specific place for potentially contaminated materials.

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EN Product warranty: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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