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EN **ATTENTION!** Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Reutilization is not recommended.

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EN **Technical name:** Impression Transfer

Trade name: Morse Cone Transfer Implant

Applicable to the following commercial models: Morse Cone Transfer Implant

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EN **Product Description**

Implacil Impression Transfers are pin-shaped parts that are screw-fixed on implants or dental implant components in order to transfer its position in the patient's mouth to obtain a mold trough impression.

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EN Accompanying parts and accessories

List of items contained in the package

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EN Specifications and technical characteristics

Each individual package is has an 80mmx80mm square shape.

Raw material: The raw material used for the Morse Cone Transfer Implant is 316L Stainless Steel.

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EN Physical Principle and product technology fundamentals, applied to its operation and action:

The product's operation is based on the reproduction of the implant or prosthetic pillar positioning in the patient's mouth to obtain a working mold which will be used in the clinic or laboratory.

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EN Optional, consumables and support materials

Optional: Not applicable. Consumables: Not applicable

Manufacturing items for exclusive use in the company products. All parts described in these Instructions for Use are to be used exclusively for the Morse Cone Transfer Implant.

Attention: The use of any parts, accessories or materials not specified or intended in these Instructions for Use is the entire responsibility of the user.

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EN Indication, purpose or use for the product

The Morse Cone Transfer Implant is indicated in cases of oral rehabilitation where it is necessary to reproduce the implant or prosthetic pillar positioning in the patient's mouth to obtain a working mold which will be used in the clinic or laboratory. Each transfer is manufactured for a specific type of implant or prosthetic pillar.

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EN How to use the product:

- Make a previous selection of the Transfer most suitable for the molding technique and the type of molding and the component to be used.
- Implacil Impression Transfer comes in a Tyvek-sealed blister packaging.
- Check if the packaging is in perfect condition. Reject the product in case the packaging is damaged.
- Check if the product is perfectly sealed.
- Remove the Transfer from the blister.
- The product is not sterile. A sterilization procedure must be performed by the professional according to current norms (NBR 14332), or any current norms or laws that complements or supersedes it.

- Screw the Impression Transfer to the component that is going to be transferred to the model.
- Use the molding technique to obtain a mold with the exact position for the Impression Transfer.
- Remove the Impression Transfer from the patient's mouth and fit it in the right position into the mold obtained.
- Fix the analogous of the component to be transferred to the mold by screwing it to the Impression Transfer.
- Pour the plaster to obtain the working model containing the component's replica in its original position inside the mouth.

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EN Warnings and/or precautions to be adopted

Storage: Keep the product in its original packaging, away from direct sunlight and rain. The product is not thermo-sensitive and presents no variation in environments with either high or low levels of humidity, which allows the product to be sterilized in autoclave. Nonetheless it is recommended that the product is kept at temperatures between 15° and 35° C and humidity between 45% and 80%, to ensure packaging stability.

Transportation: The Transfers must be transported in a clean, dry place at room temperature. Transportation must be carried out in the appropriate manner so as to prevent falls and damages to the original packaging.

Packaging conditions: Keep away from direct sunlight and rain.

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EN Implacil Impression Transfer can only be used by specialized professionals in the area. It is necessary to elaborate a prosthetic execution plan, observing the right diameter, angle and length for the adaptation of the prosthetic component to the implant in order to avoid future complications.

Implacil Impression Transfer was designed for single use, and its reprocessing is prohibited. Do not use it in any way if the packaging is violated.

The product is not sterile, so a sterilization procedure must be performed by the professional before any intervention on the patient. The professional must inform the patient about the necessity of periodic monitoring and avoiding physical and mechanical efforts in order not to submit the product to undue efforts, as well as the necessity of proper hygiene.

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EN Product warranty: IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO LTDA. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

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