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ATTENTION! Single Use Product.

Do not use if the package is opened or damaged.

Gamma radiation sterilized.

Do not re-sterilize.

For further safety, carefully read all directions herein before using this product.

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Ø máximo (mm): 4,0; 4,5; 5,5.

Altura da cinta (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

1.1 Commercial Models “Healing Abutment HI”

A) Description: The Implacil Healing Abutment HI consists of a straight abutment, manufactured in commercially pure grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period, to allow the

gingival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

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Ø máximo (mm): 4,0; 4,5; 5,5.

Altura da cinta (mm) 1,0; 2,0; 3,0; 4,0; 5,0; 6,0; 7,0.

1.2 Commercial Models "Healing Abutment HE"

A) Description: The Healing Abutment HE made by Implacil consists of a straight abutment, manufactured in commercially pure, grade 2 titanium, in accordance with norm ASTM-F67. The Healing Abutments are designed for single use, supplied with collar

heights (a) of 1.0mm; 2.0mm; 3.0mm; 4.0mm; 5.0mm; 6.0mm and 7.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M2.0 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gingival tissue to obtain the shape required for the desired esthetic profile. The product's diameters relate to the seating platforms for the available implants: 3.5mm; 4.0mm and 5.0mm.

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Ø máximo (mm): 3,3; 4,5.

Altura da cinta (mm) 0,8; 1,5; 2,5; 3,5; 4,5; 5,5.

1.3 Commercial Models "Healing Abutment CM"

A) Description: The Healing Abutment CM made by Implacil consists of an abutment made in commercially pure, grade 4 titanium, in accordance with norm ASTM --F67. The Healing Abutments are designed for single use, supplied with collar heights (a) of 0.80mm; 1.50mm; 3.0mm; 4.5mm and 6.0mm. Its upper base (b) has a 1.17 hexagon for connection with the insertion driver. Its lower base has an M1.8 thread (c) to be screwed on to the corresponding implant. It offers heights and formats that are compatible with the different abutments available and substitutes them during the cicatrization period. It allows the gengival tissue to obtain the shape required for the desired esthetic profile. Its diameters are compatible with all of the diameters for the commercially available Morse Taper implants.

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1.4 Accompanying parts and accessories : The product is packaged individually. Each package contains a titanium Healing Abutment.

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2. Physical principal and technological fundaments for the product, applied for its functionality and action:

Implacil Healing Abutments are mechanical components used to model the gingival tissue during the period of waiting before the implant is fitted. The recommended lifetime of the healing abutment is 4 to 6 weeks. They are fixed to the implant using a screw. They are also indicated for protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments.

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3. Optional extras, consumer materials and support materials:

Optional extras: Nonapplicable.

Consumer Materials: Non-applicable.

Support material: Instructions for use.

Manufactured items for exclusive use with the Company's products: All of the components described in this Instruction for Use are for exclusive use with the Healing Abutment product.

Attention: The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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4. Special conditions for product transport, storage and packing:

Storage: Store away from direct sunshine and rain and in its original packaging.

Transport: During transport, protect the packaging from direct rain.

Packing: Keep away from direct sunshine and rain.

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5. Instructions for use of the product:

- Implacil Healing Abutments come in a blister pack sealed with Tyvek. The blister is placed inside a cardboard carton.
- Check that the packaging is in perfect condition. Do not use the product if the packaging is damaged.
- Open the outer packaging. Remove the blister from the box and check that it is perfectly sealed.
- Remove the Healing Abutment from the blister pack.
- Screw it onto the implant.

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6. Warnings and/or precautions to be taken:

The use of healers should only be performed by specialized dentists. A careful routine of oral hygiene at home and regular professional treatment are necessary to keep the restorations in good condition. Implacil Healers are designed for single use, the risk of reuse is biological contamination and their reuse is strictly prohibited. Under no circumstances should they be used if the packing seal is broken. The dentist must be aware of the amount of force used in the application of the product so as not to damage it and must also inform the patient of the need for periodic monitoring to avoid physical and mechanical efforts so as not to subject the product to undue effort.

DO NOT REUSE: Risk of transmission of biologically active material.

It is recommended to use a torque of 20 N.cm to avoid microgap between the healing abutment and the implant and also to avoid bacterial proliferation.

Reconnection and maximum disconnection of the healing abutment is recommended: 2 cycles.

The time required to replace a healing abutment with a prosthetic element should be minimized (up to 20 - 40 seconds) to avoid pain, discomfort and maladjustment.

The dentist must be careful when removing the cap screw or healing abutment. In case of aspiration of foreign bodies in dental therapy, immediate management is necessary. Immediate symptoms include paroxysms, coughing, choking, choking, and possibly fatal obstructions.

Chlorhexidine mouth rinse (CHX) significantly limited plaque formation in ASA and is a valid contribution to mechanical brushing in the early stages of plaque control in dental implants.

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7. Indication, purpose or use to which the product will be put:

Healing Abutments are mechanical components used to model the gingival tissue during the period of waiting before the implant is fitted. They are fixed to the implant using a screw. They are also indicated for protecting Implacil Implants, preventing the soft tissue from closing over the implant and promoting a perfect adaptation of the previously selected prosthetic abutments.

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8. Undesired secondary or side effects and contraindications:

The end result of the application of the product may be unsuccessful if inappropriate techniques used and this may lead to loss or fracturing of the product.

Incorrect application of the product may lead to side effects in the region where it is applied, such as: pain, swelling, sensitivity, tissue reaction and infection.

The utilization of Implacil Healing Abutments is contraindicated in cases where the patient presents chronic periodontal inflammation and inadequate oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Loading where the primary stability of the implant is insufficient.

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9. Product safety and efficiency:

Implacil's Healers are completely safe as long as these instructions for use are followed in all procedures performed.

When stored and conserved according to the indications for use, the product will not undergo alterations or modifications in its characteristics. The product is suitable for use for up to 2 years from the date of manufacture.

Magnetic Resonance Imaging (MRI): Magnetic resonance imaging can cause heating and displacement of the prosthetic abutment. It is not recommended to use MRI equipment with power above 3 tesla.

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10. Installation or connection to other products:

Implacil Healing Abutments are connected to previously installed and osseointegrated implants, according to the compatible models for each type of connection.

The utilization of any component, accessory or material not specified or provided for in these instructions for use is the sole responsibility of the user.

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11. Care to be Taken in Disposal of Product:

To avoid environmental contamination or inappropriate use of the Healing Abutments, after utilization they must be disposed of in a suitable location, in other words, in a place for potentially contaminated materials.

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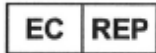
13. Term of Guarantee: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase. Provided it is suitably transported, stored and packaged in accordance with these instructions for use, the product will have a shelf life of 2 years.

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Instruction of Use for all countries except Brazil

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