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EN ATTENTION! Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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EN 1.1 Product description:

The Implacil De Bortoli non-articulated non-cutting surgical instruments are pieces produced in standard stainless steel, provided in different sizes and lengths, which are used to aid dental surgeries in general.

The Implacil Surgical Instruments are presented in individual packaging.

List of items contained in the package: - 1 Surgical Instrument.

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EN Applicable to the following commercial models:

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EN 2. Indication:

The operation of each of the different types of non-articulated non-cutting dental instruments herein is based on the specific needs of the surgical procedure. The design of each instrument is focused on the best efficiency, ergonomics and resistance of the product. The variety of instruments is necessary due to the wide range of conditions that occur during each surgery, providing more safety to both the surgeon and the patient.

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EN 3. Special transport, storage and packaging conditions of the product:

Keep it in the original package protected from direct sunlight and rain. The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels, thus it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

The drivers should be transported in a clean and dry place and at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN 4. Instructions for product use:

The Implacil non-articulated non-cutting instruments should be used to aid in dental surgeries.

They should be previously hygienized and sterilized according to the techniques below:

a) Cleaning:

The cleaning of a surgical instrument that was in contact with tissues should be performed such that all parts of the item are exposed to the cleaning process. The cleaning process must involve an initial rinse of the item with cold water, then washed with a neutral to slightly basic detergent with low foam formation, rinsed again with water (tap water or stored water) followed by a final rinse with deionized water or water purified by reverse osmosis.

The initial cold rinse is performed to remove any coarse contamination present on the item. Cold water is used since temperatures above 140°F will clot proteins, making it difficult to remove them from the contaminated items. After rinsing with cold water, the items must be cleaned with a neutral to slightly basic detergent with low foam formation and pH between 7.0 and 10.0.

If the contamination contains heavy or organic residues (protein or fat) or dry dirt, an enzymatic detergent can be used. The detergent used must be easy to rinse and must not leave residual compounds. A detergent with low foam formation reduces the formation of aerosol droplets, which may carry microorganisms. Detergent residues may cause stains and interfere in the action of chemical disinfectants. The washed items must be rinsed to remove the detergent, preferably with one final rinse in deionized or purified water by reverse osmosis.

The cleaning process must be performed such that all the parts of the surgical instrument are exposed. The remaining contamination on an item after cleaning reduces the efficacy of any decontamination or subsequent sterilization process.

Clean the instruments as soon as possible after use. Do not allow blood and residues to become dry on the instruments. If cleaning needs to be postponed, place the set of instruments in a container covered with enzymatic solution or appropriate detergent in order to delay drying.

b) Sterilization

It is a procedure that aims to completely eliminate microorganisms like: viruses, bacteria, microbes, fungi (in plant or spore form). After the cleaning process, the instruments are sterilized in an autoclave using temperature and exposure time according to the method below:

Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The surgical instruments should be arranged inside the container (perforated on the cover and lower part), or tray or in the box as long as they are arranged in a way that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to complete the Drying Cycle by itself. Thoroughly and periodically clean the autoclave, removing the dirt and excess rust formed (iron oxide).

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EN 5. Warnings and/or precautions to be adopted:

The customer is fully responsible for the validation of the sterilization process. We recommend only the steam autoclave process to sterilize the Non-Articulated Non-Cutting Surgical Instruments ST.

Standard applicable to sterilization in steam autoclave: *BS EN ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices.*

We do not recommend sterilization by ethylene oxide, gamma radiation or dry heat. We recommend the immediate use after the sterilization process.

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EN 6. Undesirable secondary or side effects and contraindications:

The incorrect use or manipulation may cause damage and possible malfunction of the product.

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EN 7. Precautions in the Case of Disposal:

To avoid environmental contamination or undue use of the Non-Articulated Non-Cutting Surgical Instruments ST, when these are no longer usable, the instruments should be disposed in the appropriate place, that is, in a place for potentially contaminated materials.

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EN Product warranty: IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO LTDA. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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