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EN ATTENTION! Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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EN 1.1 Product description:

The Implacil De Bortoli non-articulated non-cutting surgical instruments are pieces produced in standard stainless steel, provided in different sizes and lengths, which are used to aid dental surgeries in general.

The Implacil Surgical Instruments are presented in individual packaging.

List of items contained in the package: - 1 Surgical Instrument.

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EN Applicable to the following commercial models:

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EN 2. Indication:

The operation of each of the different types of non-articulated non-cutting dental instruments herein is based on the specific needs of the surgical procedure. The design of each instrument is focused on the best efficiency, ergonomics and resistance of the product. The variety of instruments is necessary due to the wide range of conditions that occur during each surgery, providing more safety to both the surgeon and the patient.

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EN 3. Special transport, storage and packaging conditions of the product:

Keep it in the original package protected from direct sunlight and rain. The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels, thus it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

The drivers should be transported in a clean and dry place and at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN 4. Instructions for product use:

The Implacil non-articulated non-cutting instruments should be used to aid in dental surgeries.

They should be previously igienized and sterilized according to the techniques below:

a) Cleaning:

The cleaning of a surgical instrument that has been in contact with tissue must be conducted in such a way that all parts of the item are exposed to the cleaning process.

For cleaning Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation*.

3. Soak the products in the prepared detergent solution and soak for at least 5 minutes. While soaking, brush items with a soft-bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) in running water.

5. Visually inspect each product for visible dirt. If there is any visible dirt, repeat the cleaning process (step 3 and 4) until no more visible dirt is left.

*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the instruments.

Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzyme solution or appropriate detergent to delay drying.

Surgical instruments must be replaced as soon as the professional perceives their reduced effectiveness.

b) Sterilization

It is a procedure that aims to completely eliminate microorganisms like: viruses, bacteria, microbes, fungi (in plant or spore form). After the cleaning process, the instruments are sterilized in an autoclave using temperature and exposure time according to the method below:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization according to the following parameters for sterilization:

| Sterilization Parameters | |
|--------------------------|-------------|
| Temperature | 132°C |
| Sterilization time | 20 min |
| Drying time | 20 - 60 min |

DO NOT open the autoclave to avoid rapid condensation, i.e., allow all the steam to come out first in order to complete the Drying Cycle by itself. Thoroughly and periodically clean the autoclave, removing the dirt and excess rust formed (iron oxide).

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EN 5. Warnings and/or precautions to be adopted:

The customer is fully responsible for the validation of the sterilization process. We recommend only the steam autoclave process to sterilize the Non-Articulated Non-Cutting Surgical Instruments ST. Standard applicable to sterilization in steam autoclave: *BS EN ISO 17665-1:2006 - Sterilization of health care products. Moist heat. Requirements for the development, validation and routine control of a sterilization process for medical devices.* We do not recommend sterilization by ethylene oxide, gamma radiation or dry heat. We recommend the immediate use after the sterilization process.

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EN 6. Undesirable secondary or side effects and contraindications:

The incorrect use or manipulation may cause damage and possible malfunction of the product.

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EN 7. Precautions in the Case of Disposal:

To avoid environmental contamination or undue use of the Non-Articulated Non-Cutting Surgical Instruments ST, when these are no longer usable, the instruments should be disposed in the appropriate place, that is, in a place for potentially contaminated materials.

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EN Product warranty: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. assures the owner of this product warranty against any material or manufacturing defect that appears within 365 days from the date of purchase by the first acquirer. If transported, stored and containerized adequately in accordance with this User Manual, the product will have an indefinite shelf life.

N.B.: the warranty does not cover wear and tear from use of the product.

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