**IFU-011 COMPONENTES CM**

COMPONENTES/COMPONENTS/COMPONENTES/COMPONENTI

**PARABÉNS!****CONGRATULATIONS! / ¡FELICIDADES! / CONGRATULAZIONI!**

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**EN** For greater safety:

- a) Carefully read all the instructions contained herein before using this product.
- b) Ensure that all the instructions are informed in this manual's content.

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**EN** This Manual was originally written in Portuguese.

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**EN** 1.1 Commercial Models “Abutment UNII CM”

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN** 1.2 Commercial Models “Abutment Angle UNII CM”

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN****1.3 Commercial Models “Mini Tapered UNII CM”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN****1.4 Commercial Models “Mini Tapered Angle UNII CM”**

- A) Product description
- B) Accompanying parts and accessories
- C) Technical specifications and features

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**EN 11. Concerns in case of disposal of the product**

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**EN** 1- Identification

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### 1.1 – Modelo Comercial “Munhão UNII Cone Morse”

**EN** 1.1 – Commercial Model “Abutment UNII CM”

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#### **EN** A) Description

The Implacil Abutment UNII CM consists of a straight abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use and have a collar height of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm or 5.5mm. It has a Morse Taper fit with angulation of 11° on its lower base that is used to adapt it to the type of implant and make it anti-rotational. Its diameters of 3.5mm and 4.5mm are compatible with all the Morse Taper implant diameters sold; see the insertion driver with the implant in the compatibility table in item C. It is used in the final production of the cemented prosthesis element, especially of single-tooth restorations.

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#### **EN** B) Accompanying parts and accessories

The product is packed individually

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#### **EN** C) Technical specifications and features

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**EN Raw Material:** The raw material used for the Abutment UNII CM is commercially pure Titanium ASTM F-67.

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**EN 1.2 Commercial Models** “Abutment Angle UNII CM”

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#### **EN A) Description**

The Implacil Abutment Angle UNII CM consists of an abutment with angulation of 15° or 25° and a fixation screw, manufactured in Titanium Alloy ASTM F136 grade V. The abutments are of single use and have a total height of 11.5 mm and collars of 0.8mm, 1.5mm; 2.5mm; and 3.5mm. It has a Morse Taper on its lower base that is used to adapt it to the implant and make it anti-rotational. Its diameters are: 3.5mm and 4.5mm. It is used in the final production of the single-tooth or multiple-teeth cemented prosthetic element, where angulation correction is necessary.

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#### **EN B) Accompanying parts and accessories**

The product is packed individually

Each pack consists of:

- one titanium component
- one titanium fixation screw

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#### **EN C) Technical specifications and features**

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**EN Raw Material:** Titanium alloy ASTM F136 grade V is used as raw material for both the Abutment Angle UNII CM and the fixation screw.

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**EN** 1.3 - Commercial Models “Mini Tapered UNII CM”

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#### **EN** A) Product description

The Implacil Mini Tapered UNII CM consists of a straight tapered abutment manufactured in commercially pure titanium grade 4, according to ASTM-F67, available in collar heights of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm and 5.5mm. The abutments are of single use. It has a morse taper on its lower base that is used to adapt it to the morse taper implant and make it anti-rotational. The diameter is 4.8 mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, especially in the esthetic zone, in cases with limited interocclusal space.

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**EN 1.4 Commercial Models** “Mini Tapered Angle UNII CM”

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#### **EN A) Description**

The Implacil Mini Tapered Angle UNII CM Abutment consists of an abutment with angulation of 17° or 30° and a fixation screw, manufactured in Titanium Alloy ASTM F136 grade V. The abutments are of single use and have collars of 0.8mm, 1.5mm; 2.5mm; and 3.5mm. It has a Morse taper on its lower base that is used to adapt it to the implant and make it anti-rotational. It has a diameter of 4.8 mm. It is used in the final production of the single-tooth or multiple-teeth screwed prosthetic element, where angulation correction is necessary.

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**EN Raw Material:** Titanium alloy ASTM F136 grade V is used as raw material for both the MINI TAPERED ANGLE UNII CM and the fixation screw.

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#### **EN** A) Description

The Implacil O’ring UNII CM consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use, available with collar heights of 0.8mm, 1.5mm, 2.5mm, 3.5mm, 4.5mm and 5.5mm. Its upper base has a hexagon with 2.5 mm in diameter, and over it is a ball of 2.0 mm in diameter. Its lower base has a morse taper fit that is used to adapt it to the implant. It is used in the final production of the complete overdenture prosthesis.

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**EN Raw Material:** The raw material used for the O'ring UNII CM is commercially pure Titanium ASTM F-67.

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**EN 1.6 - Commercial Models “Aesthetic Adapter CM”**

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#### EN A) Description

The Implacil Aesthetic Adapter CM consists of a straight cylindrical abutment manufactured in commercially pure titanium grade 4 according to ASTM-F67. The abutments are of single use, has zero collar and taper height of 4.0 mm. Its diameter of 3.5 is compatible with the diameter of all the Morse Taper implants sold. It is used in the final production of the screwed or cemented prosthetic element, especially for single-tooth restorations where the implant is at the height of the cortical.

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#### EN Accompanying parts and accessories

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#### EN C) Technical specifications and features

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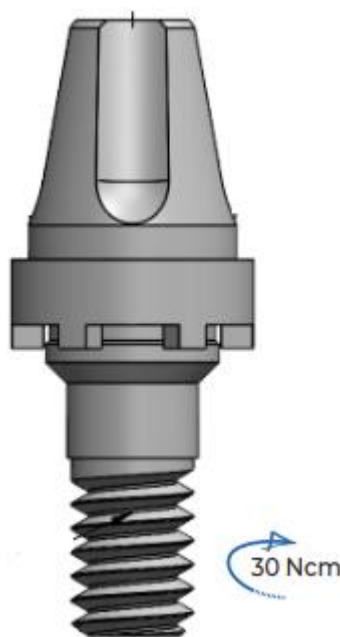
**EN Raw Material:** The raw material used for the Aesthetic Adapter CM is commercially pure Titanium ASTM F-67.

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**EN 2. Physical principle and technological fundamentals of the product, applied for its operation and its action:**

The Components are mechanical components that act as a connecting link between the implant and the dental prosthesis, providing support and able to perform height and angulation corrections between the implant and the tooth. They are threaded to the implants. The prosthetic elements can be attached to the abutments in cemented or screwed form, with the professional responsible for deciding on the best method to be used.

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**EN 4. Special transport, storage and packaging conditions of the product:**

Store at the temperature indicated on the packaging, in a cool place, protected from light and heat.

Avoid exposure to heat and excess humidity. Do not put excess load on the packaging to avoid damaging the seals that ensure the preservation of the product.

The Implacil De Bortoli Components must be transported in a clean, dry place and kept at the temperature indicated on the packaging. The transportation should be carried out in a suitable manner to avoid falls and damages to its original pack and mechanical impacts that can damage the product.

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## EN 5. Instructions for product use

- The Implacil Components have been packed inside an aluminum sachet and a secondary packaging.
- Check if the pack is in perfect conditions. Reject the product if the pack is damaged.
- Open the outer packaging. Remove the sachet from the pack and check if it is perfectly sealed.
- Remove the abutment from inside the sachet.
- The product is non-sterile, and this procedure should be carried out by the professional according to the effective standards (NBR 14332), or any effective legislation or standard that complements or substitutes it.
- Screw the abutment to the implant.
- Check the maximum torque to be applied in the model description (item C of this Instruction for Use). Use a torque wrench or torque referrer.
- This product is for single use and is supplied non-sterile.
- The Pilar is previously cleaned in a classified room. If you want to sanitize again follow the steps:

This product is for single use and is supplied non-sterile. This product must be properly sanitized and sterilized before use.

## CLEANING

Cleaning the Implacil abutment use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil abutments.

1. Remove the abutment and screw (if applicable).
2. Prepare the enzymatic detergent according to the recommendation\*.
3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.
4. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (step 3) until there is no visible dirt left.

**\* Recommendation for the cleaning step**

Stage	Product	Specifications
Cleaning	Enzyme Detergent	Composition: Protease, Lipase and Amylase enzymes Dilution: 1mL for each liter of drinking water Action time: 5 min (Manual); Water temperature: 40°C a 55°C (Manual);

## STERILIZATION

- The product is not sterile, and this procedure must be performed by the professional following the steps:

Sterilize the products the day before or the day of the procedure.

ATTENTION: It is not recommended to autoclave these products in their original packaging.

The product must be packed in a surgical envelope. The abutment and the abutment screw (if applicable) must be packed individually.

Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	Fractional Vacuum
Temperature	134°C
Sterilization time	15 minutes
Drying Time	20 minutes

<sup>1</sup> Use fractional vacuum autoclave.

<sup>2</sup> The effectively required drying time depends directly on parameters in sole responsibility of the user (load configuration and density, sterilizer conditions) and by this is to be determined by the user. Nevertheless, drying times less than 20 min must not be applied.

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**EN** All models come with 03 traceability labels, for mandatory fixing: in the medical record, in the document to be delivered to the patient, and in the tax documentation that generates the collection.

Each traceability label contains:

- a) name or business model;
- b) identification of the manufacturer;
- c) product or system component code;
- d) lot number and registration number with ANVISA.

This instructions for use will be available on the Implacil De Bortoli website ([www.implacildebortoli.com.br](http://www.implacildebortoli.com.br)) in accordance with the current legislation on the provision of instructions for use in non-printed health products - Regulation (EU) No. 207 / 2012 - regarding electronic instructions for the use of medical devices.

The printed instructions for use can be requested through the Customer Service - 0800.779.0011 or by e-mail [implacil@implacil.com.br](mailto:implacil@implacil.com.br), being sent at no additional cost.

The deadline for submission is 7 days from the request.

To find it, check the “IFU” number and the version of the instruction that is indicated on the Label of the purchased product.

To read the instructions for use on the website, the Adobe Acrobat Reader program is required.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as provided by Implacil De Bortoli.

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**EN** Type of pack:

The material and type of pack used to package the Implacil De Bortoli Components are widely known and of universal use, for both drugs and healthcare products and they are available in the ANVISA Packaging Table.

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**EN** Characteristics and dimensions of the pack:

Primary packaging: Aluminized sachet with transparent plastic film on the front part - 80 mm x 75 mm

Secondary packaging: Cardboard Box - 85 mm x 80 mm

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## **EN 6. Warnings and/or precautions to be adopted**

The Implacil De Bortoli Components should only be used by professionals specialized in the field. It is necessary to elaborate a prosthetic execution plan, observing the diameter, angle and length for adaptation of the prosthetic component to the implant in order to avoid future complications.

Implacil's Components were designed for single use, reprocessing being prohibited, the risk of reuse being that of biological contamination.

Do not use it at all if the pack is damaged. The product is not sterile, thus this procedure must be performed by the professional before any intervention on the patient.

The professional must observe the force applied during application of the product in order to not damage it and inform the patient of the need for periodic monitoring to avoid physical and mechanical stresses in order to not subject the product to undue stresses, as well as the need for correct hygiene.

The routine assessment of screw tightness is strongly recommended to minimize complications as screw loosening.

Recommend keep detailed information on the type of implant, abutment, screw, and type of cement to avoid the risk of screw loosening.

Risk of component fracture if the implant is above the bone level.

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**EN 7. Indication, purpose or intended use of the product**

The abutments are indicated for the final production of the single-tooth or multi-teeth prosthetic element, where functional and/or esthetic rehabilitation is required, attached with the use of screw or cement specific for dental procedure, able to make the necessary corrections of the implants on the axis of the teeth, both in the vertical and angulation dimension.

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**EN 8. Undesirable secondary effects or side effects and contraindications**

The final result of the application of the product may not be successful in cases where the technique used is not suitable and the patient is not subjected to the tests indicated and to the due prosthetic planning, with possible occurrence of loss or fracture of the product. The incorrect application of the product may cause side effects on the region where it is applied, such as: pain, fistulae/mucosal disorders, pus, occlusal disturbance, swelling, sensitivity, tissue reaction and infection. These aspects should be carefully considered in patients with the following conditions: Bruxism and Presence of Cantilevers.

The use of Implacil Components is contraindicated in cases where the patient has chronic periodontal inflammation, unsuitable oral hygiene and parafunctional habits, active intra-oral infection and in cases of Immediate Load where the primary stability of the implant is not suitable.

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**EN 9. Safety and efficacy of the product**

The Implacil Components are completely safe provided the safety rules are observed in all procedures performed. The product was developed with material of non-toxic medical-dental use. When stored and preserved according to the indications of use, the characteristics of the product do not suffer changes or modifications. The product is suitable for use for up to 5 years from the date of manufacture.

Magnetic Resonance Imaging (MRI): magnetic resonance can cause heating and displacement of the Components. It is not recommended to use magnetic resonance equipment with power over 3 tesla.

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**EN 10. Installation or connection to other products**

The Implacil Components are connected to previously installed and osteointegrated implants, according to the models compatible for each type of connection.

Use of components from other manufacturers is not recommended.

The components manufactured by Implacil are compatible only with the Implacil De Bortoli Implant system, the use of surgical and prosthetic instruments and / or prosthetic components from other manufacturers does not guarantee the function of the products and exempts any warranty for the related products.

The user is solely responsible for the use of any part, accessory or material not specified or foreseen in these directions for use.

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**EN 11. Precautions in the Case of Disposal**

To avoid environmental contamination or the undue use of the Components, when these are no longer useful, the disposal must be in hospital waste or site for potentially contaminated materials.

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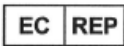
**EN 13. Warranty Term**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees the owner of this product warranty against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

Dental implants are estimated to have a service life of approximately 15 years, therefore, as established by Act no. 8078 dated September 11, 1990, provided there is a complaint during the limitation period of ninety days after the defect is observed, even if the defect is only manifested after the end of the contractual warranty period, Implacil De Bortoli agrees to replace the device with an identical or similar device.

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**EN Responsibility Term**

The company IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A., represented by its technical person responsible Nilton De Bortoli Junior and by its legal representative Mário Sérgio De Bortoli, undersigned, assume the technical and legal responsibility for the product and that all the information provided referring to the product COMPONENT CM contained in these Instructions for Use is true.

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**Mário Sérgio De Bortoli**

Responsável Legal / Legal Representative / Responsable Legal / Responsabile Legale

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**Instruction of Use for all countries except Brazil**

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IMPLACIL DE BORTOLI MATERIAL ODONTOLOGICO S.A.