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IFU-036 INSTRUCTIONS FOR USE / IFU-036 INSTRUCCIONES DE USO / IFU-036 ISTRUZIONI D'USO

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As embalagens individuais possuem o formato retangular de 50mmx80mm.

EN 1- FORMS OF PRODUCT BUSINESS PRESENTATION

The Molding Transfers are presented in individual packages.

The primary packaging consists of a blister pack produced from pure PET-type plastic, and is sealed with Tyvek.

List of items contained inside the package

- 01 Transfer

- 03 Medical Record Tags

- This instruction for use will be available in electronic format on the website of Implacil De Bortoli (www.debortoli.com.br/manuais.asp) in accordance with the legislation in force on the availability of instructions for use in a non-printed format for health products - Normative Instruction IN No. 4 of June 15, 2012.

- The printed instruction of use will be available to the customer who requests it, and will be sent free of charge, when requested through the Customer Service: 0800-779-0011.

To find it, check the instruction number: "IFU-XXX" and the version of the instruction that is indicated on the Label of the purchased product.

- Adobe Acrobat Reader is required to read the instructions on the site.

The dimensional characteristics of the plastic transfers in this process vary in diameter X height, where the first measure described corresponds to the diameter, which is 3.3 or 4.5 for the Transfer Pilar Ideale Cone Morse and 3.5 or 4.5 for Transfer Munhão Cone Morse. The second measure corresponds to the height that can be 4 or 6 for the two transfer models of this registration.

Dimensional characteristics of packaging:

Individual packaging have the rectangular format of 50mmx80mm.

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EN 2- INDICATION OF USE AND PURPOSE

The Molding Transfer is indicated for cases of oral rehabilitation in which it is necessary to reproduce the positioning of the implant or prosthetic abutment that is in the patient's mouth for the model that will be worked on in the office or laboratory. Each transfer is manufactured for a specific type of implant or prosthetic abutment.

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EN 3- PRODUCT INSTRUCTIONS FOR USE

- Pre-select the appropriate Transfer for the impression technique and the type of impression and component to be used.
- The Implacil Molding Transfers come packaged inside a Tyvek-sealed blister pack.
- Check if the packaging is in perfect condition. Discard the product if its packaging is damaged.
- Check if it is perfectly sealed.
- Remove the Transfers from inside the blister.
- The Transfer is previously sanitized in a classified room. If you wish to sanitize again, here are the steps:
 - Initial rinse the item with cold water.
 - Washing with a neutral to slightly basic detergent
 - Final rinse with deionized or reverse osmosis purified water.
- The product is not sterile and this procedure must be performed by the professional following the steps:
 - Sterilization temperature: 132°C, Sterilization cycle: 20 minutes and Drying cycle: 20 – 60 minutes. The items should be arranged inside the container (perforated on the cover and on the bottom), or tray or in the box, or in the surgical envelope; as long as they are accommodated in such a way that they can be prepared for the sterilization process. DO NOT open the autoclave to prevent rapid condensation; i.e. let all steam escape first making the Drying Cycle complete by itself. Clean the autoclave rigorously and periodically.
- Thread the Molding Transfer Case onto the component you want to transfer to the pattern.
- Use the molding technique to obtain a mold with the exact position of the Mold Transfer.
- Remove the Molding Transfer from the patient's mouth and fit it into position in the obtained impression.
- Fit the analog of the component to be transferred into the mold by threading it into the Molding Transfer.
- Insert the dental plaster to obtain the working model containing the replica of the component in the position where it originally is in the mouth.

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EN 6- STORAGE CONDITIONS

Keep in its original packaging in a place protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low levels of humidity. Even so, it is recommended that the product be kept between 15° and 35° C and at a humidity between 45% and 80%, to guarantee the stability of the package.

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EN 7- TRANSPORT CONDITIONS

Transfers must be transported in a clean, dry place at room temperature. Transportation must be done properly, to avoid dropping and damage to its original packaging.

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EN 8- HANDLING CONDITIONS

Handling should be carried out with due care to avoid falls and damage, and it should be done on the shelves and counters, according to the activity being performed.

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EN 9- WARNINGS

The use of Implacil's Impression Transfers must be made only by specialized professionals in the area. It is necessary to elaborate a prosthetic execution plan, observing the diameter, angle and length for the adjustment of the prosthetic component to the implant in order to avoid future complications. The product is not sterile, and this procedure must be performed by the professional before any intervention on the patient.

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EN 10- PRECAUTIONS

Implacil's Molding Transfers were designed for single use, not being recommended for reprocessing. Do not use it under any circumstances if the packaging is violated. The product is not sterile, and this procedure should be performed by the professional before any intervention on the patient.

The professional should inform the patient of the need for periodic monitoring to avoid physical and mechanical efforts so as not to subject the product to undue stress, as well as the need for proper cleaning.

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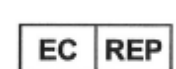
EN 14 – Warranty Term

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees to the owner of this product, warranty against any defect of material or manufacture that presents itself in it within 365 days, from the date of purchase by the first acquirer.

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