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**Nome comercial:** FRESA

**EN Technical Name:** DENTAL SURGICAL DRILLS / **Trade Name:** DRILL

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### **EN ATTENTION!!!**

DO NOT USE IF THE PACK HAS BEEN TAMPERED WITH OR DAMAGED. NON-STERILE PRODUCT. STERILIZE BEFORE USE. REUSABLE. CLEAN AND STERILIZE BEFORE REUSE.

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- 01 Fresa

**EN Commercial Presentation Forms of the Product:** The Drills are supplied in individual packs. The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

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**EN Indication of Use / Purpose:** The drills are used in dental surgeries in general.

The drills are tools that have a cutting edge, which through rotating movements (controlled by the counter-angle or manually) perforate the bone structure of the patient.

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**EN Mode of Product Use:** For the surgical procedure, remove the drill from the instrument box, adapt it to the counter-angle or digital driver through the rod, and then check the lock of the drill on the counter-angle. With the drill in movement (varying from 200 to 600 RPM, depending on the purpose) start drilling.

After reaching the desired depth, remove the counter-angle-Drill assembly from the bone cavity.

Unlock the Drill and remove it from the counter-angle. They should be previously hygienized and sterilized according to the techniques below:

**a) Cleaning:** The cleaning of a surgical instrument that has been in contact with tissues must be conducted in such a way that all parts of the item are exposed to the cleaning process.

To clean Implacil instruments, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil instruments.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present in the item. Cold water is used, as temperatures above 60°C / 140 ° F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation \*.

3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (steps 3 and 4) until no visible dirt is left.

\* Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline, disinfectants, hydrogen peroxide or alcohol for cleaning or rinsing the cutters.

Never use steel wool or sponges and abrasive products, so that the cutters are not damaged.

Cleaning: Enzyme Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40 ° C to 55 ° C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed.

Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a container covered with enzyme solution or appropriate detergent to delay drying.

The instruments must be replaced as soon as the professional realizes their diminished effectiveness or degradation of the item.

**b) Sterilization:** It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form).

After the drying process, the instruments are sent to sterilization in an autoclave, using temperature and exposure time, according to the following method.

This product must be properly sanitized and sterilized before use. Please use for steam sterilization only the steam sterilization according to the following parameters:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

Surgical instruments must be placed inside the container (perforated in the lid and at the bottom), or tray or box; provided that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

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**EN Composition:** ASTM F899 Stainless Steel is used as raw material for the Drills.

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#### 5- PRAZO DE VALIDADE

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**EN Expiry Date:** 05 years from the date of manufacture.

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**EN Storage Conditions:** Keep at room temperature, in a cool place, away from sunlight and heat. Avoid exposure to heat and excessive humidity.

The drills should be stored clean, decontaminated and completely dry. The pack in which the items are sterilized can provide an effective barrier to maintain the sterility of the item. After use, they should be kept in a clean, dry and cool place.

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**EN Transport Conditions:** The drills should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages in their original pack.

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**EN Handling Conditions:** The handling should be performed with the due cares to avoid falls and damages, it must be performed on shelves and counters, according to the activity that is being executed.

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**EN Warnings:** Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical shocks that change their anatomy and compromise their function. Check the functioning of the product before initiating surgery. It should not be used if it has anatomical changes.

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**EN Precautions:** When handling the instruments (drills), use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packed, instrument boxes (drills) do not maintain their sterility. The drills should be replaced as soon as the professional notices its reduced efficacy. Exclusive use by the dental professional.

Below is a list of potential complications:

- Injury or pain in the soft tissues, in case of incorrect use or breakage of the instruments.
- Infection, if the instrument is not conveniently clean and sterilized.
- Damages to the nerves or nearby organs as a result of slide or bad positioning of a damaged instrument.
- Damages caused by instruments used to screw, due to excess forces that occur during use.
- Increase in the time of operation due to dismounting of the instruments during surgery.

Due to the necessary mechanical functionalities, majority of the instruments (drills) are made in non-implantable materials. In case an instrument breaks, no fragment should remain inside the patient because it can cause post-operative complications, such as allergies, infections or complications of the biological kind, associated to the release of metallic components, possibly requiring future intervention.

In order to avoid environmental contamination or improper use of Implacil non-articulated cutters, when they are to be used, they should be disposed of in an appropriate place, that is, in a place for potentially contaminated materials.

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Não se aplica.

**EN** **Contraindications:** Not applicable.

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**EN** **Warranty Term:** IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees the owner of this product against any material or manufacturing defect that may appear on it within 365 days, counting from the date of purchase by the first buyer. The product will have indefinite validity if transported, stored and packed suitably according to this User Manual. Note: the warranty does not cover wear from product use.

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