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EN Technical Name: Dental Surgical Instruments / **Trade Name:** Surgical Instruments Non Articulated Non-Cutting

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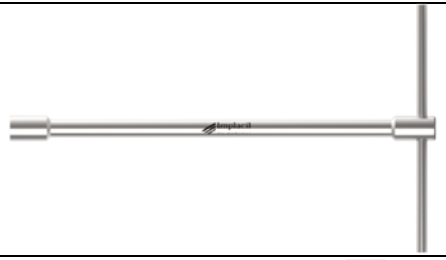
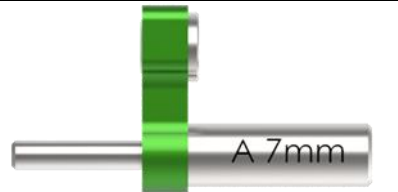
EN ATTENTION! Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

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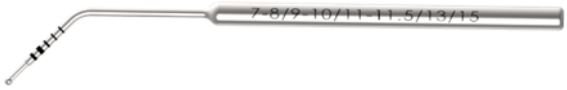
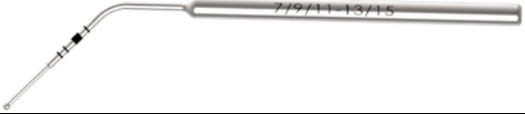
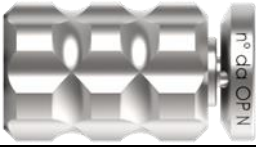
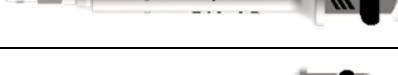
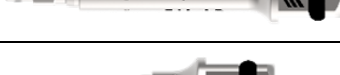
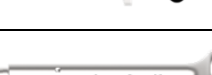
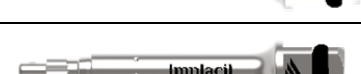
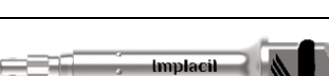
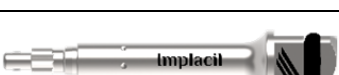
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
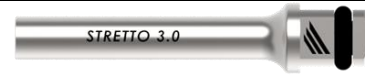
33626 - CHAVE PILAR STRETTO 3.0 EN STRETTO ABUTMENT DRIVER 3.0 ES LLAVE PILAR STRETTO 3.0 ITA CHIAVE PILAR STRETTO 3.0	
34558 - CHAVE TRANSPORTADORA PILAR STRETTO 3.0 EN STRETTO ABUTMENT TRANSPORTATION DRIVER 3.0 ES LLAVE DEL TRANSPORTADOR PILAR STRETTO 3.0 ITA CHIAVE TRASPORTATORE PILAR STRETTO 3.0	
33063 - CHAVE INSERCAO IMPLANTE STRETTO 3.0 EN STRETTO IMPLANT INSERTION DRIVER 3.0 ES LLAVE DE INSERCIÓN DEL IMPLANTE STRETTO 3.0 ITA CHIAVE DI INSERIMENTO IMPIANTO STRETTO 3.0	
34621-CHAVE CM AR TO (C) EN DRIVER CM AR TO (C) ES LLAVE CM AR TO (C) ITA CHIAVE CM AR TO (C)	
34622-CHAVE CM AR TO (M) EN DRIVER CM AR TO (M) ES LLAVE CM AR TO (M) ITA CHIAVE CM AR TO (M)	
34623-CHAVE CM AR TO (EL) EN DRIVER CM AR TO (EL) ES LLAVE CM AR TO (EL) ITA CHIAVE CM AR TO (EL)	
34624-CHAVE CM AR GUIDE TO (C) EN DRIVER CM AR GUIDE TO (C) ES LLAVE CM AR GUIDE TO (C) ITA CHIAVE CM AR GUIDE TO (C)	
34625-CHAVE CM AR GUIDE TO (L) EN DRIVER CM AR GUIDE TO (L) ES LLAVE CM AR GUIDE TO (L) ITA CHIAVE CM AR GUIDE TO (L)	
34646-CHAVE HI 3.5 TO (C) EN DRIVER HI 3.5 TO (C) ES LLAVE HI 3.5 TO (C) ITA CHIAVE HI 3.5 TO (C)	
34647-CHAVE HI 4.0 TO (C) EN DRIVER HI 4.0 TO (C) ES LLAVE HI 4.0 TO (C) ITA CHIAVE HI 4.0 TO (C)	
34648-CHAVE HI 3.5 TO (M) EN DRIVER HI 3.5 TO (M) ES LLAVE HI 3.5 TO (M) ITA CHIAVE HI 3.5 TO (M)	
34649-CHAVE HI 4.0 TO (M) EN DRIVER HI 4.0 TO (M) ES LLAVE HI 4.0 TO (M) ITA CHIAVE HI 4.0 TO (M)	
34650-CHAVE HE 3.5 TO (C) EN DRIVER HE 3.5 TO (C) ES LLAVE HE 3.5 TO (C) ITA CHIAVE HE 3.5 TO (C)	
34651 - CHAVE HE 4.0 TO (C) EN DRIVER HE 4.0 TO (C) ES LLAVE HE 4.0 TO (C) ITA CHIAVE HE 4.0 TO (C)	
34652-CHAVE HI 3.5 TO (L) EN DRIVER HI 3.5 TO (L) ES LLAVE HI 3.5 TO (L) ITA CHIAVE HI 3.5 TO (L)	
34653-CHAVE HI 4.0 TO (L) EN DRIVER HI 4.0 TO (L) ES LLAVE HI 4.0 TO (L) ITA CHIAVE HI 4.0 TO (L)	
34654-CHAVE HE 3.5 TO (L) EN DRIVER HE 3.5 TO (L) ES LLAVE HE 3.5 TO (L) ITA CHIAVE HE 3.5 TO (L)	
34655-CHAVE HE 4.0 TO (L) EN DRIVER HE 4.0 TO (L) ES LLAVE HE 4.0 TO (L) ITA CHIAVE HE 4.0 TO (L)	

34656 - CHAVE STRETTO 3.0 TO (L)
EN STRETTO DRIVER 3.0 TO (L)
ES LLAVE STRETTO 3.0 TO (L)
ITA CHIAVE STRETTO 3.0 TO (L)

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- 01 Instrumento Cirúrgico

EN Commercial Presentation Forms of the Product: Implacil Surgical Instruments are supplied in individual packs. The primary packaging consists of a blister produced with virgin PET plastic, and it is sealed with Tyvek.

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EN Indication of Use / Purpose: Implacil De Bortoli's non-articulated non-articulating surgical instruments are pieces produced in standard stainless steel, offered in various sizes and lengths, are used to assist in general dental surgeries.

None of the instruments that integrate this register has connection to equipment.

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Modelos Comerciais / Commercial Models / Modelos Comerciales / Modelli Commerciali	Indicação de Uso Específica / Indication for Specific Use / Indicación de Uso Específica / Indicazione di Uso specifico
34650-CHAVE HE 3.5 TO (C) EN DRIVER HE 3.5 TO (C) ES LLAVE HE 3.5 TO (C) ITA CHIAVE HE 3.5 TO (C) 34651 - CHAVE HE 4.0 TO (C) EN DRIVER HE 4.0 TO (C) ES LLAVE HE 4.0 TO (C) ITA CHIAVE HE 4.0 TO (C) 34654-CHAVE HE 3.5 TO (L) EN DRIVER HE 3.5 TO (L) ES LLAVE HE 3.5 TO (L) ITA CHIAVE HE 3.5 TO (L) 34655-CHAVE HE 4.0 TO (L) EN DRIVER HE 4.0 TO (L) ES LLAVE HE 4.0 TO (L) ITA CHIAVE HE 4.0 TO (L)	<p>Aplicar o torque necessário para a colocação de implantes HE.</p> <p>EN To apply the necessary torque for the insertion of HE implants.</p> <p>ES Aplicar el par de apriete necesario para la colocación de implantes HE.</p> <p>ITA Attuare il contatto necessario per inserzione di impianti HE.</p>

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EN The Implacil non-articulated non-cutting instruments should be used to aid dental surgeries. They should be previously hygienized and sterilized according to the techniques below:

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EN The cleaning of a surgical instrument that has been in contact with tissues must be conducted in such a way that all parts of the item are exposed to the cleaning process.

To clean Implacil instruments, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil instruments.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present in the item. Cold water is used, as temperatures above 60°C / 140 ° F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation *.

3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (steps 3 and 4) until no visible dirt is left.

* Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline, disinfectants, hydrogen peroxide or alcohol for cleaning or rinsing the cutters.

Never use steel wool or sponges and abrasive products, so that the cutters are not damaged.

Cleaning: Enzyme Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40 ° C to 55 ° C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a container covered with enzyme solution or appropriate detergent to delay drying.

The instruments must be replaced as soon as the professional realizes their diminished effectiveness or degradation of the item.

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EN It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form).

After the drying process, the instruments are sent to sterilization in an autoclave, using temperature and exposure time, according to the following method.

This product must be properly sanitized and sterilized before use. Please use for steam sterilization only the steam sterilization according to the following parameters:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

Surgical instruments must be placed inside the container (perforated in the lid and at the bottom), or tray or box; provided that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

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EN As raw material for surgical instruments, Stainless Steel is used according to ASTM F899 standard.

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EN STORAGE CONDITIONS: Keep it in the original packaging protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

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EN TRANSPORT CONDITIONS : The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

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EN MANIPULATION CONDITIONS : Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

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EN WARNINGS: When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility.

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EN PRECAUTIONS: The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient.

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes.

Extreme care must be taken when handling the instruments to prevent injuries. The “instrument sets” are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

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Não se aplica.

EN CONTRAINDICATIONS: Not applicable.

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EN Warranty Term: IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

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