**PORTUGUÊS**

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**PRODUCT**

Technical Name: Dental Surgical Instruments  
 Commercial Name: Surgical Instruments Non  
 Articulated Non-Cutting

**ATTENTION!**

**Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.**

**Commercial Models:**

Digital Driver  
 O'ring Driver  
 Friction Driver  
 Aesthetic Tapered Driver  
 Healing Abutment Driver  
 Short Digital Healing Abutment Driver  
 Long Digital Healing Abutment Driver  
 Cover Driver  
 Short Digital Cover Driver  
 Long Digital Cover Driver  
 UCLA Driver  
 Screwdriver  
 Short Digital Screwdriver  
 Long Digital Screwdriver  
 Mini Insertion Driver UNII 3,30  
 Medium Insertion Driver UNII 3,30  
 23748 - Mini Insertion Driver UNII 3,75  
 Medium Insertion Driver UNII 3,75  
 Type B Retractor  
 T Driver  
 Adaptor Driver  
 Anterior Parallelizer  
 Posterior Parallelizer  
 Retractor  
 Tamper  
 Mallet  
 Tunnel Check  
 Mount Driver Cm  
 Spiral Driver  
 SOS Driver  
 AR Morse Taper Abutment Driver (3,5; 4,5)  
 Square Driver (0.87; 0.9; 1.0; 1.1; 1.2; 1.3; 1.4; 1.5; 1.6;  
 1.7; 1.8; 1.9; 2.0)  
 Hexagonal Driver (0.87; 0.9; 1.0; 1.1; 1.2; 1.3; 1.4; 1.5;  
 1.6; 1.7; 1.8; 1.9; 2.0)  
 Morse Taper Abutment Driver (3,5; 4,5; 3,5x4; 3,5x6;  
 4,5x4; 4,5x6)  
 Slim Driver (2,8; 2,8x4; 2,8x6; 3.5x4; 3.5x6)  
 Tack Fixation Driver  
 AR Morse Taper Abutment Driver  
 Tunnel Check 2/3  
 Calibrated Tunnel Check 2/3  
 Measurement Probe  
 Guide Measurement Probe

Short Driver Insertion HE  
 Short Driver Insertion HI  
 Medium Driver Insertion HE  
 Medium Driver Insertion HI  
 Long Driver Insertion HE  
 Long Driver Insertion HI  
 Short Driver Insertion CM AR  
 Long Driver Insertion CM AR  
 Driver Insertion Morse Taper AR DUE CONE  
 Medium Digital Driver  
 Aesthetic Tapered Driver/Mini Tapered  
 Short Hexagonal Driver  
 Medium Hexagonal Driver  
 Long Hexagonal Driver  
 Short Square Driver  
 Medium Square Driver  
 Long Square Driver  
 Extractor Driver Cm AR  
 Orth Manual Driver  
 Mini AR Morse Taper Insertion Driver  
 Extra Long Driver Orth  
 32229 - DRIVER UNIVERSAL SMART/IDEALE 3.3/3.5 X 4  
 32230 - DRIVER UNIVERSAL SMART/IDEALE 3.3/3.5 X 6  
 32231 - DRIVER UNIVERSAL SMART/IDEALE 4.5 X 4  
 32232 - DRIVER UNIVERSAL SMART/IDEALE 4.5 X 6  
 30721 - SHORT TORX T6 DRIVER  
 30474 - MEDIUM TORX T6 DRIVER  
 30723 - LONG TORX T6 DRIVER  
 32737 - SHORT DIGITAL TORX T6 DRIVER  
 32738 - MEDIUM DIGITAL TORX T6 DRIVER  
 32739 - LONG DIGITAL TORX T6 DRIVER  
 33626 - STRETTO ABUTMENT DRIVER 3.0  
 34558 - STRETTO ABUTMENT TRANSPORTATION  
 DRIVER 3.0  
 34621 - DRIVER CM AR TO (C)  
 34622 - DRIVER CM AR TO (M)  
 34623 - DRIVER CM AR TO (EL)  
 34624 - DRIVER CM AR GUIDE TO (C)  
 34625 - DRIVER CM AR GUIDE TO (L)  
 34656 - STRETTO DRIVER 3.0 TO (L)  
 34646 - DRIVER HI 3.5 TO (C)  
 34647 - DRIVER HI 4.0 TO (C)  
 34648 - DRIVER HI 3.5 TO (M)  
 34649 - DRIVER HI 4.0 TO (M)  
 34650 - DRIVER HE 3.5 TO (C)  
 34651 - DRIVER HE 4.0 TO (C)  
 34652 - DRIVER HI 3.5 TO (L)  
 34653 - DRIVER HI 4.0 TO (L)  
 34654 - DRIVER HE 3.5 TO (L)  
 34655 - DRIVER HE 4.0 TO (L)

**1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT**

Implacil Surgical Instruments are supplied in individual packs. The primary packaging consists of a blister

produced with virgin PET plastic, and it is sealed with Tyvek.

List of items contained in the pack:

- 1 Surgical Instrument

## 2- INDICATION OF USE / PURPOSE

Implacil De Bortoli's non-articulated non-articulating surgical instruments are pieces produced in standard stainless steel, offered in various sizes and lengths, are used to assist in general dental surgeries.

None of the instruments that integrate this register has connection to equipment.

## 3- MODE OF PRODUCT USE

DIGITAL DRIVER: To apply the necessary torque for the insertion/removal of a certain prosthetic component.

O'RING DRIVER: To apply the necessary for the insertion/removal of the O'ring prosthetic component.

FRICTION DRIVER: To apply the necessary torque for the insertion/removal of the transfer prosthetic component.

AESTHETIC TAPERED DRIVER: To apply the necessary torque for the insertion/removal of the esthetic tapered and mini tapered prosthetic components

HEALING ABUTMENT DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment.

SHORT DIGITAL HEALING ABUTMENT DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment.

LONG DIGITAL HEALING ABUTMENT DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment.

COVER DRIVER: To apply the necessary torque for the insertion/removal of the cover (implant cover).

SHORT DIGITAL COVER DRIVER: To apply the necessary torque for the insertion/removal of the cover (implant cover).

LONG DIGITAL COVER DRIVER: To apply the necessary torque for the insertion/removal of the cover (implant cover).

UCLA DRIVER: To apply the necessary torque for the insertion/removal of the UCLA prosthetic component.

SCREWDRIVER: To remove/insert a slotted-head screw.

SHORT DIGITAL SCREWDRIVER: To remove/insert a slotted-head screw.

LONG DIGITAL SCREWDRIVER: To remove/insert a slotted-head screw.

MINI INSERTION DRIVER UNII 3,30: To apply the necessary torque for the insertion of HI implants.

MEDIUM INSERTION DRIVER UNII 3,30: To apply the necessary torque for the insertion of HI implants.

23748 - MINI INSERTION DRIVER UNII 3,75: To apply the necessary torque for the insertion of HI implants.

MEDIUM INSERTION DRIVER UNII 3,75: To apply the necessary torque for the insertion of HI implants.

TYPE B RETRACTOR: To release the access to the procedure site.

T DRIVER: To apply the necessary torque for the insertion/removal of a certain prosthetic component.

ADAPTOR DRIVER: To connect the different types of drivers.

ANTERIOR PARALLELIZER: To ensure the parallelism between the cavities where the implants are to be placed.

POSTERIOR PARALLELIZER: To ensure the parallelism between the cavities where the implants are to be placed.

RETRACTOR: To enable the release of the access to the procedure site

TAMPER: Used with the mallet to push the bone of the sinus window.

MALLET: Used to hit the tamper.

TUNNEL CHECK: Used to check the depth and parallelism.

MOUNT DRIVER CM: Used to release the morse taper implant model with the mount.

SPIRAL DRIVER: Used to screw the cover of the morse taper implant model.

SOS DRIVER: Used to release the mount of the morse taper implant.

AR MORSE TAPER ABUTMENT DRIVER (3,5; 4,5): To apply the necessary torque for the insertion/removal of the AR morse taper abutment.

SQUARE DRIVER (0,87; 0,9; 1,0; 1,1; 1,2; 1,3; 1,4; 1,5; 1,6; 1,7; 1,8; 1,9; 2,0): To apply the necessary torque for the insertion/removal of the UCLA and abutment prosthetic components.

HEXAGONAL DRIVER (0,87; 0,9; 1,0; 1,1; 1,2; 1,3; 1,4; 1,5; 1,6; 1,7; 1,8; 1,9; 2,0): To apply the necessary torque for the insertion/removal of the healing abutment (implant cover).

MORSE TAPER ABUTMENT DRIVER (3,5; 4,5; 3,5X4; 3,5X6; 4,5X4; 4,5X6): To apply the necessary torque for the insertion/removal of the morse taper abutment.

SLIM DRIVER (2,8; 2,8X4; 2,8X6; 3,5X4; 3,5X6): To apply the necessary torque for the insertion/removal of the slim implant.

Tack Fixation Driver: Used to attach the tack

AR MORSE TAPER ABUTMENT DRIVER: To apply the necessary torque for the insertion/removal of the AR morse taper implant.

TUNNEL CHECK 2/3: To check the depth and parallelism.

CALIBRATED TUNNEL CHECK 2/3: Used to check the depth and parallelism (scaled).

MEASUREMENT PROBE: To check the internal bone depth and walls after drilling.

Guide Measurement Probe: To check the internal bone depth and walls after drilling.

SHORT DRIVER INSERTION HE: To apply the necessary torque for the insertion of HE implants

SHORT DRIVER INSERTION HI: To apply the necessary torque for the insertion of HI implants

MEDIUM DRIVER INSERTION HE: To apply the necessary torque for the insertion of HE implants

MEDIUM DRIVER INSERTION HI: To apply the necessary torque for the insertion of HI implants

LONG DRIVER INSERTION HE: To apply the necessary torque for the insertion of HE implants

LONG DRIVER INSERTION HI: To apply the necessary torque for the insertion of HI implants

SHORT DRIVER INSERTION CM AR: To apply the necessary torque for the insertion of morse taper implants AR

LONG DRIVER INSERTION CM AR: To apply the necessary torque for the insertion of morse taper implants AR

DRIVER INSERTION MORSE TAPER AR DUE CONE: To apply the necessary torque for the insertion of morse taper implants AR.

MEDIUM DIGITAL DRIVER: To apply the necessary torque for the insertion/removal of a certain prosthetic component.

AESTHETIC TAPERED DRIVER/MINI TAPERED: To apply the necessary torque for the insertion/removal of the esthetic tapered and mini tapered prosthetic components.

SHORT HEXAGONAL DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment (implant cover).

MEDIUM HEXAGONAL DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment (implant cover).

LONG HEXAGONAL DRIVER: To apply the necessary torque for the insertion/removal of the healing abutment (implant cover).

SHORT SQUARE DRIVER: To apply the necessary torque for the insertion/removal of the UCLA and abutment prosthetic components

MEDIUM SQUARE DRIVER: To apply the necessary torque for the insertion/removal of the UCLA and abutment prosthetic components.

LONG SQUARE DRIVER: To apply the necessary torque for the insertion/removal of the UCLA and abutment prosthetic components.

EXTRACTOR DRIVER CM AR: Extract the morse taper component from of the implant.

ORTH MANUAL DRIVER: To apply the necessary torque for the insertion/removal of the ORTH implants.

MINI AR MORSE TAPER INSERTION DRIVER: To apply the necessary torque for the insertion of AR morse taper implants.

EXTRA LONG DRIVER ORTH: Connects the Orth Manual Driver and serves to apply the necessary torque for the insertion/removal of the ORTH implants.

32229 - DRIVER UNIVERSAL SMART/IDEALE 3.3/3.5 X 4;

32230 - DRIVER UNIVERSAL SMART/IDEALE 3.3/3.5 X 6;

32231 - DRIVER UNIVERSAL SMART/IDEALE 4.5 X 4;

32232 - DRIVER UNIVERSAL SMART/IDEALE 4.5 X 6;

Apply the necessary torque for the placement of the Smart and Ideale abutments.

30721 - SHORT TORX T6 DRIVER; 30474 - MEDIUM

TORX T6 DRIVER; 30723 - LONG TORX T6 DRIVER; 32737

- SHORT DIGITAL TORX T6 DRIVER; 32738 - MEDIUM

DIGITAL TORX T6 DRIVER; 32739 - LONG DIGITAL TORX

T6 DRIVER: Apply the necessary torque for placing the

TORX screws.

33626 - STRETTO ABUTMENT DRIVER 3.0; 34656 -

STRETTO DRIVER 3.0 TO (L): Apply the necessary torque for placing the STRETTO Abutments

34558 - STRETTO ABUTMENT TRANSPORTATION DRIVER 3.0: Transport the Stretto abutment until its insertion

34621 - DRIVER CM AR TO (C); 34622 - DRIVER CM AR

TO (M); 34623 - DRIVER CM AR TO (EL): To apply the

necessary torque for the insertion of AR morse taper implants.

34624 - DRIVER CM AR GUIDE TO (C); 34625 - DRIVER

CM AR GUIDE TO (L): To apply the necessary torque for

the insertion of AR morse taper implants with the assistance of the kit guide.

34646 - DRIVER HI 3.5 TO (C); 34647 - DRIVER HI 4.0 TO

(C); 34648 - DRIVER HI 3.5 TO (M); 34649 - DRIVER HI

4.0 TO (M); 34652 - DRIVER HI 3.5 TO (L); 34653 -

DRIVER HI 4.0 TO (L): To apply the necessary torque for

the insertion of HI implants.

34650 - DRIVER HE 3.5 TO (C); 34651 - DRIVER HE 4.0

TO (C); 34654 - DRIVER HE 3.5 TO (L); 34655 - DRIVER

HE 4.0 TO (L): To apply the necessary torque for the

insertion of HE implants.

NOTE: None of these instruments are connected to the equipments.

The Implacil non-articulated non-cutting instruments should be used to aid dental surgeries. They should be previously hygienized and sterilized according to the techniques below:

#### **a) Cleaning**

The cleaning of a surgical instrument that has been in contact with tissues must be conducted in such a way that all parts of the item are exposed to the cleaning process.

To clean Implacil instruments, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil instruments.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present in the item. Cold water is used, as temperatures above 60°C / 140 ° F will coagulate proteins, making it difficult to remove them from contaminated items.
2. Prepare the enzymatic detergent according to the recommendation \*.
3. Dip the products in the prepared detergent solution and let it soak for at least 5 minutes. While soaking, brush the items with a soft bristle brush.
4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) under running water.
5. Visually inspect each product for visible dirt. If any visible dirt remains, repeat the cleaning process (steps 3 and 4) until no visible dirt is left.

\* Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline, disinfectants, hydrogen peroxide or alcohol for cleaning or rinsing the cutters.

Never use steel wool or sponges and abrasive products, so that the cutters are not damaged.

Cleaning: Enzyme Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40 ° C to 55 ° C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a container covered with enzyme solution or appropriate detergent to delay drying.

The instruments must be replaced as soon as the professional realizes their diminished effectiveness or degradation of the item.

#### **b) Sterilization**

It is a procedure that aims at the total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form).

After the drying process, the instruments are sent to sterilization in an autoclave, using temperature and exposure time, according to the following method.

This product must be properly sanitized and sterilized before use. Please use for steam sterilization only the steam sterilization according to the following parameters

<b>Sterilization Parameters</b>	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

Surgical instruments must be placed inside the container (perforated in the lid and at the bottom), or tray or box; provided that they can be prepared for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

#### **4- COMPOSITION**

As raw material for surgical instruments, Stainless Steel is used according to ASTM F899 standard.

#### **5- EXPIRY DATE**

05 years from date of manufacture

#### **6- STORAGE CONDITIONS**

Keep it in the original packaging protected from direct sunlight and rain.

The product in question is not thermosensitive and does not present any variation in environments with high or low humidity levels; therefore, it can be sterilized in an autoclave. Even so, it is recommended to store the product between 15° and 35° C and humidity between 45% and 80% in order to ensure the stability of the packaging.

#### **7- TRANSPORT CONDITIONS**

The instruments should be transported in a clean and dry place at room temperature. The transport should be performed appropriately to avoid falls and damages to their original packaging.

#### **8- MANIPULATION CONDITIONS**

Handling should be performed with the proper precautions to avoid falls and damages, which must be performed on shelves and counters according to the activity that is being executed.

#### **9- WARNINGS**

When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are NOT sterile and should be sterilized before use. Instruments should be autoclaved

inside the instrument box. When not packaged, instrument boxes do NOT maintain their sterility.

#### **10- PRECAUTIONS**

The Implacil instruments must be used only by professionals specialized in the field. It is necessary to elaborate a surgery execution plan and also another type of procedure to prevent future complications before any intervention on the patient.

Maximum care must be taken in the cleaning, sterilization, accommodation (storage), transport and handling processes of the product to avoid mechanical impacts that change their anatomy and compromise their function. Check the functioning of the instrument before initiating surgery. The product should not be used if it presents anatomical changes.

Extreme care must be taken when handling the instruments to prevent injuries. The "instrument sets" are not sterile and must be autoclaved in cases or appropriate envelopes, according to the instructions for use. The instrument sets do not maintain the sterility if they are not packaged in envelopes suitable for autoclaving.

#### **11- CONTRAINDICATIONS**

Not applicable

#### **12- ADVERSE EFFECTS**

Not applicable

#### **13- WARRANTY TERM**

IMPLACIL DE BORTOLI - MATERIAL ODONTOLÓGICO S.A. guarantees this product against any material or manufacturing defect that may appear within 365 days, counting from the original date of purchase.

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