PORTUGUÊS

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PRODUCT

Technical Name: Instruments for dental use
Commercial Name: Non-cutting articulated surgical instruments

ATTENTION!

Do not re-utilize the packaging if the seal is broken or damaged. Non-sterile product. Sterilize before use. Reusable. Clean and sterilize before reuse.

Commercial Models:

Unit torque wrench (10, 20, 30 or 50 Ncm)
Multiple torque wrench (10 to 100 Ncm)
Ratchet
Torque Wrench 20 to 80 Ncm
Torque Wrench 10 to 40 Ncm
Torque Wrench 40 to 80 Ncm
Torque Rod 20 to 80 Ncm
Z-Pillar Torque Referrer
19842 – Torque Wrench 30 to 80 Ncm

1- COMMERCIAL PRESENTATION FORMS OF THE PRODUCT

Implacil Surgical Instruments are supplied in individual packs. The packaging consists of a Surgical Package Envelope.

List of items contained in the pack:
- 1 Surgical Instrument

2- INDICATION OF USE / PURPOSE

The Surgical Instruments are used in dental surgery in general.

3- INSTRUCTIONS FOR PRODUCT USE

Unit torque wrench (10, 20, 30 or 50 Ncm): Used for providing exact torque on abutment screws or implants

Multiple torque wrench (10 to 100 Ncm): Used for applying variable torque (from 10 to 100 Ncm) on abutment screws or implants.

Ratchet: Used for placement of implants.

Torque Wrench 20 to 80 Ncm: Used to apply variable torque (from 20 to 80 Ncm) to prosthetic screws or implants. Stainless steel type 316L.

Torque Wrench 10 to 40 Ncm: Used to apply variable torque (from 10 to 40 Ncm) to prosthetic screws or implants. Stainless steel type 316L.

Torque Wrench 40 to 80 Ncm: Used to apply variable torque (from 40 to 80 Ncm) to prosthetic screws or implants. Stainless steel type 316L.

Torque Rod 20 to 80 Ncm: Used to apply variable torque (from 20 to 80 Ncm) to prosthetic screws or implants. Stainless steel type 316L.

Z-Pillar Torque Referrer: Used to torque the Z-Pillar.

19842 – Torque Wrench 30 to 80 Ncm: Used to apply variable torque (from 30 to 80 Ncm) to prosthetic screws or implants. Stainless steel type 316L.

The Surgical Instruments should be used to assist in dental surgery. They should be sanitized and sterilized beforehand according to the following techniques:

A) Cleaning: The cleaning of a surgical instrument that has been in contact with tissue must be conducted in such a way that all parts of the item are exposed to the cleaning process.

For cleaning Implacil surgical instruments use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil surgical instruments.

1. Initial rinse with cold water. The initial cold rinse is conducted in order to remove any gross contamination present on the item. Cold water is used as temperatures above 60°C/140°F will coagulate proteins, making it difficult to remove them from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation*.

3. Soak the products in the prepared detergent solution and soak for at least 5 minutes. While soaking, brush items with a soft-bristle brush.

4. Rinse: after removing the instruments from the detergent solution, rinse thoroughly for at least 3 times (for at least 1 minute) in running water.

5. Visually inspect each product for visible dirt. If there is any visible dirt, repeat the cleaning process (step 3 and 4) until no more visible dirt is left.

*Recommendation for the cleaning step

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the instruments.

Never use steel wool or sponges and abrasive products, so that the instruments are not damaged.

Cleaning: Enzymatic Detergent Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzyme solution or appropriate detergent to delay drying.

Surgical instruments must be replaced as soon as the professional perceives their reduced effectiveness.

B) Sterilization: Sterilization is a procedure designed for total elimination of microorganisms, such as: viruses, bacteria, microbes, fungi (either in vegetative or sporulated form). After the drying process, the instruments are sent for sterilization in an autoclave, using temperature and exposure time according to the following method:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization according to the following parameters for sterilization:

Sterilization Parameters	
Temperature	132°C
Sterilization time	20 min
Drying time	20 - 60 min

DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam out first, making the Drying Cycle complete by itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

4- COMPOSITION

As raw material for surgical instruments, Stainless Steel.

5- EXPIRY DATE

Undetermined.

6- STORAGE CONDITIONS

Store at room temperature, in a cool place, away from light and heat. Avoid exposure to heat and excessive humidity.

Non-cutting articulated surgical instruments must be stored clean, decontaminated and completely dry. The packaging in which items are sterilized can provide an effective barrier to maintain the sterility of the item. After use, they must be stored in a clean, dry and cool place..

7- TRANSPORT CONDITIONS

Non-cutting articulated surgical instruments must be transported in a clean, dry place at room temperature. Transport must be done properly to avoid falling and damage to its original packaging..

8- MANIPULATION CONDITIONS

Handling must be carried out taking due care to avoid falls and damage. The same must be carried out on shelves and counters, according to the activity being carried out.

9- WARNINGS

Particular care should be taken in the cleaning, sterilization, accommodation (stockpiling), transportation and handling of the product to avoid mechanical shocks that change its anatomy and affect its functionality. Before the start of the surgery verify the product operation. The product should not be used if it exhibits anatomical alterations.

10- PRECAUTIONS

When handling the instruments, use extreme caution in order to avoid injuries. Unless indicated otherwise, sets of instruments are not sterile and should be sterilized before use. Instruments should be autoclaved inside the instrument box pursuant to item 4b. Instrument boxes removed from their packaging do not remain sterile. The Surgical Instruments should be replaced as soon as the professional perceives their diminished efficacy. For exclusive use by dental professionals.

10.1 - PRECAUTIONS IN CASE OF DISPOSAL: To avoid environmental contamination or improper use of the Surgical Instruments, these must be disposed of in an appropriate place, that is, in a specific place for potentially contaminated materials.

11- CONTRAINDICATIONS

Incorrect cleaning and maintenance may render the instruments inappropriate for their intended use, provoke corrosion, disassembly, distortion and/or breakage or also cause injuries to the patient or operation team. See below a list of potential complications:

- Lesion or pain in the soft tissues in case of incorrect use or breakage of the instruments.
- Infection, if the instruments are not properly cleaned and sterilized.
- Damage to the adjacent nerves or organs, as a result of slipping or incorrect positioning of a damaged instrument.
- Damage caused by the instruments used for screwing due to excessive forces that occur during use.
- Increase of surgical time due to the need to dismantle the instruments during surgery.
- Due to necessary mechanical functionalities, most instruments are made from non-implantable materials. In the eventuality of an instrument breaking, no fragment should remain in the patient, as it may cause postoperative complications, such as allergies, infections, or complications of a biological nature, associated with the release of metal components, possibly requiring subsequent intervention.

12- ADVERSE EFFECTS

Not applicable

13- WARRANTY TERM

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO
S.A. guarantees this product against any material or
manufacturing defect that may appear within 365 days,
counting from the original date of purchase.

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