PORTUGUÊS

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PRODUCT

Technical Name: Instrumental Kit

Commercial Name: PROSTHETIC SELECT KIT CONE MORSE

ATTENTION!!!

Do not use if the packaging is broken or damaged.

Non-Sterile Product. Sterilization is required before use.

Product subject to reprocessing.

Commercial Models:

34846 - PROSTHETIC SELECT KIT CM
34847 - PROSTHETIC SELECTION CM CASE
34818 - COLLAR SELECT CM 0.8
34819 - COLLAR SELECT CM 1.5
34820 - COLLAR SELECT CM 2.5
34821 - COLLAR SELECT CM 3.5
34822 - COLLAR SELECT CM 4.5
34823 - COLLAR SELECT CM 5.5
34824 - IDEALE ABUTMENT 3.3X4-SELECT
34825 - IDEALE ABUTMENT 3.3X6-SELECT
34826 - IDEALE ABUTMENT 4.5X4-SELECT
34827 - IDEALE ABUTMENT 4.5X6-SELECT
34828 - IDEALE ABUTMENT ANGLE 4X17°-SELECT
34829 - IDEALE ABUTMENT ANGLE 4X30°-SELECT
34830 - IDEALE ABUTMENT ANGLE 6X17°-SELECT
34831 - IDEALE ABUTMENT ANGLE 6X30°-SELECT
34832 - COPING IDEALE ABUTMENT 3.3X4-SELECT
34833 - COPING IDEALE ABUTMENT 3.3X6-SELECT
34834 - COPING IDEALE ABUTMENT 4.5X4-SELECT
34835 - COPING IDEALE ABUTMENT 4.5X6-SELECT
34836 - COPING MICRO TAPERED SELECT
34837 - COPING MINI TAPERED SELECT
34838 - MINI TAPERED SELECT
34839 - MINI TAPERED ANGLE 17°-SELECT
34840 - MINI TAPERED ANGLE 30°-SELECT
34841 - MICRO TAPERED SELECT
34842 - T BASE ABUTMENT - 3.5-SELECT
34843 - T BASE ABUTMENT - 4.5-SELECT
34849 - DRIVER COLLAR SELECT
34850 - TUNNEL CHECK IMPLANTE CONE MORSE-SELECT
34851 - DRIVER HEX 1.17-SELECT

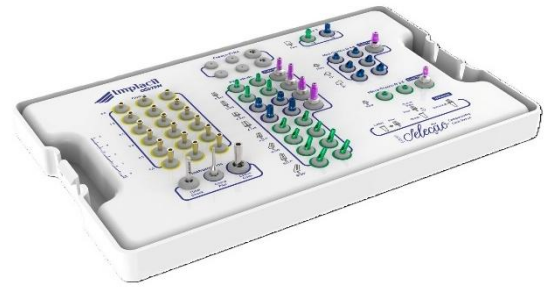
1- INDICATION OF USE AND PURPOSE

The Implacil PROSTHETIC SELECT KIT CONE MORSE are pieces offered in various sizes and lengths, used to assist in the selection and planning of prosthetic components, are not implantable, and are for temporary use.

Its purpose is to assist in the selection of the transmucosal, type of component, angulation, diameter and height of the element to be used.

This Kit consists of Collar, Abutment, Copings, Mini Tapered, Micro Tapered, Driver and Tunnel Check..

2- HOW TO USE

PRODUCT FOR DENTAL USE ONLY
NON-STERILE PRODUCT - AUTOCLAVABLE
STERILIZE BEFORE USE

- Select the prosthetic selection abutments to be used in advance;
- Check that the packaging is in perfect condition. Reject the product if the packaging is damaged.

They must be previously cleaned and sterilized using the following techniques:

a) Cleaning:

Cleaning must be carried out in such a way that all parts of the item - products that make up the case - are exposed to the cleaning process.

To clean the Implacil PROSTHETIC SELECT KIT CONE MORSE, use only the following manual cleaning process.

Automated cleaning has not been validated. Do not use automated cleaning for Implacil cases.

1. Initial rinse with cold water. The initial cold rinse is carried out in order to remove any gross contamination present on the items. Cold water is used, as temperatures above 60°C/140°F will coagulate proteins, making them difficult to remove from contaminated items.
2. Prepare the enzymatic detergent according to the recommendation*.
3. Submerge the products in the prepared detergent solution and let soak for at least 5 minutes. While soaking, brush the items with a soft-bristled brush.
4. Rinse: After removing the cases from the detergent solution, rinse them intensively at least 3 times (for at least 1 minute) under running water.
5. Visually inspect each product for visible soil. If any visible soil remains, repeat the cleaning process (steps 3 and 4) until no more visible soil remains.

*Recommendation for the cleaning step:

Always use distilled, deionized or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide or alcohol to clean or rinse the burs.

Never use steel wool or sponges or abrasive products, so that the burs are not damaged.

Cleaning: Enzymatic Detergent - Composition: Protease, Lipase and Amylase enzymes

Dilution: 1mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted in such a way that all parts of the cases and their components are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean the cases and their components as soon as possible after use. Do not allow blood and residue to dry on cases. If cleaning needs to be postponed, place the case in a covered container with an enzymatic solution or appropriate detergent in order to delay drying.

The cases should be replaced as soon as the professional notices their reduced effectiveness or degradation. According to the manufacturer of the raw material, the cases withstand more than 1,000 steam sterilization cycles at 134 °C without any significant loss of properties.

b) Sterilization:

This product must be properly sanitized and sterilized before use. Please use only steam sterilization for sterilization according to the following Sterilization Parameters:

Temperature: 132°C

Sterilization time: 20 min

Drying time: 20 - 60 min

DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam escape first, allowing the Drying Cycle to complete itself. Thoroughly and periodically clean the autoclave, removing dirt and excess rust (iron oxide).

Finally, adapt the PROSTHETIC SELECT KIT CONE MORSE to the implant you want to check the prosthesis model to be used.

3.FORMS OF COMMERCIAL PRESENTATION OF THE PRODUCT

The PROSTHETIC SELECT KIT CONE MORSE, with reference code 34846, is sold non-sterile and reusable, packaged in an autoclavable case (BASE, LID, BOTTOM AND RUBBER). The packaging consists of a cardboard box.

It consists of the following items, supported on an appropriate autoclavable tray: Belts, Abutments, Copings, Mini Conical, Micro Conical, Keys and Tunnel Check.

This instruction for use will be available in electronic format on the Implacil website (www.implacil.com.br) in accordance with current legislation on the provision of instructions for use in non-printed format for health

products – RDC No. 751/2022. The printed instruction for use will be available to the customer who requests it, and will be sent free of charge. The printed instruction for use can be requested through the Customer Service – 0800.779.0011, and will be sent at no additional cost.

To locate it, check the “IFU” number and the version of the instruction that is indicated on the label of the product purchased. To read the instructions for use on the website, the Adobe Acrobat Reader program is required.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as made available by Implacil.

4. PHYSICAL PRINCIPLE AND FUNDAMENTALS OF THE PRODUCT'S TECHNOLOGY, APPLIED TO ITS OPERATION AND ACTION:

Planning is undoubtedly one of the most important phases of treatment and a determining factor in the success of implant-supported prostheses.

With the PROSTHETIC SELECT KIT CONE MORSE, it is possible to analyze:

- Diameter;
- Type of cemented or screw system;
- Straight or angled.

The following criteria must be observed when planning a treatment involving implant-supported prostheses, namely:

- Ideal position of the crown in the arch;
- Ideal position of the implant (taking as a reference the already established position of the crown).
- Selection of the most appropriate component to obtain the best crown/implant connection result.

5. COMPOSITION

As raw material for the Kit, most of them are composed of Aluminum, with the exception of the Keys, Tunnel Check, which are composed of Stainless Steel.

6. VALIDITY PERIOD

Undetermined

7. SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND HANDLING OF THE PRODUCT:

Keep in its original packaging in a place protected from direct sunlight and rain.

The product in question is not heat-sensitive and does not present any variation in environments with high or low levels of humidity, which is why it can be sterilized in an autoclave.

Even so, it is recommended that the product be kept between 15° and 35° C and at a humidity between 45% and 80%, to ensure the stability of the packaging.

After use, the PROSTHETIC SELECT KIT CONE MORSE must be stored clean, decontaminated and completely dry.

The packaging in which the items are sterilized can provide an effective barrier to maintain the sterility of the item.

The Kit must be transported in a clean, dry place at room temperature.

Transportation must be done properly, to avoid falling and damage to its original packaging.

The utmost care must be taken in the process of cleaning, sterilization, storage, transportation and handling of the product to avoid mechanical shocks that alter its anatomy and impair its functionality.

Before starting the surgery, check that the product is working properly.

It should not be used if it presents anatomical alterations.

Exclusive use by dental professionals.

8. WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED

Exclusive use by dental professionals. The product is not sterile, and this procedure must be performed by the professional before any intervention on the patient.

Do not use it under any circumstances if the packaging is broken. The product is not sterile, and this procedure must be performed by the professional before any intervention on the patient.

Incorrect maintenance, cleaning and maintenance may render the instruments unsuitable for their intended use, cause corrosion, disassembly, distortion and/or breakage or even cause injury to the patient or operating team.

9. CONTRAINDICATIONS

Not applicable

10. ADVERSE EFFECTS

Not applicable

11. PRECAUTIONS IN CASE OF DISPOSAL

To avoid environmental contamination or misuse of Prosthetic Abutments, when they are no longer used, they must be disposed of in hospital waste or in a location for potentially contaminated materials.

12. WARRANTY TERMS

IMPLACIL DE BORTOLI - MATERIAL ODONTOLOGICO SA guarantees the owner of this product a guarantee against any material or manufacturing defect that may arise within 365 days, counting from the date of purchase by the first purchaser.

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