PORTUGUÊS

Implante Ortodôntico

INGLÊS

Implant Orthodontic

ESPAÑHOL

Implante Ortodoncia

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Revisão: 08

PRODUCT:

Technical Name: Orthodontic Fixation Screw

Commercial Name: Implant Orthodontic

ATTENTION!!!

SINGLE-USE PRODUCT.

DO NOT USE IF PACKAGING IS BROKEN OR DAMAGED.

STERILE PRODUCT. DO NOT RESTERILIZE. STERILIZED BY GAMMA RADIATION.

Commercial Model and reference code:

Commercial Model A:

- 2599-7 - Implant Orthodontic 1.5 X 7.0 MM
- 2600-0 - Implant Orthodontic 1.5 X 9.0 MM
- 2601-7 - Implant Orthodontic 1.5 X 11.0 MM
- 2602-4 - Implant Orthodontic 1.8 X 7.0 MM
- 2603-1 - Implant Orthodontic 1.8 X 9.0 MM
- 2604-8 - Implant Orthodontic 1.8 X 11.0 MM
- 2605-5 - Implant Orthodontic 2.0 X 7.0 MM
- 2606-2 - Implant Orthodontic 2.0 X 9.0 MM
- 2607-9 - Implant Orthodontic 2.0 X 11.0 MM
- 23023 - Implant Orthodontic Bracket D 1.5 X 7.0 MM
- 23033 - Implant Orthodontic Bracket D 1.5 X 9.0 MM
- 23034 - Implant Orthodontic Bracket D 1.5 X 11.0 MM
- 23035 - Implant Orthodontic Bracket E 1.5 X 7.0 MM
- 23036 - Implant Orthodontic Bracket E 1.5 X 9.0 MM
- 23037 - Implant Orthodontic Bracket E 1.5 X 11.0 MM
- 23246 - Implant Orthodontic Bracket D 1.8 X 7.0 MM
- 23247 - Implant Orthodontic Bracket D 1.8 X 9.0 MM
- 23248 - Implant Orthodontic Bracket D 1.8 X 11.0 MM
- 23249 - Implant Orthodontic Bracket E 1.8 X 7.0 MM
- 23250 - Implant Orthodontic Bracket E 1.8 X 9.0 MM
- 23251 - Implant Orthodontic Bracket E 1.8 X 11.0 MM
- 23252 - Implant Orthodontic Bracket D 2.0 X 7.0 MM
- 23253 - Implant Orthodontic Bracket D 2.0 X 9.0 MM
- 23254 - Implant Orthodontic Bracket D 2.0 X 11.0 MM
- 23255 - Implant Orthodontic Bracket E 2.0 X 7.0 MM
- 23256 - Implant Orthodontic Bracket E 2.0 X 9.0 MM
- 23257 - Implant Orthodontic Bracket E 2.0 X 11.0 MM

Commercial Model B:

- 254922 - Implant Self Drilling 1,5 X 3,0MM
- 254946 - Implant Self Drilling 1,5 X 4,0MM
- 254960 - Implant Self Drilling 1,5 X 5,0MM
- 254984 - Implant Self Tapping Boader Head 3,5 X 1,5 X 8,0MM
- 255004 - Implant Self Tapping Boader Head 3,5 X 1,5 X 10,0MM
- 32233 - Implant Self Tapping Boader Head 4.6X1.5X8.0MM
- 32235 - Implant Self Tapping Boader Head 4.6X1.5X10.0MM
- 32237 - Implant Self Tapping Boader Head 5.4X1.5X8.0MM
- 32239 - Implant Self Tapping Boader Head 5.4X1.5X10.0MM
- 30301 - Implant Self Tapping Boader Head 5.7X1.5X8.0MM
- 30303 - Implant Self Tapping Boader Head 5.7X1.5X10.0MM
- 255028 - Implant Fully Threaded 1,5 X 8,0MM
- 255042 - Implant Fully Threaded 1,5 X 10,0MM

1. INDICATION OF USE AND PURPOSE

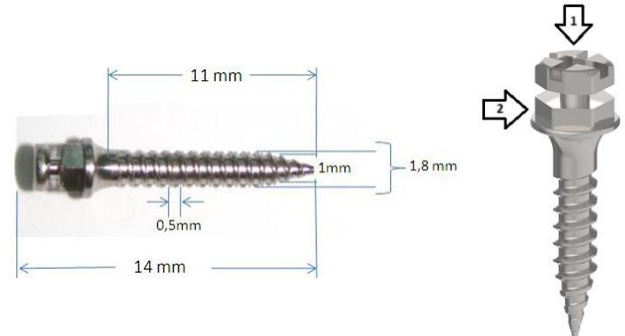
The Bortoli Implacil Orthodontic Implant is indicated for temporary fixation to the maxillary or mandibular bone, providing assistance for orthodontic movements through increased anchorage.

It is produced in ASTM F136 Titanium Alloy and sterilized by gamma radiation.

2. HOW TO USE: DETAILED DESCRIPTION

Commercial Model A:

Implant Orthodontic / Implant Orthodontic Bracket

The Implant Orthodontic's Implacil consist of a cylindrical body with threads (a) that aid their installation and anchoring in the bone bed along their entire length. Their head has a hexagonal shape for fitting the manual placement and drilling key for fixing orthodontic wire.

The Implant Orthodontic Bracket D/E consist of a cylindrical body with threads that aid their installation and anchoring in the bone bed along their entire length. Their head has a cross-shaped fitting (1) to receive the orthodontic wires and a hexagonal fitting (2) below the cross platform for fitting the manual fixation key.

The Implant Orthodontic Bracket D have threads that aid their installation arranged in a clockwise direction (Left to Right), this way when the orthodontic wires pass through there is no risk of the implant becoming unscrewed.

The Implant Orthodontic Bracket have threads that help their installation arranged in an anti-clockwise direction (Right to Left), so when the orthodontic wires pass through there is no risk of the implant coming loose.

It is produced by turning titanium alloy bars, meeting the ASTM F136 standard.

Note: Direction of the threads that define whether the implant is Implant Orthodontic Bracket D or E

Modelo Comercial B:

Implant Self Drilling

Implant Self Tapping Boader Head

Implant Fully Threaded

The Implant ORTH's Implacil consist of a cylindrical body with threads that aid its installation and anchoring in the bone bed along its entire length.

Its head has a Philips screwdriver fitting.

The Implant Self Drilling consists of a cylindrical body with threads at the distal end that aid its installation before reaching the proximal end and has a smooth neck.

The Implant Self Tapping Boader Head consists of a cylindrical body with threads that aid its installation, and has an expanded head to improve fixation.

2.1 SPECIFICATIONS AND TECHNICAL CHARACTERISTICS:

The Implant Orthodontic's Implacil are packaged in a 50x80 Tyvek sealed blister. They are available in diameters of 1.5mm, 1.8mm and 2.0mm and in lengths of 7mm, 9mm and 11mm.

The Implant Orthodontic's Implacil is for single use.

They are available in diameters of 1.5mm and in lengths of 3mm, 4mm, 5mm, 8mm and 10mm.

The Implants ORTH Expanded Head model have the following dimensional variations:

Head Diameter (mm)	Body Diameter (mm)	Length (mm)
3,5 / 4,6 / 5,4 / 5,7	1,5	8 and 10

3. PRODUCT COMMERCIAL PRESENTATION FORM

The Implants ORTH's Implacil are packaged in a 50x80 Tyvek-sealed blister. The product is individually packaged. Each package consists of:

- 01 Implant ORTH's Implacil
- 03 traceability labels

These instructions for use will be available on the Implacil website (www.implacil.com.br) in accordance with current legislation on the provision of instructions for use in non-printed format for health products – RDC 751/2022. The printed instructions for use will be available to the customer who requests them and will be sent free of charge.

The printed instructions for use can be requested through Customer Service – 0800.779.0011 and will be sent at no additional cost.

To locate it, check the "IFU" number and the version of the instructions indicated on the label of the product

purchased. To read the instructions for use on the website, you need Adobe Acrobat Reader.

User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as made available by Implacil .

4. PHYSICAL PRINCIPLE AND FUNDAMENTALS OF THE PRODUCT TECHNOLOGY, APPLIED TO ITS OPERATION AND ACTION:

The operating principle of the Implant Orthodontic Implacil is the biocompatibility of the material in which it is produced. The conical thread is designed to be self-drilling and conveniently distribute the forces to which it is subjected.

The Implant Orthodontic is introduced into the bone through an intra-oral surgical process, using a manual tool or contra-angle tool, always observing torque control, which should not exceed 20N.cm.

5. OPTIONS, CONSUMABLE MATERIALS AND SUPPORT MATERIALS

Options: Not applicable

Consumables: Not applicable

Support materials: Instructions for Use

Manufacturing items for exclusive use in the company's products: All parts described in this Instruction for Use are for exclusive use in the Implant Orthodontic Implacil product.

Attention: The use of any part, accessory or material not specified or foreseen in these Instructions for Use is the sole responsibility of the user.

6. INSTRUCTIONS FOR USE OF THE PRODUCT

Insertion:

1. The Implacil Orthodontic Implant / Bracket D/E Orthodontic Implant / ORTH Implant comes packaged inside a blister sealed with Tyvek.
2. Check that the packaging is in perfect condition. Reject the product if the packaging is damaged.
3. Remove the Orthodontic Implant from the blister.
4. The product is sterile.
5. Screw the Orthodontic Implant / Bracket D/E Orthodontic Implant / ORTH Implant. Use 20 N.cm of maximum torque during insertion.

Explantation:

1. Unscrew the Orthodontic Implant / Orthodontic Implant Bracket D/E / ORTH Implant. Use 20 N.cm of maximum torque during removal.
2. Dispose of the Orthodontic Implant / Bracket D/E Orthodontic Implant as indicated in item 7 of this instruction for use.

7. COMPOSITION

As raw material for Implants, Titanium Alloy is used in accordance with ASTM F136.

8. EXPIRY DATE

02 years from the date of manufacture.

9. SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND PACKAGING OF THE PRODUCT:

Store in its original packaging at room temperature, in a cool place, away from light and heat. Avoid excessive exposure to heat and humidity and exert excessive load on the packaging to avoid compromising the seals that guarantee the sterility of the product.

Implants Orthodontic must be transported in a clean, dry place at room temperature. Transport must be done appropriately to avoid falls and damage to its original packaging.

10. WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED

For the safe and effective use of Implacil Orthodontic Implants, we strongly suggest specific training, as the surgical techniques required to place these Implants are highly specialized and complex procedures. Using an improper technique may lead to surgery failure and/or loss of supporting bone or other adverse effects.

A rigorous screening of candidates must be carried out, including the presence of parafunctional habits or diseases that affect healing.

Visual inspection and periapical radiographs are essential for determining anatomical features, occlusion conditions, periodontal status and bone adequacy.

- The critical threshold (temperature/time relationship) for bone tissue necrosis is a temperature of 47°C applied for one minute. At a temperature of 50°C for more than a minute, the critical threshold of non-peri-implant bone regeneration is reached.

- The use of cutting drills, of progressive diameter, with appropriate speed (1500 to 2000 RPM/min), associated with abundant irrigation, avoids excessive thermal rise.

11. PERFORMANCE ON ESSENTIAL REQUIREMENTS OF SAFETY AND EFFECTIVENESS OF THE MEDICAL PRODUCT, AND POSSIBLE UNDESIRABLE SIDE EFFECTS

11.1 - INDICATION, PURPOSE OR USE FOR WHICH THE PRODUCT IS INTENDED

Indication: The Implacil Orthodontic Implant is indicated for temporary fixation to the bone of the maxilla or mandible, providing assistance with orthodontic movements through increased anchorage.

Purpose: Aid in tooth movement through the anchoring of springs, elastics and orthodontic wires. It is a temporary implant for single use, with permanence varying depending on the intended treatment. Removal is done by turning the

orthodontic implant in the opposite direction of installation, with no suturing required after removal.

11.2 - SIDE EFFECTS OR UNDESIRABLE SIDE EFFECTS AND CONTRAINDICATIONS

The surgery to install the Implant Orthodontic Implacil should only be carried out by professionals who have adequate technical training, as it is a highly specialized and complex surgical technique.

Localized edema and mild discomfort may occur as it is a surgical procedure.

Physiological and anatomical conditions can impair the anchorage of the Implant Orthodontic Implacil. This must be taken into consideration when Orthodontic Implants are placed in patients with the following conditions: poor quality oral hygiene; medical conditions such as metabolic or systemic diseases that affect bone or wound healing, such as uncontrolled diabetes and clotting disorders; chemo or radiotherapy treatments; untreatable occlusion or joint problems; cigarette and alcohol abuse and lack of quantity and quality of remaining bone.

Visual inspection and periapical radiographs are essential for determining anatomical features, occlusion conditions, periodontal status and bone adequacy.

The Orthodontic Implant has no contraindication from a tissue point of view.

The maximum Insertion Torque is 20N.cm, and must not exceed this limit.

11.3 - PRODUCT SAFETY AND EFFECTIVENESS:

The Implant Orthodontic Implacil is completely safe, as long as safety rules are followed in all procedures performed. The product was developed with non-toxic medical grade material, made from ASTM F-136 titanium alloy. When maintained (stored) and preserved as mentioned in this Instruction for Use, the product will not lose or alter its physical and dimensional characteristics. The product is sterile and suitable for use for up to 2 years from the date of sterilization.

12. INSTALLATION OR CONNECTION TO OTHER PRODUCTS

The Implant Orthodontic Implacil, to be implanted, needs to be connected to a specific key so that the desired insertion and anchorage can be carried out.

Attention: The use of any part, accessory or material not specified or foreseen in these Instructions for Use is the sole responsibility of the user.

13. PRECAUTIONS IN CASE OF DISPOSAL

To avoid environmental contamination or misuse of the Implant Orthodontic, when it is unusable, it must be disposed of in hospital waste or in a place for potentially contaminated materials.

14. WARRANTY TERM

IMPLACIL OSSTEM - MATERIAL ODONTOLOGICO S.A. guarantees the owner of this product, warranty against any defect in material or manufacturing that appears within 365 days, counting from the date of acquisition by the first purchaser.

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