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Revisão: 07

PRODUCT

Technical Name: Dental Instruments

Commercial Name: SURGICAL INSTRUMENTS IMPLACIL

ATTENTION!!!

Do not use if the packaging is broken or damaged.

Non-Sterile Product. Sterilize before use. Reusable. Clean and sterilize before reuse.

Commercial Models:

DRIVER IMPLANT INSERTER HE 3.5

DRIVER IMPLANT INSERTER HE 4.0

DRIVE ANGLED MINI TAPERED

DRIVER SHORT HAND

DRIVER LONG HEALING ABUTMENT

DRIVER SHORT HEALING ABUTMENT

DRIVER SHORT COVER

DRIVER LONG COVER

DRIVER MANUAL SCALPEL

DRIVER MOTOR SCALPEL

DRIVER AUTOGENOUS GRAFT 1.2

DRIVER SHORT O'RING

DRIVER LONG O'RING

CONE MORSE TUNNEL CHECK FOR PROSTHESIS

MEDIUM RATCHET EXTENDER

LONG RATCHET EXTENDER

F IRRIGATION

IMPLANT RETRIEVER

TÚNEL CHECK IMPLANT STRETTO

PROSTHETIC SCREW RETRIEVER

IMPLANT RETRIEVER 3.5

IMPLANT RETRIEVER 4.0

DRIVER HE 3.5 TO

DRIVER HE 4.0 TO

IMPLANT RETRIEVER HI

IMPLANT RETRIEVER – HE – CM

IMPLANT RETRIEVER – STRETTO

1- INDICATION OF USE AND PURPOSE

Implacil instruments are made of standardized stainless steel, available in various sizes and lengths, and are used to assist in general dental surgeries.

None of the instruments included in this notification are connected to equipment.

2- HOW TO USE

Implacil instruments should be used to assist in dental surgeries. They must be previously sanitized and sterilized using the following techniques:

CLEANING

Cleaning a surgical instrument that has been in contact with tissue should be carried out so that all parts of the item are exposed to the cleaning process.

To clean Implacil instruments, use only the following manual cleaning process.

Automated cleaning has not been validated.

Do not use automated cleaning for Implacil instruments.

1. Initial rinse with cold water. The initial cold rinse is performed to remove any gross contamination present on the item. Cold water is used because temperatures above 60°C/140°F will coagulate proteins, making them difficult to remove from contaminated items.

2. Prepare the enzymatic detergent according to the recommendation*.

3. Immerse the products in the prepared detergent solution and let them soak for at least 5 minutes. While soaking, brush the items with a soft-bristled brush.

4. Rinse: After removing the instruments from the detergent solution, rinse them thoroughly at least 3 times (for at least 1 minute) under running water.

5. Visually inspect each product for visible soiling. If any visible soiling remains, repeat the cleaning process (steps 3 and 4) until no more visible soiling remains.

*Recommended for the cleaning step

Always use distilled, deionized, or demineralized water for this procedure.

Never use saline solutions, especially sodium hypochlorite and saline solution, disinfectants, hydrogen peroxide, or alcohol to clean or rinse instruments.

Never use steel wool, scouring pads, or abrasive products to avoid damaging the instruments. Cleaning: Enzymatic Detergent

Composition: Protease, Lipase, and Amylase enzymes

Dilution: 1 mL for each liter of water;

Action time: 5 min (Manual);

Water temperature: 40°C to 55°C (Manual);

Note: The cleaning process must be conducted so that all parts of the surgical instrument are exposed. Contamination remaining on an item after cleaning reduces the effectiveness of any subsequent decontamination or sterilization process.

Clean instruments as soon as possible after use. Do not allow blood and debris to dry on the instruments. If cleaning needs to be delayed, place groups of instruments in a covered container with an enzymatic solution or appropriate detergent to delay drying.

STERILIZATION

This product must be properly sanitized and sterilized before use.

Please use only steam sterilization according to the following parameters:

Sterilization Parameters

Temperature: 132°C
Sterilization time: 20 min
Drying time: 20 - 60 min

3. FORMS OF COMMERCIAL PRESENTATION OF THE PRODUCT

Surgical Instruments Implacil are presented in individual packaging.

The primary packaging consists of a blister made of virgin PET plastic and sealed with Tyvek.

List of items contained in the package

- 01 Surgical Instrument

These instructions for use will be available electronically on the Implacil website (www.implacil.com.br) in accordance with current legislation regarding the provision of instructions for use in non-printed format for healthcare products – as per RDC No. 751/2022. The printed instructions for use will be available to the customer.

upon request, and will be sent free of charge. Printed instructions for use can be requested through Customer Service at 0800.779.0011, and will be sent at no additional cost.

To locate them, check the "IFU" number and the version of the instructions indicated on the label of the purchased product. To read the instructions for use on the website, Adobe Acrobat Reader is required.

User Alert: Always observe the correlation with the version of the instructions for use indicated on the label of the purchased product, as provided by Implacil.

4. PHYSICAL PRINCIPLE AND FUNDAMENTALS OF THE PRODUCT'S TECHNOLOGY, APPLIED TO ITS OPERATION AND ACTION:

The operation of each of the different types of non-articulated, non-cutting dental instruments mentioned here is based on the specific needs of the surgical procedure. The design of each instrument focuses on the best efficiency, ergonomics, and product resistance. A variety of instruments is necessary due to the wide range of conditions encountered in each surgery, providing greater safety for both the surgeon and the patient.

5. COMPOSITION

The raw material for surgical instruments is ASTM F899 440C stainless steel.

6. VALIDITY PERIOD

5 years from the date of manufacture.

7. SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND HANDLING OF THE PRODUCT:

Keep in its original packaging in a place protected from direct sunlight and rain.

The product in question is not heat-sensitive and does not exhibit any variation in environments with high or low humidity levels, which is why it can be sterilized in an autoclave. Even so, it is recommended that the product be stored between 15° and 35° C and at a humidity level between 45% and 80% to ensure packaging stability. Instruments must be transported in a clean, dry place at room temperature. Transportation must be carried out properly to prevent drops and damage to their original packaging.

Handling must be done with due care to prevent drops and damage, and should be done on shelves and counters, according to the activity being performed.

8. WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED

Caution: The use of any part, accessory, or material not specified or provided for is the sole responsibility of the user.

Implacil instruments must be used only by specialized professionals. A plan for the surgical procedure or other type of procedure must be developed to avoid future complications before any intervention on the patient.

The utmost care must be taken during cleaning, sterilization, storage, transportation, and handling of the product to avoid mechanical shocks that alter its anatomy and impair its functionality. Before beginning surgery, the instrument's functionality must be checked. If the product presents anatomical alterations, it should not be used.

Use extreme caution when handling the instruments to avoid injury. The instrument sets are not sterile and must be autoclaved in appropriate cases or envelopes, according to the instructions for use. Instrument cases will not maintain sterility if not packaged in autoclavable envelopes.

9. CONTRAINDICATIONS

Not applicable

10. ADVERSE EFFECTS

Not applicable

11. PRECAUTIONS IN CASE OF DISPOSAL

To avoid environmental contamination or misuse of SURGICAL INSTRUMENTS IMPLACIL, when they are no longer used, they must be disposed of in hospital waste or in a location for potentially contaminated materials.

12. WARRANTY TERMS

IMPLACIL OSSTEM - MATERIAL ODONTOLOGICO SA guarantees the owner of this product a guarantee against any material or manufacturing defect that may arise within 365 days, counting from the date of purchase by the first purchaser.

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