PORTUGUÊS TRANSFER DIGITAL BLACK

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ESPAÑHOL TRANSFER DIGITAL BLACK

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Revisão: 00

PRODUCT

Technical Name: IMPRESSION TRANSFER

Commercial Name: TRANSFER DIGITAL BLACK

ATTENTION!!!

Do not use if the packaging is broken or damaged.

Non-Sterile Product. Sterilization is required before use.

Product subject to reprocessing.

Commercial Models:

1074810MNC - TRANSFER DIGITAL BLACK MINI CONICO R
1073510MCC - TRANSFER DIGITAL BLACK MICRO CONICO R
1074810CEAR - TRANSFER DIGITAL BLACK CONICO ESTETICO AR
1074810CER - TRANSFER DIGITAL BLACK CONICO ESTETICO R
1073510ID34 - TRANSFER DIGITAL BLACK IDEALE 3.3X4 AR
1073510ID36 - TRANSFER DIGITAL BLACK IDEALE 3.3X6 AR
1073510ID44 - TRANSFER DIGITAL BLACK IDEALE 4.5X4 AR
1073510ID46 - TRANSFER DIGITAL BLACK IDEALE 4.5X6 AR
1074010BT35 - TRANSFER DIGITAL BLACK BASE T 3.5 AR
1074010BT45 - TRANSFER DIGITAL BLACK BASE T 4.5 AR
1073010PS35 - TRANSFER DIGITAL BLACK PILAR STRETTO 3.5 AR
1073010PS48 - TRANSFER DIGITAL BLACK PILAR STRETTO 4.8 AR
1073906SC35 - SCANCORP BLACK 3.5
1074306SC40 - SCANCORP BLACK 4.0
1074811MNC - TRANSFER DIGITAL BLACK MNC – MU INL LAB
1074811MCC - TRANSFER DIGITAL BLACK MCC – MB INL LAB

Accessories:

2033510 - CLIP BLACK TRANSFER DIGITAL SMALL 10MM
2033515 - CLIP BLACK TRANSFER DIGITAL SMALL 15MM
2033520 - CLIP BLACK TRANSFER DIGITAL SMALL 20MM
2034810 - CLIP BLACK TRANSFER DIGITAL LARGE 10MM
2034815 - CLIP BLACK TRANSFER DIGITAL LARGE 15MM
2034820 - CLIP BLACK TRANSFER DIGITAL LARGE 20MM

1- INDICATION OF USE AND PURPOSE

The TRANSFER DIGITAL should be used during the model scan to allow alignment with the original library, indicating the correct position of the analog/implant.

The Clip accessory should be used to aid in scanning accuracy along with the Digital Transfer, which facilitates analysis of the upper and lower arches without the presence of soft tissue.

2- HOW TO USE

- Pre-select the transfer pad appropriate for the component to be used, and the clip, when necessary.
- The Implacil transfer digital pad and clip are packaged in a blister sealed with Tyvek.

- Check that the packaging is in perfect condition. Discard the product if the packaging is damaged.
- Check that it is perfectly sealed.
- Remove the transfer pad from the blister.
- The transfer pad has been previously sanitized in a classified room. If you wish to sanitize it again, follow these steps:
 - Initial rinse of the item with cold water.
 - Wash with a neutral to slightly basic detergent.
 - Final rinse with deionized or reverse osmosis-purified water.
- The product is not sterile, and this procedure must be performed by a professional following these steps:
 - Sterilization temperature: 132°C, Sterilization cycle: 20 minutes, and Drying cycle: 20–60 minutes. The items must be arranged inside the container (perforated in the lid and bottom), or in a tray, box, or surgical envelope; they must be arranged so they are ready for the sterilization process. DO NOT open the autoclave to avoid rapid condensation; that is, let all the steam escape first, allowing the Drying Cycle to complete itself. Thoroughly and periodically clean the autoclave.
 - Clean the clips in the same way as the transfer materials.

Step 1: Define the CAD-CAM transfer material to be used according to the characteristics of the implant or prosthetic abutment;

Step 2: Screw the transfer material onto the implant or prosthetic abutment; Step 3: Scan the intraoral transfer or model.

For planning in the software, check the library's compatibility with our files.

STL files or corresponding files in other planning software, according to the codes described for CadCam use, available in the instructions for use.

The Clip accessory, when necessary, must be connected to the Transfer Digital to scan the dental arch.

3.FORMS OF COMMERCIAL PRESENTATION OF THE PRODUCT

Digital Transfers are presented in individual packaging.

The primary packaging consists of a blister made of virgin PET plastic and sealed with Tyvek.

List of items contained in the packaging

- 1 Transfer
- 3 Traceability Labels

Clips are presented in individual packaging and sold separately.

The primary packaging consists of a blister made of virgin PET plastic and sealed with Tyvek. List of items contained in the package

- 1 Clip

This instruction for use will be available in electronic format on the Implacil website (www.implacil.com.br) in accordance with current legislation on the provision of instructions for use in non-printed format for health products – RDC No. 751/2022. The printed instruction for use will be available to the customer who requests it and will be sent free of charge. The printed instruction for use can be requested through the Customer Service – 0800.779.0011, and will be sent at no additional cost. To locate it, check the “IFU” number and the version of the instruction that is indicated on the label of the product purchased. To read the instructions for use on the website, the Adobe Acrobat Reader program is required. User Alert: Always observe the correlation of the version of the instructions for use indicated on the label of the product purchased, as made available by Implacil.

4. PHYSICAL PRINCIPLE AND FUNDAMENTALS OF THE PRODUCT'S TECHNOLOGY, APPLIED TO ITS OPERATION AND ACTION:

Its operation is based on reproducing the positioning of the implant or prosthetic abutment in the patient's mouth on the model that will be processed in the office or laboratory. Use for intraoral or benchtop scanning in the laboratory. The Clip aids in the accuracy of scanning the dental arch.

5. COMPOSITION

The raw material for the TRANSFER DIGITAL BLACK is the case made of PSU. For the transfers and clip accessories, we have:

- TRANSFER DIGITAL BLACK: TECAPEEK MT
- Accompanying screw: Titanium F136 Gr5
- SCANCORP BLACK 3.5: TECAPEEK MT
- CLIP BLACK: TECAPEEK MT

6. VALIDITY PERIOD

Undetermined

7. SPECIAL CONDITIONS FOR TRANSPORTATION, STORAGE AND HANDLING OF THE PRODUCT:

Keep in its original packaging in a place protected from direct sunlight and rain.

The product is not heat-sensitive and does not exhibit any variation in environments with high or low humidity levels, which is why it can be sterilized in an autoclave. Even so, it is recommended that the product be stored between 15° and 35° C and at a humidity level between 45% and 80% to ensure packaging stability.

The Transfers and Clips should be transported in a clean, dry place at room temperature. Transport must be carried out properly to prevent drops and damage to their original packaging.

Handling must be done with due care to avoid drops and damage. This should be done on shelves and counters, depending on the activity being performed.

8. WARNINGS AND/OR PRECAUTIONS TO BE ADOPTED

The Implacil Digital Transfers and Clips should only be used by specialized professionals. A prosthetic installation plan must be developed, taking into account the diameter, angle, and length for adapting the prosthetic component to the implant to avoid future complications. The product is not sterile, and these procedures must be performed by a professional before any intervention on the patient.

The Implacil Digital Transfers and Clips are designed for single use only, and reprocessing is not recommended. Do not use them under any circumstances if the packaging is broken.

The professional must inform the patient of the need for periodic monitoring to avoid physical and mechanical stress to avoid subjecting the product to undue stress, as well as the need for proper cleaning.

9. CONTRAINDICATIONS

Not applicable

10. ADVERSE EFFECTS

Not applicable

11. PRECAUTIONS IN CASE OF DISPOSAL

To avoid environmental contamination or misuse of TRANSFER DIGITAL BLACK, when they are no longer used, they must be disposed of in hospital waste or in a location for potentially contaminated materials.

12. WARRANTY TERMS

IMPLACIL OSSTEM - MATERIAL ODONTOLOGICO SA guarantees the owner of this product a guarantee against any material or manufacturing defect that may arise within 365 days, counting from the date of purchase by the first purchaser.

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